

Ivory[®]

Rubber Dam Clamp Stainless Steel

Heraeus

Heraeus Kulzer, LLC
300 Heraeus Way
South Bend, IN 46614-2517
1-800-431-1785
www.heraeus-kulzer-us.com

Made in USA

55370104 R 11/09

For use by dental professionals only.

Inspect and sterilize clamp before each use. For optimum use, use a general-purpose cleaner prior to sterilization. Do not heat clamp above 350°F/177°C.

Choose the appropriate size and style clamp giving consideration to the tooth size, location, gingival architecture, and matrix needs. It is recommended that correct rubber dam technique be utilized at all times. Attach a floss safety line to bow or clamp jaws. Set clamp into rubber dam, mount on rubber dam frame, and tie safety line to the frame. Place rubber dam assembly onto tooth in one step, double check stability of clamp before proceeding.

Caution: Modification, over-extending, bending, or extended use of the clamp may cause breakage.

For Technical Assistance, call 1-800-431-1785