

# VacuKleen *E*<sup>2</sup>™

## Effervescent & Enzymatic Evacuation Cleaner

Add contents to one gallon (approx. 4 liters) of warm water. Powder will quickly dissolve.

Aspirate through evacuation hose.

**Draw one quart (approx. 1 liter) of solution through each hose.**

**ONE ENVELOPE MAKES ONE GALLON  
(APPROX. 4 LITERS) OF SOLUTION**

Net Wt. 28 g

**NOT FOR  
HUMAN CONSUMPTION**

**Heraeus  
Kulzer**

Partnership First.

Patent Pending  
**1-800-431-1785**

Heraeus Kulzer, Inc.  
99 Business Park Dr.  
Armonk, N.Y. 10504  
MADE IN U.S.A.  
53610212 R1/03