 **Manufacturer**
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

EC **REP**

SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

03:2019-03
300030281

EN

READ CAREFULLY BEFORE USE

CompoSite and CompoSite Fine polishers will polish and final polish composite restorations without scratching surrounding tooth structure. Use CompoSite for finishing and pre-polishing and CompoSite Fine for polishing.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Use CompoSite FG with lower speed airmotor handpiece. High speed air turbine units can break cups or points.
- Read all instructions on the back side carefully before use.

STERILIZATION

Perform autoclave under the conditions described in Table B.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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(CAUTIONS)

●Never exceed the maximum permissible rotation speeds. ●Use CompoSite FG with lower speed airmotor handpiece. High speed air turbine units can break cups or points. ●Insert the shank completely into the chuck of the handpiece. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use. ●Perform test run to check safety and smooth operation without run-out prior to use. ●Avoid any use at acute angles or with excessive pressure. ●Use local dust extractor and protective equipment while using. ●Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. ●This product is intended for use by dental professionals only. ●Do not use this product for any purposes other than the intended use.

(STERILIZATION)

●Perform autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F) for sterilization. ●When using intraorally, sterilize before each use. ●For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize. ●Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. ●Do not perform dry heat sterilization. ●Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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