# BEAUTIFIL Flow Plus Tips

**Manufacturer**  
SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashisone, Kyoto 605-0893,

**CE 0123**

EN

READ CAREFULLY BEFORE USE

## Fluoride Releasing Flowable Restorative Material

BEAUTIFIL Flow Plus is a light-cured radiopaque flowable restorative material indicated for anterior and posterior restoration. Two different flowabilities, F00 and F03, are available. F00 exhibits minimal flowability, while F03 exhibits slight flowability.

### SHADES

F00:A1, A2, A3, A3.5, A4, B1, B2, C2, D2, A10, A20/  
SA0.5, SA0.50, Inc (Incisal), BW (Bleaching White)

F03:A1, A2, A3, A3.5, A4, B1, B2, C2, D2, A20, A30/  
SA0.5, Inc, BW, CV (Cervical), MI (Milky)

**COMPOSITION**  
Bis-GMA, TEGDMA, S-PRG filler based on fluoroboroaluminosilicate glass, Polymerization initiator, Pigments and others

### INDICATIONS

➤ Anterior and posterior restorations.  
➤ Repairs of restorations and prostheses.

### PRECAUTIONS

1. Do not use this product on patients with known allergies to this material and/or methacrylate monomer.  
2. Operators with known allergies to this material and/or methacrylate monomer must not use this product.

3. Do not re-use tips.

4. If any inflammation or other allergic reactions occur on either patient or operator, immediately discontinue use and seek medical advice.

5. Use medical (dental) gloves and protective glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. Medical (dental) gloves provide some protection against direct contact with this material; however some monomers can penetrate through the gloves very quickly to reach the skin. Therefore, should this material contact even the gloves, immediately remove them and wash hands with plenty of water.

6. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.

7. This product is intended for use by dental professionals only.

### DIRECTIONS FOR USE

[Anterior and posterior restorations]

1. Cleaning of tooth surface  
Thoroughly clean the tooth surface to remove plaque.  
2. Shade selection  
Select appropriate shade using a shade guide while the tooth is still moist.

3. Cavity preparation  
Prepare the cavity following the conventional method.

4. Isolation  
Use a rubber dam etc to isolate the treatment area from contamination.

5. Pulp protection  
In the case where the pulp is exposed or the affected area is close to the pulp, cap the pulp with calcium hydroxide etc.

6. Adhesive application  
Apply dentin adhesive following the manufacturer's instructions for use.  
7. Delivery  
Insert a BEAUTIFIL Flow Plus Tip into the dispenser.  
➤ When applying the material directly into the cavity, sterilize the dispenser before and after use.  
8. Placement  
Apply necessary amount of material directly into the cavity and create the shape desired. BEAUTIFIL Flow Plus Tip may also be extruded onto a paper pad to be applied into the cavity with a suitable instrument.  
➤ When storing these materials in a refrigerator, allow them to reach room temperature before use.  
➤ This product is not for the zahnärztlichen Gebrauch vorgesehen.

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9. Finishing and polishing  
After light-curing, contour, finish and polish the restoration following the conventional method.

**[Repair of restorations and prostheses]**

1. Cleaning of the restoration  
Thoroughly clean the restoration to remove plaque.

2. Shade selection  
Select appropriate shade using a shade guide.

3. Isolation  
Use a rubber dam etc to isolate the treatment area from contamination.

4. Pretreatment of the material to be bonded  
Perform appropriate pretreatment for each material to be bonded (composite resin, porcelain, metal or natural tooth).

5. Follow the instructions for use of each adhesive.

6. Follow the steps 7-10 in [Anterior and posterior restorations] as described above.

**NOTE ON USE**

1. Do not use eugenol containing materials since they might inhibit polymerization or adhesion of the material.

2. Follow the instructions for use of each dental material, equipment or instrument to be used in conjunction with this product.

3. Do not use this product for any purposes other than specifically outlined in the INDICATIONS in these instructions for use.

4. Use this product within the expiration date indicated on the package and container.  
(Example ☐ YYYY-MM-DD → Year-Month-Date of the expiration date)

**STORAGE**

Store in a dark place at room temperature (10 °C/34-86 °F). Avoid high temperatures and high humidity. Keep away from direct sunlight and any source of ignition.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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9. Finishing and polishing

After light-curing, contour, finish and polish the restoration following the conventional method.

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1. Cleaning of the restoration

Thoroughly clean the restoration to remove plaque.

2. Shade selection

Select appropriate shade using a shade guide.

3. Isolation

Use a rubber dam etc to isolate the treatment area from contamination.

4. Pretreatment of the material to be bonded

Perform appropriate pretreatment for each material to be bonded (composite resin, porcelain, metal or natural tooth).

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6. Follow the steps 7-10 in [Front- and Seitenzahnfüllungen] as described above.

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F03: A1, A2, A3, A5, A4, B1, B2, C2, D2, A20, A30/ SA0,5, Inc, BW, CV (Cervical), MI (Milky)