



ENGLISH

DirectDia is a diamond impregnated polishing paste used both intra and extra orally for achieving a high luster polish on both the restoration and natural tooth surface.

INDICATIONS

1. High luster polishing of restorative materials (porcelain, direct composite, lab composite, metals etc.)
2. Polishing and cleaning of natural teeth

DIRECTIONS FOR USE

1. Remove cap from syringe and attach the delivery tip.
2. Apply a small quantity of paste to the Super-Snap Buff Disk (sold separately).
3. A dry surface is not required. Use on a saliva moist surface is acceptable.
4. Rinse thoroughly with water to remove all paste.

CAUTIONS AND RECOMMENDATIONS

- Finish the surface with silicone polisher before using.
- Attach the delivery tip to syringe securely and tightly.
- Use with Super-Snap Buff Disks - standard or mini (Shofu PN L523 or L524).
- To prevent splatter and avoid excessive heat generation always use rotary instruments at speeds below 10,000 r.p.m.
- The quality of the product is not affected if liquid separation and pale color by liquid separation occurs. If the paste hardens it can be adjusted by adding water.
- Store in a dark place away from direct sunlight.
- Store at temperatures between 1°C and 30°C.
- Replace cap tightly after use to prevent drying.
- Wearing protective glasses is recommended for patient and operator.

PACKAGE

DirectDia Paste: 1 syringe, 3g
Tip with cap: 2 each