

Gold Anodized Crowns

Indications:

- Short-term provisional coverage
- Premolars and molars

Tooth Preparation:

- Prepare tooth and determine mesio-distal width.
- Select appropriate crown and trial fit.

Crown Preparation:

- Seat crown on preparation. If gingival blanching occurs, trim crown using a 3M™ ESPE™ Crown Scissors. Reseat in mouth and establish occlusion.



1

- Contour crown using a 3M™ ESPE™ Contouring Pliers.



2

- Crimp crown margin using a 3M™ ESPE™ Crimping Pliers. Reseat and check fit.



3

- Outline margin with pencil and trim crown back to this line using scissors.



4

Gold Anodized Crowns



- Smooth and polish, as required, using 3M™ ESPE™ Sof-Lex™ Finishing and Polishing Discs, stone, and rubber wheel.



Crown Cementation:

- Mix cement and fill crown, assuring voids are not present in cement.

Note: For custom margins and extended durability, line crown with acrylic or resin material.



- Seat crown and allow the cement to set.



- Remove excess cement with scaler or explorer. Floss interproximal areas.