Endodontic Contra Angle Handpiece

M4 Safety Handpiece

OPERATION MANUAL

: Autoclavable

: Thermo-Disinfectable (M-type)

TABLE OF CONTENTS

ENGLISH	3
FRANÇAIS	6
ESPAÑOL	9
DEUTSCH	12
ITALIANO	15
DANSK	18
PORTUGUÊS	21
NEDERLANDS	24
ΕΛΛΗΝΙΚΑ	27
SVENSKA	30
HRVATSKI	33
ČESKÝ	36
MAGYAR	39
LIETUVIU	42
POLSKI	45
ROMÂNEȘTE	48
РУССКИЙ	51
SLOVENSKÝ	54
SPRSKI	57
TÜRKÇE	60
中文版	63
日本語	66
한국어	69
عربي	72

M4 Safety Handpiece

OPERATION MANUAL

: AUTOCLAVABLE: Thermo-Disinfectable (M-type)

Please read this Operation Manual carefully and file for future reference.

“Warning: Electrical micromotors generate significantly more power than traditional air turbines and air motors. Poorly maintained, worn, damaged, or misused handpieces may generate frictional heat capable of causing serious burn injuries to the patient. Heat generation may occur quickly and without prior warning. Handpieces must be properly maintained according to the specified maintenance schedule and inspected for signs of wear prior to each use.”

Caution

- For use by qualified clinicians only.
- If you experience any problems, stop immediately. Do not use and contact SybronEndo.
- Do not drop the unit onto a hard surface.
- Disconnect the sheath only after the driver unit has completely stopped.
- Do not wipe or clean with high acid water or sterilizing solutions.
- When operating the handpiece always consider the safety of the patient.
- Do not use non-ISO standard files.
- Do not quickly heat nor quickly cool after autoclaving.
- Do not disassemble nor alter the handpiece.
- Always watch for vibration, sound, temperature rise, bur wobble and other abnormalities.
- Do not exceed 10,000 rpm on motor 1:1 setting.
- Not for use with air-driven motors. Use only electric motors.

DIRECTIONS FOR USE

The M4 Safety Handpiece, designed for the mechanical preparation of root canals, features a simplified chuck mechanism permitting one handed insertion of manual endodontic instruments and precludes the necessity of handling the sterilized instrument. It is recommended that clinicians use the M4 Safety Handpiece on plastic endo blocks or extracted teeth before adopting the handpiece into standard use.

USE

The M4 Safety Handpiece is designed to effectively and safely instrument canals. The chuck oscillates 30 degrees in an efficient, watch-winding motion. The gear reduction ratio is 4:1. The M4 Safety Handpiece accommodates instruments with a maximum handle diameter of 4mm. It can be connected to E-type motors, and is in accordance with DIN 13.940 and ISO DIS 3964. A spray cooling attachment is not required.

1. ATTACHING AND REMOVING FILE

(1) Attaching

- Press the push button to open the chuck. (Fig.1)
- Insert the instrument handle into the chuck (Fig.2) and release the button.

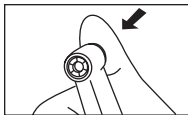
Fig. 1

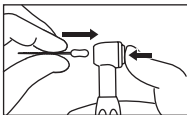
Fig. 2

Caution

- Make sure the file is mounted firmly by pulling before use. (Fig.3)

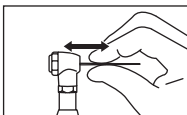
Fig. 3

(2) Removing

- Press the push button to remove the file.

2. SPEED

Allowable Max. Motor Speed.....10,000 min⁻¹ (rpm)

3. LUBRICATION

Lubricate with PANA-SPRAY Plus after each use and/or before autoclaving.

1. Push E-type spray nozzle attachment over the PANA-SPRAY Plus nozzle until it seats firmly.
2. Shake the can 3-4 times to mix lubricant and propellant well.
3. Insert the E-type spray nozzle into the rear of the handpiece and spray for approximately 2-3 seconds until the oil comes out of the handpiece head or nose.

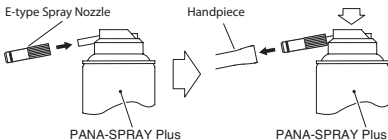
Fig.4

Cautions on Lubrication

- Be sure to hold the handpiece firmly to prevent the handpiece from slipping out during the lubrication.
- Keep the PANA-SPRAY Plus can upright.

4. CLEANING AND STERILIZATION

 Sterilization by autoclaving is recommended.

Sterilization required after each patient use as noted below.

- **Autoclaving procedure**

1. Brush off dirt and wipe clean with alcohol-soaked cloth or paper towel. Do not use a wire brush.
2. Remove reduction sheath from the motor and lubricate. Refer to Section 3. Lubrication.
3. Insert into a sterilization bag and seal it.
4. Autoclavable up to a max. 135° C. ex.) Autoclave for 20 min. at 121° C, or 15 min. at 132° C.
5. Keep the handpiece in the autoclave pouch to keep it clean until next use.

* Sterilization at 121° C for more than 15 minutes is recommended by EN13060 or EN ISO17665-1.

* Performing sterilization according to our instruction has minimal effect on the instrument. Life span is generally determined by wear and damage due to use.

Cautions on Sterilization

- Skip dry cycle.
- Heating the unit at the bottom of the chamber, may exceed the set value of the unit.
- Place the handpiece on the central or upper tray.
- Do not autoclave the handpiece, when it is bagged together with other instruments. This helps avoid discoloration and/or damage the plating of the handpiece.

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Kerr Italia S.r.l.,