PROTEMP® II TECHNIQUE



STEP 1:

Before preparing teeth, make impression and trim after set.



STEP 2:

Cut interdental grooves out of impression.



STEP 3:

If edentulous gap exists, cut grooves between abutment teeth.



STEP 4:

After teeth have been prepared, cleaned and dried; mix equal amounts of base/catalyst and place into syringe. Gently extrude keeping syringe tip immersed in material.



STEP 5:

Gently position impression matrix into patient's mouth.



STEP 6:

Monitor set of material in mouth and remove between 2 and 3 minutes after mixing.



STEP 7:

Remove temporary restoration and allow it to bench set for approximately 2 minutes.



STEP 8:

Trim and finish temporary restoration.



STEP 9:

Cement temporary restoration using standard temporary cement.

PROTEMP® II

elf-curing, bis-acryl composite provisional material with audible-click base/catalyst delivery system for direct chairside temporary restorations.

INDICATIONS

- Multiple-unit bridges
- Full and partial crowns
- Inlays and onlays
- Veneers

ADVANTAGES

- Easy-to-extrude, audible-click base/catalyst delivery system
- Exceptionally tight marginal seal with precision fit
- Secure removal from mouth without tearing, distortion or sticking
- No unpleasant odor or taste
- Reliable, long-lasting and wear resistant restorations with fewer repairs
- Outstanding aesthetics and polishability with great color stability
- Available in a range of three Vita shades: A1 extra light/A3 light/B3 yellow

*Vita is a registered trademark of Vita Zahnfabrik

TO PLACE AN ORDER, CALL YOUR AUTHORIZED ESPE DISTRIBUTOR OR CALL ESPE AT 1-800-344-8235.



PACKAGING

Type of Package:	Item Number
PROTEMP II Bis-Acryl Provisional Material	
INTRO PACK A1 EXTRA LIGHT /A3 LIGHT - (1) Base (28g) each - A1 and A3,	
(1) Catalyst (4.8g), (1) Pad, (1) Syringe	
INTRO PACK A3 LIGHT/B3 YELLOW - (1) Base (28g) each - A3 and B3,	
(1) Catalyst (4.8g), (1) Pad, (1) Syringe	
DOUBLE PACK A1 EXTRA LIGHT - (2) Base A1 (28g), (1) Catalyst (4.8g),	
(1) Pad, (1) Syringe	
DOUBLE PACK A3 LIGHT - (2) Base A3 (28g), (1) Catalyst (4.8g),	
(1) Pad, (1) Syringe	
DOUBLE PACK B3 YELLOW - (2) Base B3 (28g), (1) Catalyst (4.8g),	
(1) Pad, (1) Syringe	
GIANT PACK A1 EXTRA LIGHT - (10) Base A1 (28g), (5) Catalysts (4.8g),	
(5) Pads, (5) Syringes	
GIANT PACK A3 LIGHT - (10) Base A3 (28g), (5) Catalysts (4.8g),	
(5) Pads, (5) Syringes	
GIANT PACK B3 YELLOW - (10) Base B3 (28g), (5) Catalysts (4.8g),	
(5) Pads, (5) Syringes	
(1) CATALYST (4.8g)	
(5) Syringes	

Working Time (including mixing time): 2 minutes Setting Time (from start of mix): 7 minutes