




Protect from light
Store in cool environment
(Room temperature or cooler)




The Aseptico HandiDam is a single-use product. Discard after one use.




This product contains latex rubber, which may cause allergic reactions.




Manufacturer



Aseptico Inc
8333 216 St SE
Woodinville
WA 98072
USA



Advena Ltd
Tower Business Centre, 2nd Flr,
Tower Street, Swatara, BKR 4013
Malta



EC REP

HandiDam® is manufactured exclusively by Aseptico. It saves time, aids in infection control, and preserves patient goodwill.

HandiDam®

Pre-framed Rubber Dam



Easy to use, convenient,
and saves time.

Instructions for Use



1. Insert the stick through the holes as pictured.
(Use of stick is optional)



2. The dam should be curved inward as shown




3. Punch the hole for your dam clamp.



4. Place your clamp through the hole as shown



5. Flip the loose side of the dam back for better access.

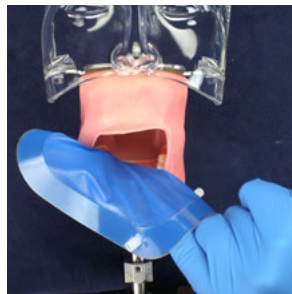

6. Pull on the stick for easy access to mouth for suctioning or inserting radiograph film.

Autoclave sticks and save for re-use.

Sterilization Procedure:	Sterilize the plastic sticks between each patient use. Steam autoclave at 132° C (275° F) for 3 minutes.
---------------------------------	---

Package Contents:

<u>Item No.</u>	<u>Dams Qty.</u>	<u>Sticks Qty.</u>
I-70BM	20 dams	5 sticks
I-70BM-100	100 dams	20 sticks