SeptoPlus Mandrel / SeptoDiscs Mandrel

Accessory device for fixing different finishing and/or polishing systems in contra-angle. Available in two systems: mandrel by pressure (SeptoPlus Mandrel) and pop-on mandrel (SeptoDiscs Mandrel).

Use: Install the mandrel at the contra-angle. Fit the disc onto the mandrel. SeptoDiscs Mandrel: Compatible with SeptoPolisher and SeptoWheels PX. SeptoDiscs Mandrel: Compatible with SeptoDiscs.

Warnings, precautions and restrictions: Professional use only. The user will be entirely responsible for the mandrel or the disc to the disc with similar products. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operatory field. If there is proven allergy to any of its components, the product must not be used.

Cleaning and Sterilization: The mandrel is autoclavable (126 ° C, 147 kPa, 16 min). Before sterilization, clean using water and enzymatic detergent. The durability of the mandrel is directly related to care during use.

Composition: Metal.

Storage: To be stored in a cool and dry place, protected from sunlight.

Connection with contra-angle: The SeptoPlus Mandrel is manufactured in compliance with specifications of ISO 1797. The SeptoDiscs Mandrel is manufactured in compliance with specifications of ISO 1797.

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Obelis Ltd.
SEPTODONT, Inc
Obelis S.A