



INSTRUCTIONS FOR USE

BACTERIAL RESISTANT DENTAL BOTTLE RESERVOIR

Constructed with silver infused plastic for superior microbiological control and confidence.

FEATURES AND BENEFITS

- Actively suppresses bacterial growth on internal bottle surfaces
- Expanded bottle mouth enables easy removal and replacement of straws
- Molded volumetric indicators for easy water measurement
- Available in 1L and 2L sizes
- Convenient enhancement to Sterisil's effective silver-based disinfection products
- Only Sterisil products are EPA registered to maintain the bottle's antimicrobial efficacy

PRECAUTIONARY STATEMENTS

STORAGE: Store in a cool dry area out of direct sunlight.

CONTAINER HANDLING: Dispose of bottle in trash.

INTENDED USE: FOR USE IN DENTAL DELIVERY UNITS.

IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING.

PRODUCT SPECIFICATIONS

2L Sterisil® Antimicrobial Bottle (BF-B2L)

- Max Pressure: 65psi
- Dimensions: 4" W x 14 23/32" H overall
- Standard 28mm threads
- Suppresses growth of bacteria, algae, fungus, mold and mildew

.7L Sterisil® Antimicrobial Bottle (BF-B)

- Max Pressure: 65psi
- Dimensions: 3" W x 11" H overall
- Standard 28mm threads
- Suppresses growth of bacteria, algae, fungus, mold and mildew

INSTRUCTIONS

1. If replacing another bottle, first turn the dental unit off to release pressurized air
2. Remove existing bottle
3. Feed new bottle over the pickup tube
4. Rotate bottle in counter clockwise direction to screw the bottle into place
5. Tighten firmly
6. Turn dental unit on