

confused about
which sponge
is right
for your practice?

{ Let  Richmond
DENTAL & MEDICAL clear things up }



Richmond Dental & Medical has been a manufacturer of cotton products for over 114 years and our parent company, Barnhardt Manufacturing Company is a manufacturer and provider of bleached cotton for the medical and dental industry worldwide. Because of this vertical integration we have unique knowledge, experience and expertise in the manufacturing and selection criteria of dental and medical sponges. This comprehensive reference guide has been developed at the request of dental professionals to clear up any possible confusion and assist you when making your sponge selection.

ALL GAUZE SPONGES



All Gauze Sponges are layers of cotton, woven into a cheese cloth appearance. Their loose weave allows for a good grip and creates a somewhat abrasive surface ideal for instrument cleaning. These sponges can be used in many applications. These sponges are 8-ply* and 12-ply* and available in 2" x 2", 3" x 3" and 4" x 4".

COTTON FILLED EXODONTIA SPONGES



This variety of sponge is made in the same way as an all gauze sponge, but contains a layer of pure cotton in the middle. This cotton layer provides better absorbency and patient comfort. Traditionally, cotton filled sponges were the dentists first choice. These sponges are 4-ply* and 8-ply* and available in 2" x 2", 3" x 3" and 4" x 4".

STERIPOCKET® COTTON FILLED SPONGES



As hard as dental professionals work, any effort to make life easier and more efficient is always welcome. That's why SteriPocket® Sponges continue to be as popular as they are effective. SteriPocket® Sponges are individual, pre-sterilized packets of 2 cotton filled (exodontia sponge) sponges. Easy, economical and professional, they are always ready when you need them. Perfect for dispensing to patients after extractions and oral surgeries. Available in 2" x 2" only.

RAYON/POLY NONWOVEN SPONGES



Folded layers of rayon and polyester fabric are used to make this product. The fabric is lint free and will not leave fibers in the dental work, nor will they leave lint residue on instruments. They are the economical choice in the nonwoven category. These sponges are 4-ply* and available in 2" x 2", 3" x 3" and 4" x 4".

100% COTTON NONWOVEN SPONGES



The 100% Cotton nonwoven sponge combines the best possible characteristics of sponges into one product. It provides maximum absorbency with the added benefit of being lint free. The perfect general use sponge. These sponges are 4-ply* and available in 2" x 2", 3" x 3" and 4" x 4".

MULTIPLY® RAYON/POLY NONWOVEN 8-PLY SPONGES

















Eight folded layers of rayon and polyester fabric make these sponges twice as absorbent as traditional gauze, exodontia or 4-ply nonwoven sponges. These sponges have the same appearance and feel of the 4-ply rayon/poly sponges but the additional layers provide thickness perfect for bite pressure. These sponges are 8-ply* and available in 2" x 2", 3" x 3" and 4" x 4".

STERIPOCKET® RAYON/POLY-NONWOVEN 8-PLY SPONGES



As hard as dental professionals work, any effort to make life easier and more efficient is always welcome. That's why SteriPocket® Sponges continue to be as popular as they are effective. SteriPocket® Sponges are individual, pre-sterilized packets of 2 rayon/poly (MultiPly® sponge) nonwoven lint free sponges. Easy, economical and professional, they are always ready when you need them. Perfect for dispensing to patients after extractions and oral surgeries. Available in 2" x 2" and 4" x 4".



	Sponge Ply	Handkerchief Folded	Ideal for Surface & Instrument Cleaning & Disinfecting	Lint-Free	Absorbency	All Natural & Bio-Degradable
ALL GAUZE SPONGES	8 or 12-ply				★	
COTTON FILLED EXODONTIA SPONGES	4 or 8-ply				★★	
STERIPOCKET® COTTON FILLED SPONGES	8-ply				★★	
RAYON/POLY NONWOVEN SPONGES	4-ply				★★	
100% COTTON NONWOVEN SPONGES	4-ply				★★★	
MULTIPLY® RAYON/POLY NONWOVEN 8-PLY SPONGES	8-ply				★★★★	
STERIPOCKET® RAYON/POLY-NONWOVEN 8-PLY SPONGES	8-ply				★★★★	

*Ply: Refers to the number of layers in a sponge after it is folded.

ALL GAUZE SPONGES



COTTON FILLED EXODONTIA SPONGES



STERIPOCKET® COTTON FILLED SPONGES



RAYON/POLY NONWOVEN SPONGES




100% COTTON NONWOVEN SPONGES



MULTIPLY® RAYON/POLY NONWOVEN 8-PLY SPONGES



STERIPOCKET® RAYON/POLY-NONWOVEN 8-PLY SPONGES

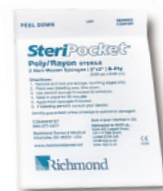

Call Richmond Dental & Medical Customer Service at **1-855-841-0743**,
email us at **RDCustSrv@barnhardt.net** or visit our website at **[www.Rich-](http://www.RichmondDentalandMedical.net)**
[mondDentalandMedical.net](http://www.RichmondDentalandMedical.net) for further information on how
our variety of sponges can contribute to the success of today's
dental professionals.



www.RichmondDentalandMedical.net | 1-855-841-0743

