

# INSTRUCTIONS FOR USE



## Operatory Protection

### Plastic Headrest Covers

#### Instructions for use:

1. Use a surface disinfectant as recommended by chair manufacturer to clean the headrest.
2. Hold cover with opening on the bottom.
3. Place cover over headrest until headrest is completely covered.



Ref.#'s LOCP, LOCPW, L3CP, L3CPW



Do not reuse.



Not made with natural rubber latex.



**CROSSTEX**  
A CANTEL MEDICAL COMPANY



MDSS GmbH  
Schiffgraben 41  
30175 Hannover, Germany



Made in USA

Manufactured by: Crosstex International, Inc.  
10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | [Crosstex@crosstex.com](mailto:Crosstex@crosstex.com)

PLASTICHEADCVR-IFU  
Rev. 0 – 5/17