

Posey® Seizure Side Rail Pads

Application Instructions



Posey Seizure Pads are form fitted to fit snugly over the existing bed side rails with a 1" (3 cm) layer of medium density foam to help safeguard against accidental patient injury. The outer shell is constructed from lightweight sheer nylon denier to help prevent patient "rub" burns and allows unobstructed movement of the side rails to the lowered position. The nylon cover is backed with a urethane waterproof coating that easily wipes clean with liquid disinfectant. Open windows in upper section allow for easy access to controls on most bed models. Navy Blue.



Cat. No. **Description** **Pads**

Hill-Rom® Beds

REF 5725	Fits Advanta™ and Century Plus® beds	4
REF 5729	Fits Advanta™ (Pre- 02/2004), Advance®, Century 8500, and Century CC Beds	4
REF 5742	Fits CareAssist® and Advanta™ 2 Beds	4
REF 5743	Fits VersaCare® Med / Surg Beds	4
REF 5744	Fits Affinity III® and IV® Maternity Beds	2
REF 5746	Fits TotalCare® Med / Surg Beds	4
REF 5747	Fits Resident® LTC Beds	2
REF 5756	Fits Progressa® Beds	4
REF 5760	Fits Hill-Rom Low® Beds	2
REF 5761	Fits Affinity II® Maternity Beds	2

Stryker® Beds

REF 5755	Fits Secure I and II® Beds with Rounded Side Rails	4
REF 5762	Fits Secure I and II® Beds with Rectangular Side Rails	4
REF 5763	Fits GO Bed II® with Rectangular Side Rails	4
REF 5764	Fits In-Touch™ Critical Care Beds	4
REF 5766	Fits S3™ Beds	4

Other Manufacturers


REF 5760	Fits Carroll® Beds with Care Assist Arms	2
REF 5765	Fits Med-Mizer® Beds with Med Support Arms	2

Monitoring

⚠ WARNING Posey Seizure Side Rail Pads will not prevent a determined patient from exiting the bed, going over or attempting to go between or through side rail gaps. Not all hospital beds allow for full articulation when this product is applied. Slowly articulate the head of the bed to ensure compatibility. Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation.

Application Instructions:

1. Inspect product set prior to use. Do not use soiled or damaged products.
2. Identify each of the components in the set. Each pad has sewn-in tags, identifying its intended location from the perspective of lying down in the bed (Fig. 1).
3. Place each of the pads onto the proper side rail and shift it into position (Fig. 2).

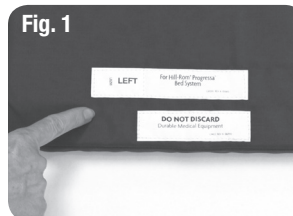

4. Securely fasten all the hook-and-loop fasteners along the edges of each pad. Some pads may have additional hook-and-loop fasteners in their mid-sections, for those pads please see Fig. 3 and Fig. 4.
5. Lower each of the side rails to check clearances. Note that there may be some resistance as the rails are completely lowered. This should be expected as there are close manufacturing tolerances between the bed and the side rail. The shear nylon covers are chosen to minimize any potential binding.
6. Check for proper fit. Modified beds may not be compatible with Posey Seizure Side Rail Pads.

Disposal

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

Storage and Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

Cleaning Instructions:

Wipe clean with soap and water, or disinfectants that DO NOT contain the chemicals Phenol or Benzyl.
DO NOT submerge pads under any liquid solution.



Advance, Advanta, Affinity, CareAssist, Century Plus, Progressa, Resident, TotalCare and VersaCare, are registered trademarks of Hill-Rom® Services, Inc. Med-Mizer is a registered trademark of Med-Mizer Inc. Go-Bed II, In-Touch, Secure, S3 and Stryker are registered trademarks of the Stryker® Corporation.



Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA
Phone: 1.800.447.6739 • 1.626.443.3143 • Fax: 1.800.767.3933 • www.posey.com