

QUILTED LONG BOOT

The Posey Quilted Long Boot offloads the heel and helps protect heels, ankles, and calves from skin irritation. Intended for use by patients who are seated or in bed. The boot helps reduce pressure on the heel by dispersing weight evenly along the calf. The protectors are filled with polyester fiber, covered with a cotton fabric, and have three padded adjustable straps with hook and loop closures. Machine washable and one size fits most applications. One pair per package.



REF 6113 Pastel Plaid, 14"L x 9"H (36 cm x 23 cm)

REF 6114 Solid Navy, 14"L x 9"H (36 cm x 23 cm)

MONITORING

⚠ WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

INSPECTION

⚠ WARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

APPLICATION INSTRUCTIONS

1. Unfasten the hook and loop padded straps.
2. Place the product on the foot so that two straps can be secured around the calf and one strap secured across the toes.
3. Fasten the hook and loop padded straps for the desired fit.

⚠ CAUTION DO NOT allow the patient to walk or ambulate while wearing this product.

WASHING INSTRUCTIONS:

Close hook and loop before washing.



DISPOSAL

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.



6113



6114

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.