

Posey metal bed cradles help keep bed sheets and blankets away from sensitive skin. The multi-piece take apart design reduces storage space. Self-locking clamps lock the cradles in position on the mattress. Designed to fit mattresses 35" to 37" (89 cm to 94 cm) wide. Not for use on air mattresses.



REF 6430 Bed Cradle, standard bed, rectangular, 36"L x 21½"H (91 cm x 55 cm)

REF 6435 Bed Cradle, standard bed, triangular, 36"L x 24"H (91 cm x 61 cm)

REF 6436 Foot Support, poly/cotton cover for 6430, 36"L x 21½"H (91 cm x 55 cm)

REF 6437 Foot Support, poly/cotton cover for 6435, 36"L x 24"H (91 cm x 61 cm)

MONITORING

⚠ WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

INSPECTION

⚠ WARNING Inspect before each use: check for broken stitches or parts; torn, cut or frayed material; or buckles that do not hold securely. DO NOT use soiled or damaged products.

APPLICATION INSTRUCTIONS:

Be sure to follow your facility's policies and protocols regarding frequency of patient monitoring. Posey recommends following Agency for Healthcare Research and Quality guidelines for removing skin coverings at least every two hours to check skin integrity and circulation.

1. Raise the sliding "V" spring clamp and place the lower metal base under the mattress in the desired position.
2. Lower the spring clamp against the mattress to secure the bed cradle. Press the center of clamp down to lock. To release, press down on the inner ends of the clamp fingers (fig. 1).
3. Assemble upper rod support and insert into uprights.
4. Apply Posey Foot Support Cover (6436/6437) to the applicable bed cradle if so desired.

NOTE: The Posey Foot Support (Cat. No. 6436/6437) will not prevent lateral footdrop. Only specific Posey Footdrop Splints (Cat. #'s 6412/6405) should be used to help prevent severe footdrop.

DISPOSAL

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.



Fig. 1



6435



6430



6436 and 6430 Shown with optional Foot Support Cover

LAUNDERING INSTRUCTIONS:

Covers:

- These products can be machine washed under CDC guidelines for material soiled with blood or bodily fluid.
- For non-contaminated material, use lower temperature wash and dry cycles to extend product life.
- Fasten all buckles to reduce risk of damage during wash and dry cycles. DO NOT put buckles through extractors.
- Inspect all buckles and metal parts after drying. Discard if damaged.



CLEANING INSTRUCTIONS:

Metal Frames:

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.

