

PREMIUM ELBOW PROTECTOR

Designed for long term use to help protect and prevent pressure ulcers and skin irritation. The fabric cover allows for air circulation. One size fits all. One per package.

REF 6222 Premium Elbow Protector,
11"L x 9"W (28 cm x 23 cm)

MONITORING

⚠ WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

INSPECTION

⚠ WARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

APPLICATION INSTRUCTIONS:

1. Place patient's elbow into the pocket of the pad. When fastening the hook and loop strap across the arm, be careful not to restrict circulation.

WASHING INSTRUCTIONS:



DISPOSAL

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

FOOT ELEVATOR

Lifts the lower leg completely off the bed to help offload the heel and prevent and treat pressure ulcers. One per package.

REF 6532 Foot Elevator, 4" (10 cm) width-spiral cut
REF 6533 Foot Elevator, 6" (15 cm) width-spiral cut

PRODUCT FITTING INSTRUCTIONS:

1. Measure the patient's ankle circumference.
2. This product is designed to optimally fit patients ankle circumferences measuring between 8" and 12½" (20 cm and 32 cm).

MONITORING

⚠ WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

INSPECTION

⚠ WARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

APPLICATION INSTRUCTIONS:

1. Wrap the liner, towel or washcloth around the ankle.
2. Insert the ankle through the opening in the foam and secure with hook and loop closure strap.

NOTE: To reduce pressure on the ankle and hypertension of the knee, a pillow may be placed under the patient's knee and leg (Fig. 1).

WASHING INSTRUCTIONS:



DISPOSAL

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.