

3.11"
This panel visible when folded

3.11"
This panel visible when folded

3.11"


Wire Bracket 8276 Application Instructions



The 8276 Wire Bracket is suitable for bed mounting. It fits chairs, headboards, and footboards that are ½" – 2" (1 cm - 5 cm) thick (Fig. 1).

TO MOUNT THE 8276 WIRE BRACKET:

1. Slide alarm onto bracket from top down until it is firmly in place (Fig. 2). Choose location on chair, headboard, or footboard where patient cannot reach or tamper with the alarm or connections.
2. Pull bracket wire away from alarm to create an opening wide enough to fit the intended clip point. Slide bracket onto bed and push down to ensure a snug fit. Make sure alarm indicator lights are in clear view of staff.
3. To remove alarm, gently push release lever IN while sliding alarm up and out (Fig. 3).
4. Make sure sensor, nurse call cables, and/or AC adapter cables can be secured out of the way and do not present a tripping hazard.

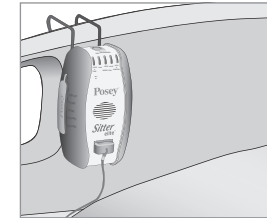

8276 Wire Bracket

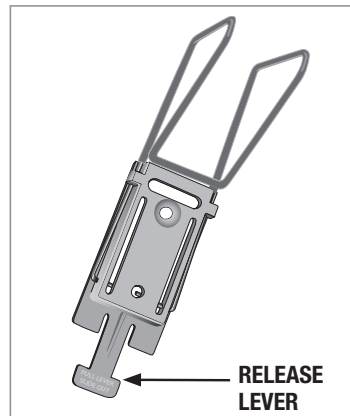


Fig. 1

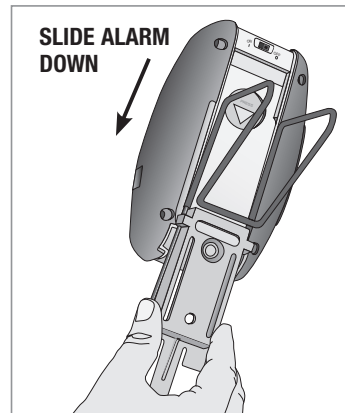


Fig. 2

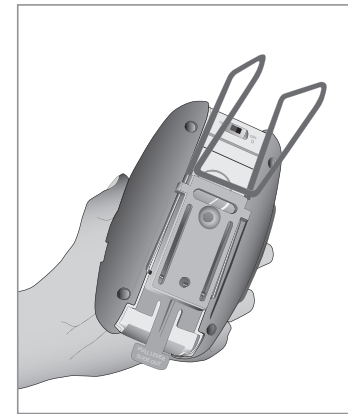


Fig. 3



Posey Products, LLC (Posey) • 5635 Peck Road, Arcadia, CA 91006-0020 USA
Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com

© 2019 Posey Products, LLC. (Posey) All rights reserved.



M7002 REV A 030119

Printer: please note dashed fold lines. The upper left and upper middle sections should be visible when document is fully folded.
Folded size will be ~3.11 x 3.375