



Quick Start Guide



SITTER ELITE

1. Connect AC power cord or insert batteries*
2. Connect sensor
3. Turn alarm on

ADJUST PATIENT WHILE ON SENSOR

Hold: Quick tap of button = 30 second delay

Suspend: Longer tap of button (3 sec) = 5 minute delay

PROGRAMMING

Press indicated button to advance through options. The last option or audible sample selected will be the one programmed for use.

1. **Select MODE:** 4 options (voice & tone, tone, voice, mute)
2. **Select TONE:** 5 options
3. **Select VOLUME:** 3 options (high, medium, low)
4. **Record a Message:** Press and hold RECORD; wait for beep; record message; release button

GREEN flash **GO** / **RED** flash **NO**

*If the alarm low battery alert is chirping, or the alarm does not power up, the batteries are depleted and must be removed.





Sitter Elite Alarm Product Details



WARNING: Before use, make sure to read all alarm and sensor instructions. Test alarm and sensor to verify that each function works properly before leaving a patient unattended.

A copy of the alarm manual can be downloaded at www.posey.com.

