Posey® Pediatric Mitts

Application Instructions

DESCRIPTION OF PRODUCT: A double-padded mitt with an open end/flap for access to fingers.

Rx ONLY

Indications for Use:

- Patients assessed to be at risk of disrupting life-saving treatments (e.g., chronic tube pulling).
- Patients assessed to be at risk of line pulling, which may prevent monitoring of vital signs.
- Patients whose picking, pulling, scratching, or peeling exacerbates a skin condition, causes self-injury, or compromises wound site integrity.

Contraindications:

- DO NOT** use on a patient who is or becomes highly aggressive, combative or agitated.
- NEVER** use mitts on a patient:
 - If an IV or wound site could be compromised by the device.
 - With a dislocation or fracture on the affected limb.
 - See the Posey Catalog for other options for such a patient.

Adverse Reactions:

Severe emotional, psychological, or physical problems may occur if the applied device is uncomfortable, or if it severely limits movement. If symptoms of these problems ever appear for any reason, get help from a qualified medical authority and find a less restrictive product or intervention.

Application Instructions:

(Repeat Steps 1-3 for each mitt):
Insert the patient's hand into the mitt, palm down. When the inspection flap is open, the back of the hand should be visible (Fig. 1).

Wrap the wrist strap around the smallest part of the patient's wrist and secure with the hook and loop fastener.

Slide ONE finger (flat) between the device and the inside of the patient's wrist to ensure proper fit. The strap must be snug, but not compromise circulation.

To view hand/fingers:

Reach under the inspection flap, detach the hook-and-loop fastener and pull back the flap to expose the hand.
To close the inspection flap, tuck the end of the mitt under the flap and press the hook-and-loop closure together firmly.

ADDITIONAL SAFETY AND LAUNDERING INSTRUCTIONS ON OTHER SIDE

PRODUCT not made with NATURAL RUBBER LATEX.

Posey Pediatric Mitts

- REF 2911S** Infant Mitts (0-1 year old)
- REF 2911M** Toddler Mitts (1-3 years old)
- REF 2911L** Youth Mitts (3-7 years old)

ADDITIONAL OR DIFFERENT BODY OR LIMB RESTRAINTS MAY BE NEEDED (See Posey Catalog):

- To reduce the risk of the patient getting access to the line/wound/tube site;
- To prevent the patient from falling or backing up and down and causing self-injury.

MONITOR PER FACILITY POLICY. Check to ensure that:

- Mitts are properly secured. If applied too tightly, circulation will be restricted; if applied too loosely, the patient may be able to slip his or her limb from the device;
- Mitts are intact, not torn or damaged, and hook and loop closes securely. DO NOT allow patients to ingest mitt material;
- The patient cannot use his or her teeth or otherwise remove the device and inflict self-injury;
- Monitor closely when the patient is out of bed. Patients who ambulate while wearing this device may be at risk of injury from a fall.

BEI SAFETY

- ALWAYS** use Hospital Bed Safety Workgroup (HBSW) (<http://www.fda.gov/search/keyword?keyword=HBSW>) compliant side rails in the UP position and fill ALL gaps to reduce the risk of entrapment.
- Use side rail covers and gap protectors to help prevent the patient's body from going under, around, through or between the side rails. A failure to do so may result in serious injury or death if a patient becomes suspended or entrapped. Posey offers a full range of side rail pads and/or gap protectors to cover gaps.

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