

3.11"
This panel visible when folded

3.11"
This panel visible when folded

3.11"



Narrow Saddle Bracket 8278N


Application Instructions



The 8278N Narrow Saddle Bracket is suitable for wheelchair mounting.

TO MOUNT THE 8278N NARROW SADDLE BRACKET:

1. Choose a location on rear of chair, out of the patient's reach, to attach bracket.
2. To attach the Posey Alarm to a wheelchair using the hook-and-loop strap and chair bracket, wrap the strap firmly around the wheelchair frame.
3. Make sure sensor cables can be secured away from moving parts of the chair.

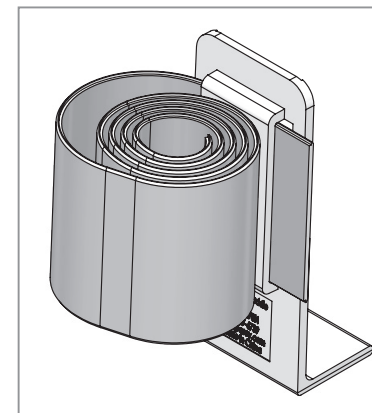

8278N Narrow Saddle Bracket



Posey Products, LLC (Posey) • 5635 Peck Road, Arcadia, CA 91006-0020 USA
Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com

© 2019 Posey Products, LLC. (Posey) All rights reserved.



M7004 REV A 031819

Printer: please note dashed fold lines. The upper left and upper middle sections should be visible when document is fully folded.
Folded size will be ~3.11 x 3.375