Fitting Instructions

Must be followed each time respirator is worn.

- Remove respirator from packaging and hold with straps facing upward (1a). Place bottom strap under center flaps next to **WARNING** statement as shown (1b).
- Fully open top and bottom panels, bending nospiece around thumb at center of foam. Straps should separate when panels are opened. Make certain bottom panel is unfolded and completely opened.
- Place respirator on your face so that the foam rests on your nose and the bottom panel is open under chin. Hold the bottom panel securely under your chin. Pull the top strap over your head and position it high on the back of the head (3a). Then pull the bottom strap over your head and position it around the neck and below the ears (3b). Adjust for a comfortable fit by pulling top panel toward the bridge of the nose and bottom panel under chin.
- Place your fingertips from both hands at the top of the metal nospiece. Using two hands, mold the nose area to the shape of your nose by pushing inward while moving your fingertips down both sides of the nospiece.
▲ Pinching the nospiece using one hand may result in improper fit and less effective respirator performance. Use two hands.
- Perform a User Seal Check prior to each wearing. To check the respirator-to-face seal, cover the middle panel with one or both hands. For the 9205+ and 9210+ respirator, inhale and exhale sharply. For the 9211+ respirator, inhale sharply. Be careful not to disturb the position of the respirator. If air leaks around the nose, readjust the nospiece as described in step 4. If air leaks around respirator edges, adjust panels and straps. **If you CANNOT achieve a proper fit, DO NOT enter the contaminated area. See your supervisor.**

Removal Instructions

Cup respirator in hand to maintain position on face and pull bottom strap over head. Still holding the respirator in position, pull top strap over head and remove respirator.

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NIOSH Approved: N95
At least 95% filtration efficiency against solid and liquid aerosols that do not contain oil.

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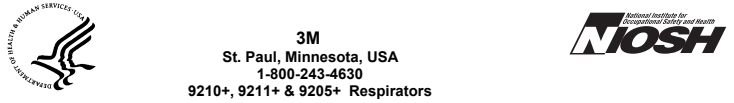
THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS					
RESPIRATOR COMPONENTS					CAUTIONS AND LIMITATIONS ²
FILTERING FACEPIECE					
TC-	PROTECTION ¹	9210+	9211+	9205+	
84A-5668	N95	X			ABC,MN,OP
84A-5669	N95		X		ABC,MN,OP
84A-8590	N95			X	ABC,MN,OP

- PROTECTION**
N95 - Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.
- CAUTIONS AND LIMITATIONS**
A - Not for use in atmospheres containing less than 19.5 percent oxygen.
B - Not for use in atmospheres immediately dangerous to life or health.
C - Do not exceed maximum use concentrations established by regulatory standards.
J - Failure to properly use and maintain this product could result in injury or death.
M - All approved respirators shall be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations.
N - never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
O - Refer to User's Instructions, and / or maintenance manuals for information on use and maintenance of these respirators.
P - NIOSH does not evaluate respirators for use as surgical masks.

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