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
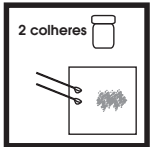
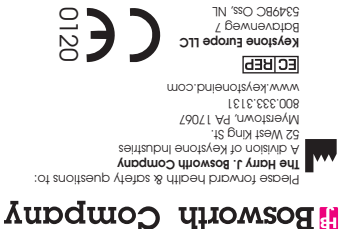
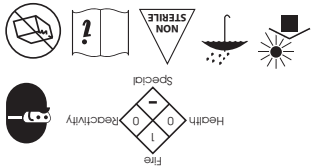
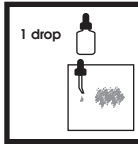
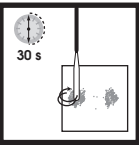
30 s

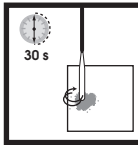30 s

1. Place two scoops of powder on a clean dry glass slab or mixing pad.

2. Dispense one drop of liquid alongside the powder.

3. Add up to 3/4 of the powder to the liquid and spatulate for 30 sec. using a stainless steel, chromium cobalt or agate spatula.

4. Add the remaining powder and mix another 30 sec. until a thick putty like consistency is obtained.

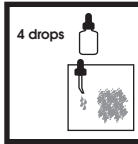
5. Apply the mixture to the cavity with an amalgam plugger which has been dipped in water or alcohol.

6. Exposures can be directly capped with a thin mixture. Spray with warm water to accelerate setting time.


DIRECTIONS FOR FINAL CEMENTATION


4 scoops

4 drops

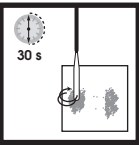
30 s

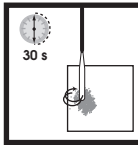30 s

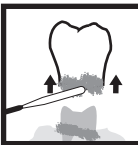Masque

1. Place four scoops of powder on a clean dry glass slab or mixing pad.

2. Dispense four drops of liquid alongside the powder.

3. Add up to 3/4 of the powder to the liquid and spatulate for 30 sec. using a stainless steel, chromium cobalt or agate spatula.

4. Add the remaining powder and mix another 30 sec. until a smooth creamy mix is obtained.

5. Smear surrounding restoration areas with Bosworth Masque to allow easy removal of excess cement.

6. Coat fitting surface and shoulders with a thin film of cement. Retain under steady finger pressure until set.

SETTING TIME (MOUTH)

- Regular (Slow) Set 10-15 minutes
- Fast Set 4-8 minutes

SETTING TIME (SLAB)

- Regular (Slow) Set >15 minutes
- Fast Set 10-15 minutes

TIPS FOR USING SUPEREBA®

- The degree of latitude in mixing SuperEBA® is very wide and will be affected by the temperature and humidity in your operator. Higher temperature and humidity may shorten work and set times. Lower temperatures and humidity may increase work and set times.
- As you become accustomed to this cement, with its improved flow, you will observe that thicker mixes than those recommended below can be used for some cementing procedures, e.g. Orthodontic cementation. Use thicker mixes only if practical. Remember, the thicker the mix the stronger the cement.

Liquid	
1) Ortho-Ethoxy Benzoic Acid	62.5%
2) Eugenol	37.5%
Powder	
1) Zinc Oxide	60%
2) Alumina	34%
3) Natural Resin	6%

Powder/Liquid Ratios	
Lining	.26 g powder - .03 g liquid
Cementing	.52 g powder - .12 g liquid

For use by a dental professional only.

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