

Sodium Hypochlorite

DIRECTIONS FOR USE

REF 11507 – Sodium Hypochlorite – 16 oz / 473 ml

For Dental Use Only.

Caution: US Federal Law restricts this device to the sale by or on the order of a dentist.

1. DESCRIPTION

This product is a 3-6% solution of sodium hypochlorite, intended for use as an endodontic irrigation solution.

2. INDICATIONS

Indicated for the irrigation of root canals as part of complete endodontic therapy.

3. CONTRAINDICATIONS

This product should not be used with individuals with known sensitivities to sodium hypochlorite.

4. PRECAUTIONS

Use only as directed. Sodium hypochlorite can potentially irritate eyes, skin and mucosa. Avoid skin contact and use eye protection when handling. Use a rubber dam at all times to protect tissue. For dental use only.

5. WARNINGS

Do not swallow. Keep out of reach of children. For professional dental use only.

6. DOSAGE AND ADMINISTRATION

1. Dispense sodium hypochlorite into a disposable cup.
2. Fill an endodontic irrigation syringe, fitted with a blunt, side dispensing endodontic irrigation needle. Do not use a sharp injection type needle when dispensing this product.
3. During the instrumentation process, liberally use sodium hypochlorite to irrigate canals to remove cutting debris and tissue debris. Follow proper safety precautions, including using eye protection for both the operator and patient. Always use a rubber dam.
4. Irrigate by placing the irrigation needle into the canal space and irrigate from the bottom of the canal. Gently dispense the sodium hypochlorite irrigation solution into the canal space while simultaneously collecting expelled irrigant using high speed suction.
5. Repeat as needed.
6. After the procedure is finished, discard the used irrigation syringe. Do not attempt to reuse either the irrigation syringe or the sodium hypochlorite solution.

7. STORAGE

Store between 59°-77°F (15°-25°C). Protect from freezing.

Manufactured for:



Sultan Healthcare
1301 Smile Way • York, PA 17404-0807 • USA
Toll Free: 800-637-8582 • Phone: 201-871-1232
Fax: 201-871-0321 • www.sultanhealthcare.com

Made in the USA
0011507DF, R1-071714