

INSTRUCTIONS FOR USE

Treatment Accessories



Premium Plus & Advantage Saliva Ejectors

Instructions for use:

1. Attach Ejector head to hand piece.
2. Place in mouth and bend to desired position.
3. After use, dispose of the ejector.



Ref.#'s ZCB, ZCW, ZWW, ZPP,
ZCMC, ZBBI, ZCBI, ZCCI, ZWWI



Do not reuse.



Not made with natural rubber latex.



CROSSTEX[®]
A CANTEL MEDICAL COMPANY



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



Made in USA

Manufactured by: Crosstex International, Inc.
10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | Crosstex@crosstex.com

Z-IFU
Rev. 0 – 11/16