3M ESPE

Pentamix™ 3

- Wall Fixture
- Wandhalterung
- Support mural
- Supporto alla parete
- Soporte de pared
- Supporte de parède
- Wandhouder
- Επιτοίχια βάση
- Väggfäste
- Seinäteline
- Vægghøng
- Veggholder

Installation Instructions
Gebruichsinformation
Mode d'emploi
Istruzioni di montaggio
Instrucciones de montaje
Instruções de montagem

Montageaanwijzing
Οδηγίες εγκατάστασης
Monteringsanvisning
Asennusohjeet
Monteringsvejledning
Monteringsanvisning

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Dental Products

Carl-Schurz-Str. 1

41453 Neuss - Germany

3M ESPE

Dental Products

St. Paul, MN 55144-1000 U.S.A.

3M, ESPE and Pentamix are trademarks of 3M Company or

3M Deutschland GmbH. Used under license in Canada.

© 2012, 3M. All rights reserved.

ENGLISH

Product Description

The wall fixture can be used solely for the Pentamix 3 unit.

In view of the weight of the unit (9,8 kg), check the wall to make sure that it is

suitable for the secure attachment of the fixture. Depending on the strength of the

wall, the appropriate wall plugs and screws with a diameter of at least 6 mm

must be provided. The wall fixture itself can be used as the template for drilling the

required holes.

Use
Mounting of Pentamix 3 on a suitable wall.

Mounting
The metal fixture is mounted as shown in Figure 1:

Fig.1

► Attach the metal fixture to the desired position on the wall, using screws of the

appropriate size. The two upper hole positions correspond to those of the

Pentamix 2 wall fixture. The lower screw secures the unit from forces coming

from below, i.e., when the mixing tips are attached.

Caution: Make sure that wall plugs and screws are adequately

dimensioned and that the fixture is securely attached to the wall.

Prepare the Pentamix 3 for mounting on the wall:

► Close the cover and lay the Pentamix 3 unit on a suitable pad with the front

of the unit facing down (Fig. 2).

Fig. 2

► Remove the cover on the back of the housing by pressing the two snap locks

in the direction of the arrows (Fig. 3) and replace it with the cover which is

provided with the wall fixture.

Fig. 3

► Thread the power cable through the opening in the cover of the wall fixture

and plug it into the power socket on the unit (Fig. 4).

Fig. 4

► Attach the cover of the wall fixture to the underside of the unit. Make sure

that the noses of the cover slide firmly into the corresponding openings in the

housing (Fig. 5 + 6) and that the snap locks audibly click into place (Fig. 7).

Caution: Never operate the unit without the cover.

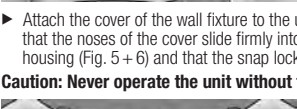
Fig. 5

Fig. 6

► Thread the power cable through the channel in the wall fixture (Fig. 7).

Fig. 7

► Slide the Pentamix 3 unit all the way to the wall, then push it down to lock it

Fig. 8

► Attach the enclosed cable lead to the power cable. Hold the power cable in

one hand. Use the other hand to slide the cable lead upwards into the end of

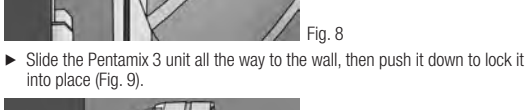
Fig. 9

If the Pentamix 3 unit has been correctly mounted, the entire base surface of the

wall fixture will be flat against the wall.

► Attach the enclosed cable lead to the power cable. Hold the power cable in

one hand. Use the other hand to slide the cable lead upwards into the end of

the metal fixture (Fig. 10).

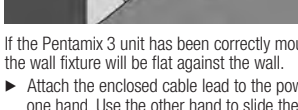
Fig. 10

Removing the unit from the wall and/or converting it to a tabletop unit:

Caution: We recommend asking a second person for assistance when removing

the Pentamix 3 unit.

► Remove the cable lead by pulling it downwards from the metal fixture (Fig. 11).

► Press the ends of the metal fixture together and simultaneously slide the

Pentamix 3 unit upwards until it releases from the metal fixture.

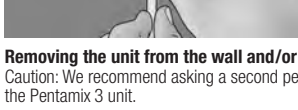
Fig. 11

► Complete the removal by following the steps in the Mounting Instructions

(see above) in reverse order.

Customer Information

No person is authorized to provide any information that deviates from the

information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product to be free from defects in material

and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES,

INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR

A PARTICULAR PURPOSE. User is responsible for determining the suitability of the

product for user's application. If this product is defective within the warranty

period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall

be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any

loss or damage arising from this product, whether direct, indirect, special,

incidental or consequential, regardless of the theory asserted, including warranty,

contract, negligence or strict liability.

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