
Instructions for Use
Mode d'emploi

3M Deutschland GmbH
Dental Products
Carl-Schurz-Str.1
41453 Neuss – Germany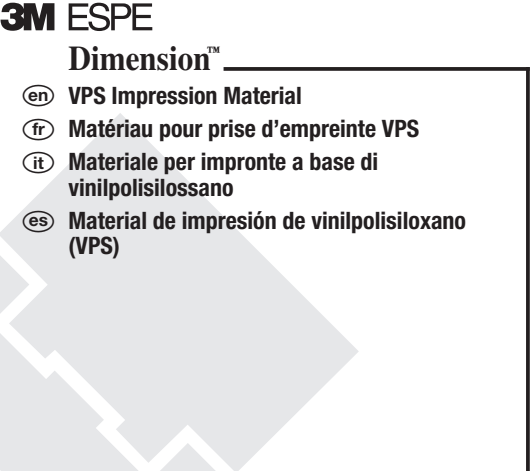

3M ESPE
Dental Products
St. Paul, MN 55144-1000 U.S.A.

3M, ESPE, Dimension, Garant, Penta and Pentamix are trademarks of 3M Company or 3M Deutschland GmbH. Used under license in Canada.
© 2012, 3M. All rights reserved.

4400018724301

ENGLISH

Product Description

These instructions for Use apply to the subsequently listed addition-type vinyl polysiloxane impression materials, which are used for the one-step or two-step impression technique.

Product Name	ISO 4823	ISO 4823 Consistency	Paste Color	Mixing Ratio (Vol.)	Mixing Device
Dimension™ Penta™ Putty	Type 0	putty	green	5:1	Pentamix 2 and Pentamix 3
Dimension™ Regular Body	Type 2	medium-bodied	petrol	1:1	Garant Dispenser
Dimension™ Ultra-Light Body Quick	Type 3	light-bodied	orange	1:1	Garant Dispenser

* Dimension Penta Putty can only be mixed in Pentamix 2 and Pentamix 3; earlier Pentamix units (before September 1999) extrude the material too slowly.

☞ These Instructions for Use should be kept for the duration of product use. For all additionally mentioned products please refer to the corresponding Instructions for Use.

Areas of Application

- All precision impressions (of e.g.: crown, bridge, inlay, and onlay preparations)

Precautionary Measures

3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Preparation

- Block out undercuts or interdental spaces sufficiently to aid impression removal from the teeth after setting as otherwise impression removal from the mouth may be difficult or could lead to the extraction of natural teeth or prostheses. Blocking is particularly necessary when using Dimension Penta Putty or any other impression material with increased shore hardness.

Impression Tray:

- Rigid impression trays are recommended for two-step impression technique and when using Dimension Penta Putty.
- For sufficient adhesion, apply VPS Tray Adhesive on the tray and allow it to dry completely, whereby the ideal drying time is 5–15 minutes.

Penta Material:

- Place the Penta foil bags into the designated metal-reinforced Penta cartridges.
- Insert the loaded cartridge into the Pentamix mixing unit.
- Attach a red Penta™ Mixing Tip.
- Turn the plungers of the mixing unit downward with the hand-wheel until resistance is noticed; do not force.
- When using new foil bags, start mixing and extrude a small amount of paste until it is uniform in color. Discard this first amount of paste.
 - Paste color: please see “Product Description”.
 - Material that has set inside the Penta Mixing Tip should not be extruded by force as this may damage the Penta Cartridge and Penta Mixing Tip and result in the formation of leaks.
- Please follow the section “Impression Taking” for tray filling procedure.

Garant Dispenser:

- Place the cartridge of the wash or medium bodied material in the Garant Dispenser.
- Prior to application, check that both Garant cartridge openings are not clogged.
- When using new Garant cartridges, extrude a small amount of paste without the mixing tip until base paste and catalyst are evenly extruded.
- Attach a yellow Garant™ Mixing Tip to the cartridge.
- For intraoral application attach a yellow Garant Intraoral Tip onto the Garant Mixing Tip.
- If required: enlarge the exit opening for the intraoral tip by cutting it with a scalpel.
- Make sure that base paste and catalyst are mixed completely and are extruded in a uniform color.
 - Paste color: see “Product Description”.
 - Material that has set inside the Garant Mixing Tip should not be extruded by force as this may damage the Garant cartridge and Garant Mixing Tip and result in the formation of leaks.

Retraction

- Suitable retraction agents include aluminum hydroxide chloride, aluminum sulfate or ferric sulfate solutions.
- Keep the areas from which the impression is to be taken dry during impression.
- The use of retraction cords may be required for subgingival preparations.
- Prior to the impression, thoroughly remove residual retraction material by flushing and drying.

Product	Working Time at 23°C/74°F from start of mixing* min:sec	Intraoral Syringe Time min:sec	Intraoral Setting Time min:sec
Dimension Penta Putty	1:30	–	3:00
Dimension Regular Body	2:00	1:00	3:00** (3:30***)
Dimension Ultra-Light Body Quick	1:30	0:40	2:30

* Start of mixing = Entry of paste into the mixing tip.

** In combination with Dimension Penta Putty when using one-step technique.

*** When using two-step and one-step impression technique, if Dimension Penta Putty is not used.

These are the processing times for products that have been stored at a temperature of 23°C/74°F and 20–80% relative humidity. These times are reduced due to higher temperatures of the product or prolonged if the temperature of the product is lower.

Impression Taking

One-step Impression Technique:

Recommended product combination	
Dimension Penta Putty	Dimension Regular Body

- At the start of mixing the Penta impression material, be sure to observe the entry of the paste into the Penta Mixing Tip and ensure that base paste and catalyst paste flow evenly into the mixing tip. The mixed paste must be extruded in a uniform color free of streaks.
 - Paste color: see “Product Description”.

- Load the impression tray prepared with adhesive, keeping the mixing tip permanently immersed in the paste to avoid the formation of air bubbles.
- While the tray is being loaded, apply the wash material around the cleaned and dried preparation from the bottom up. Keep the application nozzle permanently immersed in the paste to avoid the formation of air bubbles.
 - Depending on the number of units to be syringed, initiate application so that the tray loading and application of material around the preparation are completed simultaneously.
 - Do not exceed processing times of tray material and syringe material to avoid incomplete and inaccurate impressions.
- Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth and hold it in place without exerting pressure.
- Remove the impression from the mouth after completed setting.

Two-step Impression Technique:

Recommended product combination	
Dimension Penta Putty	Dimension Ultra-Light Body Quick

- After start of mixing the Penta impression material, be sure to observe the entry of the paste into the Penta Mixing Tip and ensure that base paste and catalyst paste flow evenly into the mixing tip. The mixed paste must be extruded in a uniform color, free of streaks.
 - Paste color: see “Product Description”.
- Load the impression tray prepared with adhesive, keeping the mixing tip permanently immersed in the paste to avoid the formation of air bubbles.
- Slowly insert the loaded tray into the mouth parallel to the long axes of the prepared teeth and hold it in place without exerting pressure.
- Remove the impression from the mouth after completed setting, clean and dry.
- Excise undercuts and interdental septa, etc., and cut spillways, if required.
- Clean and dry the impression thoroughly to provide for optimal coalescence of the two materials.
- Insert the wash impression material into the preliminary impression at the appropriate sites and/or apply around the cleaned and dried preparations while keeping the application nozzle permanently immersed in the paste to avoid the formation of bubbles.
- Replace the preliminary impression into the mouth and hold it in place.
- Remove the impression from the mouth after completed setting.

After Impression Taking

- Thoroughly examine and explore the sulcus of the prepared teeth and surrounding dentition. Remove any residual cured impression material from the mouth.

Hygiene

- Place the impression in a standard disinfectant solution. The duration complies with the instructions of the manufacturer. Do not leave the impression submerged in the disinfection solution for more than the specified amount of time as otherwise the impression may be damaged.
- After disinfection, clean the impression of any remaining disinfection solution by rinsing under running water for approx. 15 seconds.

Model Preparation

- The impression can be poured at the earliest two hours after removal from the mouth. There are no other time limitations for pouring.
- In order to attain a bubble-free model, rinse the impression with water shortly before pouring, remove excess water or use a silicone surfactant.
- Pour the impression with a standard dental die or model stone. The impres-sions can be silver or copper plated.

Cleaning

- Garant Dispenser: remove paste that has not set using a cloth soaked in alcohol. The dispenser handle and plunger can be autoclaved up to a temperature of 135°C/275°F. Disassemble the dispenser before autoclaving. Glutaraldehyde-based solutions can be used for disinfection.
- Impression Tray: the adhesive can be removed from re-usable impression trays using acetone.

Notes

- Completely remove the oxygen inhibition layer of composite materials, e.g., fillings or core buildups; it may impair or retard the setting of silicone impression materials.

- Keep the filled Penta or Garant Mixing Tip on top of the cartridge to serve as a closure until the next use. Removing and replacing the spent Penta or Garant Mixing Tip may lead to carry-over of paste and ensuing formation of clogs.

- Vinyl polysiloxane impression materials must not be combined with condensation type silicones or polyether impression materials. Even small traces can affect the setting process.
- Disposable latex gloves impair the setting of silicone impression materials. Vinyl gloves are more suitable.
- Under no circumstances may impressions come into contact with solvent-containing fluids. This could result in expansion and inaccurate stone models.

Storage and Stability

Store the product at 15–25°C/59–77°F.

Do not use after the expiration date.

Store set impressions at temperatures below 30°C/86°F.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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