



# STERILIZATION QUICK GUIDE

ENSURE PROPER USE OF YOUR STERILIZER. FOLLOW THESE STEPS FOR STERILITY ASSURANCE, **EVERY TIME.**

## STEAM STERILIZERS

Steam sterilizers are recommended as the sterilization method of choice by the CDC



### GRAVITY DISPLACEMENT STEAM STERILIZER

Manufacturer Models Include: TUTTNAUER™: Manual M/MK series (2340M, 2540M, 2540MK, 3870M) and Valueclave™ 1730 Fully Automated: EZ9™, EZ10™, EZ10k™, 3870EA

### DYNAMIC AIR REMOVAL FOR STEAM FLUSH PRESSURE PULSE (SFPP)

Manufacturer Models Include: SCICAN: Statim 2000 / 5000 / 2000 G4 / 5000 G4. MIDMARK: M3 / M9 / M11. TUTTNAUER - EZ Plus Series (EZ9 Plus, EZ11 Plus)

1



**Sure-Check® Pouches**  
For use with **EVERY LOAD**  
(Internal/external multi-parameter chemical indicators)

2



**STEAMPlus™ Type 5\* Integrator**  
For use with **EVERY LOAD** or at least **EVERY DAY** for added sterility assurance

3



**Mail-In or In-Office ConForm® Biological Monitors**  
Spore test **AT LEAST WEEKLY**

### DYNAMIC AIR REMOVAL PRE-VACUUM (CLASS B)

Pre-vacuum (Class B) – Requires daily air removal test (Bowie-Dick) to verify vacuum pump is working properly.

Manufacturer Models Include: SCICAN: Bravo, W & H: Lisa, TUTTNAUER: Elara 11



**Run Warm-up Cycle**  
with an empty chamber at the start of **EVERY DAY**

2



**AirView™ II Bowie-Dick Test**  
Use in empty chamber after warm-up cycle **EVERY DAY**

3



**Sure-Check® Pouches**  
For use with **EVERY LOAD**  
(Internal/external multi-parameter chemical indicators)

4



**STEAMPlus™ Type 5\* Integrator**  
For use with **EVERY LOAD** or at least **EVERY DAY** for added sterility assurance

5



**Mail-In or In-Office ConForm® Biological Monitors**  
Spore test **AT LEAST WEEKLY**



## CHEMICAL VAPOR STERILIZERS

Chemical Vapor Sterilizers use formaldehyde solution instead of distilled water

Chemical-Vapor sterilizers look like, but do not act like steam sterilizers and are found primarily on the West Coast. Chemical-Vapor sterilizers are pre-set for 270°F for 20 minutes exposure. No dry time is needed. Maximum load is 3.2lbs.

1



**Sure-Check® Chemical Indicator Strips**  
For use in **EVERY POUCH**

2



**In-Office SporView® Culture Set**  
Spore test **AT LEAST WEEKLY**

-OR-



**Mail-In ConForm® Biological Indicators**  
Spore test **AT LEAST WEEKLY**

## DRY HEAT STERILIZERS

Used primarily in Ortho, Perio and Oral Surgery Practices

Dry Heat sterilizers are available in two different types – Convection (low cost and slow) or Rapid Heat Transfer (high cost and fast).

**Convection:** 320°F for 1-2 hours depending on load size. **Rapid Heat Transfer:** 375°F for 6 or 12 minutes, depending on the cycle selection.



1



**Nylon Pouches with Type 1 Process Indicator Labels**  
For use with **EVERY LOAD.**

2



**Dry Heat Chemical Indicator Strips**  
For use in **EVERY POUCH.**

3



**In-Office SporView® Culture Set**  
Spore test **AT LEAST WEEKLY**

-OR-



**Mail-In ConForm® Biological Indicators**  
Spore test **AT LEAST WEEKLY**

Ordering Information  
**1-888-276-7783**  
customer@crossstex.com

Infection Control Helpline  
(1-855-878-3745)  
**TOLL FREE 1-8558-STERILE**  
infectioncontrol@crossstex.com

**CROSSTEX®**  
A CANTEL MEDICAL COMPANY



# STERILIZATION QUICK GUIDE

## Sure-Check® Sterilization Pouches

<b>500/box</b>		<b>100/box</b>	
SCS25	3.5" x 9"	SCL10152	10" x 15"
		SCL10172	10.5" x 17"
<b>200/box</b>		SCL12152	12" x 15"
SCXX2	2.25" x 4"	SCL12182	12" x 18"
SCX2	2.75" x 9"		
SCXS2	3.5" x 5.25"		
SCS2	3.5" x 9"		
SCL3222	3.5" x 22"		
SCW2	5.25" x 6.5"		
SCM2	5.25" x 10"		
SCL5152	5.25" x 15"		
SCL2	7.50" x 13"		
SCL8162	8" x 16"		



## Disposable AirView™ II Bowie-Dick Test Pack

**30/case**  
**MBD030**



## Sure-Check® Multi-Parameter Strips

**100/bag**  
**SCK | 3.75" x .625"**



## STEAMPlus™ Type 5\* Integrators

**100/bag**  
**SSI-100 | 4" Strips**



## Dry Heat Nylon Self-Seal Pouches

**100/box**  
**NSP-400 | 2" x 10"**  
**NSP-405 | 3" x 10"**  
**NSP-410 | 4" x 10"**  
**NSP-420 | 7" x 10.5"**  
**NSP-430 | 9.5" x 13"**

Type 1 Process Indicator Labels Included



## ConForm® 10 In-Office Biological Monitoring System

**C10SK** | Digital Incubator and Record Keeper (Specify 110 or 120 Voltage)  
**C10B125** | Steam Biological Indicators (25/box)  
**NDB-601** | ConForm® 10 Standard Incubator and Record Keeper



**ConForm® 10 Biological Indicators with digital incubator**

## Dry Heat Chemical Indicator Strip

**100/box**  
**DIS-100 | 4" Strips**



## Mail-In ConForm® Biological Indicators

**Premium Test Service - Postage Included**

**CST060** | 6 Tests (3 Strip Test)  
**CST120** | 12 Tests (3 Strip Test)  
**CST480** | 48 Tests (3 Strip Test)

**Value Test Service Tests**  
**Postage Not Included**

**CVT120** | 12 (2 Strip Test)  
**CVT520** | 52 Tests (2 Strip Test)



## SporView® Culture Set

**SK-037** | SporView® Culture Set Starter Kit (Dry Heat & EO Gas)\*  
**SK-056** | SporView® Culture Set Starter Kit (Chemical-Vapor)\*  
**CS-020** | SporView® Culture Set (20 tubes and 20 spore strips)  
**CS-100** | SporView® Culture Set (100 tubes and 100 spore strips)



\* Starter kits include box of 20 tubes/strips, dry block incubator (NDB601), record notebook and transfer rack to provide offices with all the necessary components for in-office testing.

\*According to ANSI/AAMI/ISO 11140-1:2014

All copyrights and trademarks are property of Crosstex International, Inc.