

Label Size: 5.6226' x 4.5092"  
Pantone 326C - IMS Green  
Pantone 1945C - HF Red  
BLACK



+D6711361231%

SDS Available on Request or at Hu-Friedy.com.  
F.S. disponible sur demande ou au Hu-Friedy.com.



## DANGER

May be fatal if swallowed and enters airways.

**PREVENTION:** Observe good industrial hygiene practices. **RESPONSE:** If swallowed: Immediately call a poison center/doctor. Do NOT induce vomiting. **STORAGE:** Store locked up. **DISPOSAL:** Dispose of contents/container in accordance with local/regional/national/international regulations.

4% of the mixture consists of component(s) of unknown acute oral toxicity.



## DANGER

Peut être mortel en cas d'ingestion et de pénétration dans les voies respiratoires.

**PRÉVENTION:** Suivre les règles de bonnes pratiques chimiques. **INTERVENTION:** En cas d'ingestion : Appeler immédiatement un centre antipoison/médecin. NE PAS faire vomir. **STOCKAGE:** Garder sous clef. **ÉLIMINATION:** Éliminer le contenu/les conteneurs selon la loi internationale/nationale/régionale/locale.

4 % du mélange sont constitués de composants dont la toxicité aiguë par voie orale est inconnue.

An IMS® Product

### INSTRUMENT LUBRICANT LUBRIFIANT POUR INSTRUMENT

- Keep hinged instruments moving freely
- Water soluble
- Silicone, phosphate and chlorine free
- Safe for use with autoclave sterilizers
- Neutral pH
- Rust inhibitor

#### Helps Prevent

- Discoloration
- Electrolytic mineral deposits
- Build up of baked-on protein

ILS | 8 fl. oz. (237 mL)

KEEP OUT OF REACH OF CHILDREN.

GARDER HORS DE LA PORTÉE DES ENFANTS.

#### DIRECTIONS

1. Shake well before using.
2. Spray instruments after cleaning (box lock open), and before autoclaving
3. Let instrument drain.
4. Wipe off excess liquid.

#### INSTRUCTIONS D'UTILISATION

1. Bien secouer avant utilisation.
2. Vaporiser les instruments après les avoir nettoyés (mécanisme auto-bloquant ouvert) et avant la stérilisation en autoclave.
3. Laisser égoutter l'instrument.
4. Essuyer le liquide en trop.

#### Manufactured for:

**Hu-Friedy Mfg. Co., LLC**

3232 N. Rockwell Street

Chicago, IL 60618 | USA

**Telephone:** 1-800-Hu-Friedy

(1-773-975-6100)

**Website:** Hu-Friedy.com

**Europe:** Zweigniederlassung,

Deutschland Ziegeleiweg 1

D-78532 Tuttlingen-Möhringen

**(Free Call) Tel.:** 00800 48 37 4339

**Website:** Hu-Friedy.eu

**24-Hour Number:** 1-800-535-5053

**International:** 1-352-323-3500

ILS/0115

**Hu-Friedy**