



SIDEKICK® SHARPENER

Sharpening innovation at your fingertips



FACT

Scalers and curettes can dull over time as a result of routine, every day use. Keeping instruments sharp can improve deposit removal and decrease the potential for hand fatigue.*

CHALLENGE

Achieving a fine cutting edge without compromising the original shape of the instrument blade can be challenging and time-consuming.

SOLUTION

Hu-Friedy's Sidekick is a cordless, lightweight sharpening device designed to provide convenient and consistent sharpening results and accurate angles every time.

POINTS OF PERFORMANCE


- Easy-to-use guide channels and vertical backstops allow simple positioning of scaler and curettes to provide consistent sharpening results every time.
- Provides consistent sharpening results aiding in instrument longevity.
- Cordless, lightweight, portable sharpening...anywhere, anytime.
- Reciprocating stone does the work so you don't have to
- Maintains sharp blade and accurate angles



How the best perform




SIDEKICK® SHARPENER

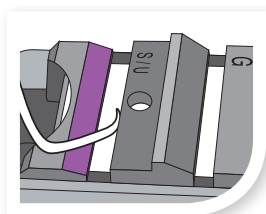


SHARPENING MADE EASY!

- Easy-to-read letters indicate Graceys (**G**) or Sickles and Universals (**S/U**)
- Position the instrument in the specific channel, with the terminal shank resting on the incline of the channel
- Position the back of the instrument along the backstop of the guideplate
- Turn the unit on, and glide the instrument within the channel from side to side
- Repeat 2 - 3 times or until blade is sharp


ACHIEVE ACCURATE ANGLES EVERY TIME

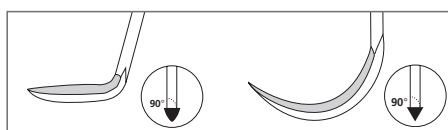

SICKLE SCALERS

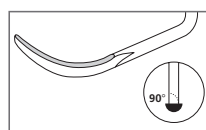

UNIVERSAL CURETTES

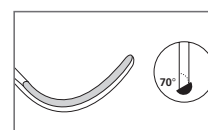

GRACEY CURETTES

ADDITIONAL ITEMS

REPLACEMENT PARTS

Stone Clip	SDKCLIP
Guideplate Screw	SDKGPSCREW
Guideplate	SDKPLATE
Ceramic Stone	SDKSTONEC
Sharpen-EZ™ Stone Oil	SSO
Plastic Test Sticks	PTS
Magnifying Lens	LENS
Large Parts Box	IMS-1273

SIDEKICK SHARPENING KIT: SDKKIT

Kit includes: Sidekick Unit, User Manual, Ceramic Sharpening Stone, Plastic Test Stick, Magnifying Lens and 2 AA Batteries.

Visit us online at Hu-Friedy.com

Call 1-800-Hu-Friedy or contact your authorized Hu-Friedy representative for more information.
©2011 Hu-Friedy Mfg. Co., LLC. All rights reserved. Hu-Friedy, the Hu-Friedy oval logo, EverEdge and Sharpen-EZ are trademarks of Hu-Friedy Mfg. Co., LLC. HF-577 /1111.

