How to Use

Pro Tector®

Needle Sheath Prop





For a How-To-Use video on this product, please visit us online at: www.Certol.com/ProTectorVideo



1. Assemble syringe with anesthetic and capped needle as usual.



2. Holding assembled syringe, insert cap through hole in Needle Sheath Prop, aligning ridges of needle cap with grooves around hole.



3. Insert needle cap through hole up to collar of cap.



4. Hold onto cap behind Pro Tector Prop and pull cap off.



5. Place Pro Tector Prop on tray with open end of cap facing up.



 Use one-handed technique to recap. Insert needle into cap; bring syringe vertical, push down slightly.

