

Sinfony™ Magic Shade

Internal tinting (technique shown), roughen internal buccal, mesial, and distal proximal surfaces above the shoulder margin with a diamond or crosscut carbide bur.

[External tinting, roughen surface(s) to be tinted (follow steps 2–9).]

Apply 3M[™] ESPE[™] RelyX[™] Ceramic Primer, gently blow dry for 5 seconds.





Apply 2 consecutive coats of Single Bond, blow dry gently for 5 seconds after second coat.



Light cure for 10 seconds.



Dispense tint or restorative onto mixing pad.





Sinfony™ Magic Shade



Apply tint to inside prepared surface of crown. For external tinting follow steps 1–5, then add to the prepared outer surface.



Light cure 10 seconds for initial set.



Fit check restoration on prep.



Final cure for 40 seconds.