

# 3M post and core solution. Complexity simplified.

**For predictable results – in less time:** The endodontic procedure is very complex. So the simpler and more reliable the endodontic restorative procedure is, the better – for both the dentist and the patient. With fewer clinical steps and fewer products, the 3M post and core solution lets you work faster, easier and more efficiently.

And, it helps you deliver predictable, robust outcomes. All you need is the new RelyX™ Fiber Post 3D Glass Fiber Post plus three other easy-to-use 3M products: RelyX™ Unicem 2 Automix Cement, Scotchbond™ Universal Adhesive, and Filtek™ Bulk Fill Posterior Restorative. These four products have one common objective: Simplify the complex!



**Filtek™ Bulk Fill**  
Posterior Restorative




**Scotchbond™ Universal**  
Adhesive



**RelyX™ Fiber Post 3D**  
Glass Fiber Post



**RelyX™ Unicem 2 Automix**  
Self-Adhesive Universal  
Resin Cement

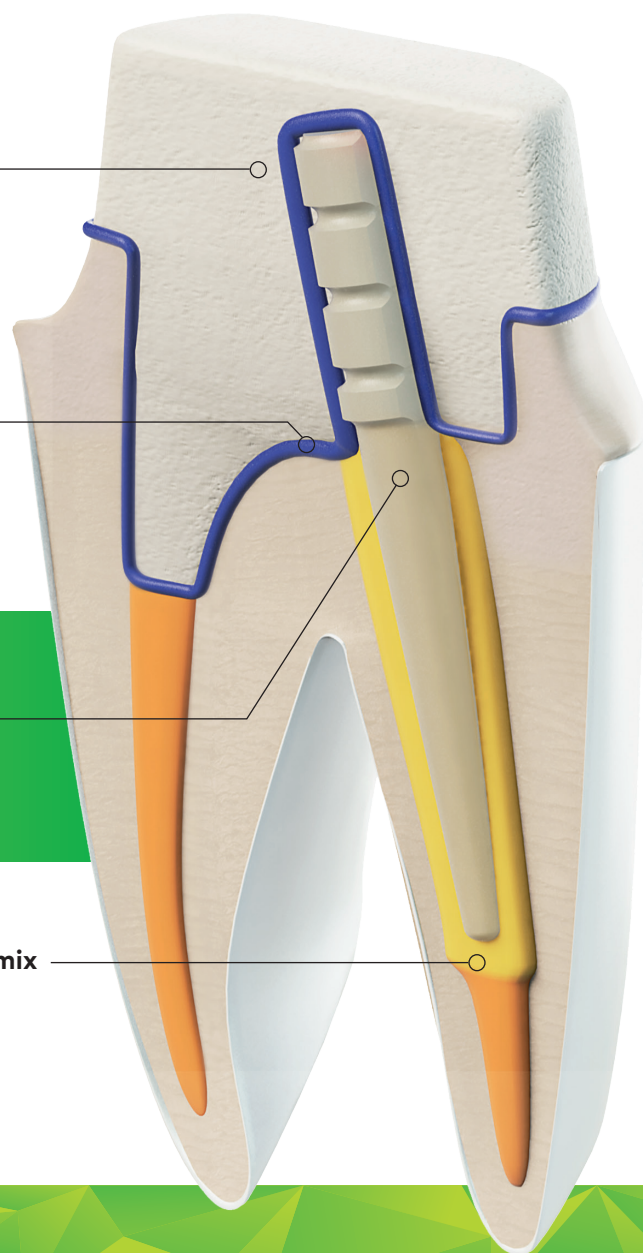

**RelyX™ Unicem 2 Automix Self-Adhesive Universal Resin Cement** offers excellent bond strength plus saves you precious time, as there is no need to etch, prime or bond. The automix syringe, together with the endo tip, lets you apply the ready-mixed cement directly in the root canal from bottom to top. In one application – without voids.

**RelyX™ Fiber Post 3D Glass Fiber Post's** coronal 3D macro retention offer higher mechanical retention to the core build-up material. Due to its microporous surface it also provides a strong bond in the root canal without a silanating step. Post position control is greatly improved as a result of its enhanced radiopacity.

**Scotchbond™ Universal Adhesive** provides a reliable bond of the core build-up material to the post and tooth structure – with no need for an additional primer. One application – done!

**Filtek™ Bulk Fill Posterior Restorative** is ideally suited as core build-up material, especially for big lesions in combination with a post: The new capsule design ensures a void-free application. The material is sculptable but also easy to adapt to both tooth structure and post. And, for easier preparation, the cured material cuts like dentin. Core build-ups can be done faster and easier.



## Ordering Information

Item No.	Product Information
<b>RelyX™ Fiber Post 3D Glass Fiber Post Kits</b>	
56957	<b>Introductory Kit:</b> 20 posts sizes 0, 1, 2, 3 (5 posts of each size), 5 drills sizes 0, 1, 2, 3, Universal (1 drill of each size)
56958	<b>Trial Kit with RelyX™ Unicem 2 Automix Cement:</b> 10 posts sizes 1, 2 (5 posts of each size), 3 drills sizes 1, 2, Universal (1 drill of each size), 1 RelyX™ Unicem 2 Automix Syringe A2 – 8.5g, 10 mixing tips, 10 endo tips
56960	<b>Post and Core Build-up Kit with RelyX™ Unicem 2 Automix Cement:</b> 1 Trial Kit RelyX™ Fiber Post 3D/RelyX™ Unicem 2 Automix Cement, 20 capsules Filtek™ Bulk Fill Posterior Restorative A3 – 0.2g each, 1 vial Scotchbond™ Universal Adhesive – 5ml
<b>RelyX™ Fiber Post 3D Glass Fiber Posts and Drills</b>	
56948	10 posts, size 0 (1.1 mm dia), white
56949	10 posts, size 1 (1.3 mm dia), yellow
56950	10 posts, size 2 (1.6 mm dia), red
56951	10 posts, size 3 (1.9 mm dia), blue
56873	1 drill, size 0 (1.1 mm dia), white
56864	1 drill, size 1 (1.3 mm dia), yellow
56865	1 drill, size 2 (1.6 mm dia), red
56866	1 drill, size 3 (1.9 mm dia), blue
56867	1 Universal drill
<b>RelyX™ Unicem 2 Automix Self-Adhesive Resin Cement</b>	
	<b>Refills</b> – each containing 1 syringe – 8.5g, 10 mixing tips (regular), 5 mixing tips (wide), 5 endo tips
56846	<b>Shade A2 Universal</b>
56847	<b>Shade A3 Opaque</b>
56848	<b>Shade Translucent</b>
	<b>Value Packs</b> – each containing 3 syringes – 8.5g, 30 mixing tips (regular), 20 mixing tips (wide), 15 endo tips, 5 intra-oral tips
56849	<b>Shade A2 Universal</b>
56857	<b>Shade A3 Opaque</b>
56858	<b>Shade Translucent</b>
<b>Accessories – Cement Mixing Tips</b>	
56851	30 mixing tips (regular)
56919	15 mixing tips (wide)
56918	15 mixing tips (wide), 15 intra-oral tips

Item No.	Product Information	
Scotchbond™ Universal Adhesive		
41254	Introductory Kit: 1 vial Scotchbond™ Universal Adhesive – 5 ml, 50 disposable mixing wells, 50 disposable microbrushes, 1 etchant syringe – 3 ml, 25 dispensing tips for etchant syringe	
41258	Refill: 1 vial Scotchbond™ Universal Adhesive – 5 ml	
Filtek™ Bulk Fill Posterior Restorative		
Refill	Item No.	Item No.
Shade	Syringe incl. 1 syringe – 4 g	Capsule incl. 20 capsules – 0.2g each
A1	4863A1	4864A1
A2	4863A2	4864A2
A3	4863A3	4864A3
B1	4863B1	4864B1
C2	4863C2	4864C2
Filtek™ Bulk Fill Flowable Restorative		
Refill	Item No.	Item No.
Shade	Syringe incl. 2 syringes – 2 g each, 20 dispensing tips	Capsule incl. 15 capsules – 0.2 g each
A1	4862A1	4861A1
A2	4862A2	4861A2
A3	4862A3	4861A3
Universal	4862U	4861U
3740T	20 dispensing tips 19-gauge	



**3M Oral Care**  
2510 Conway Avenue  
St. Paul, MN 55144-1000  
USA  
1-800-634-2249

**3M Canada**  
Post Office Box 5757  
London, Ontario N6A 4T1  
Canada  
1-888-363-3685

**3M.com/RelyX3D**

3M, “3M Science. Applied to Life.”, ESPE, Filtek, RelyX and Scotchbond are trademarks of 3M Company or 3M Deutschland GmbH. Used under license in Canada. All other trademarks are owned by other companies. Please recycle. Printed in U.S.A.  
© 3M 2015. All rights reserved. 70-2013-0726-4 rev 8/2016