


Fast sensitivity relief. Slow fluoride release.


Vanish™ XT Extended Contact Varnish

Long-term protection

Vanish™ XT Extended Contact Varnish seals and protects dentin tubules

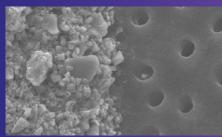

SEM image of exposed dentin tubules (right) and Vanish[™] XT Extended Contact Varnish (left) at 3000x magnification illustrates the barrier effect.

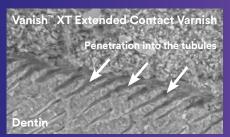

Cross-sectional SEM image of Vanish[™] XT Extended Contact Varnish coated on dentin at 1000x magnification illustrates the resin tags penetrating the tubules.



Vanish⁵ XT Extended Contact Varnish is virtually invisible when applied to the tooth surface.

The protective barrier that immediately relieves sensitivity, delivers fluoride and lasts 6 months.

Vanish[™] XT Extended Contact Varnish is a site-specific, light cured, durable coating that provides an immediate layer of protection to relieve dentinal hypersensitivity.

- Great for protecting exposed, sensitive root surfaces, especially newly cleaned surfaces following scaling and root planing
- Remains on the tooth for 6 months or longer
- Moisture tolerant: no need to dry or pre-treat the root surface
- Does not require the use of a rubber dam
- Delivers fluoride, calcium and phosphate


Vanish XT reduces dentin permeability by 88%¹

Occluding dentinal tubules in-office provides immediate relief from pain associated with the movement of fluid through open tubules. In addition to creating a protective layer on exposed dentin, Vanish XT has been shown to both penetrate and seal tubules. Laboratory testing demonstrates that Vanish XT varnish reduces dentin permeability by 88%.

Vanish XT protects the tooth surface from acid attack

In vitro tests conducted with Vanish XT demonstrate that the coating:


- Creates a barrier against demineralization and acid erosion²
- Provides protection beyond the coating²
- Helps repair demineralized enamel adjacent to the coating³

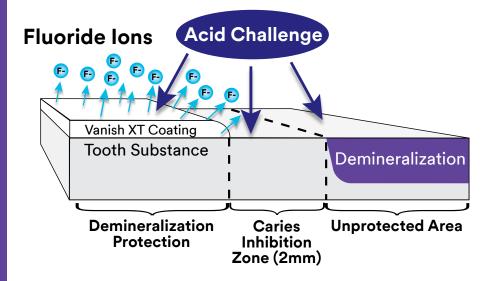


Immediate relief

Vanish™ XT Extended Contact Varnish contains fluoride and is "recharged" by fluoride toothpaste

Vanish[™] XT Extended Contact Varnish is a resin-modified glass ionomer material that releases fluoride, calcium and phosphate. This makes it unique versus other in-office desensitizers. In the first 24 hours, Vanish XT releases more fluoride than a conventional fluoride varnish. Plus, Vanish XT is recharged every time the patient brushes with fluoride toothpaste. Your patients not only get the protection of a physical barrier, they get the benefit of long-term fluoride delivery.

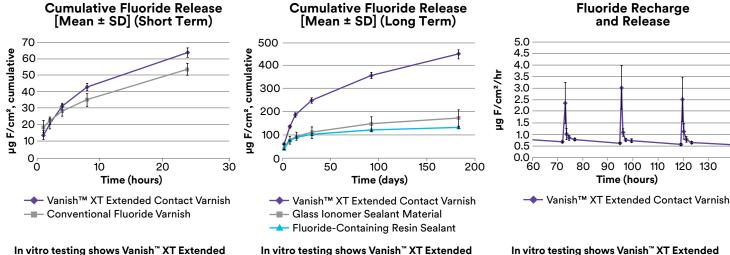

In vitro testing shows Vanish⁻⁻ X I Extended Contact Varnish releases more fluoride over 24 hours than a conventional varnish.

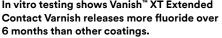



A STATUTE

Source: 3M internal data

NA ESPE

The Complete Solution—check out the differences

Existing dentinal hypersensitivity relief products offer multiple benefits, but no other product offers all the advantages of Vanish XT Varnish.

	Immediate Sensitivity Relief	Long-Term Coating	Moisture Tolerant	Fluoride Release	Calcium Release
Vanish [™] XT Extended Contact Varnish	1	1	\checkmark	\checkmark	1
Glutaraldehyde-based Desensitizers (e.g. Gluma [™] , Calm-itº)	\checkmark	\checkmark			
Resin-based Desensitizers (e.g. Seal & Protect [™])	\checkmark	\checkmark			
Conventional Fluoride Varnishes (e.g. Duraphat®)	\checkmark		\checkmark	\checkmark	

Rusin et al. (2008). Dentin Permeability of a New Protective Coating Material. J. Dent. Res. 87 (Spec Iss A): #1041.
Rusin et al. (2008). Demineralization Protection of a New Protective Coating. J. Dent. Res. 87 (Spec Iss B): #627.
33 M internal data

Vanish[™] XT Extended Contact Varnish is easy to use and does not require the use of a rubber dam.

Vanish[™] XT Extended Contact Varnish Exposed Root Surface Sensitivity Application



1. Clean the tooth surface



5. Mix for 15 seconds (2.5-minutes working time)



2. Rinse with water



6. Apply thin layer to tooth surface



3. Remove excess, pooled water



7. Light cure for 20 seconds



4. Dispense onto mixing pad



8. Wipe the coating with a moist cotton applicator

Ordering Information



Product Description Vanish™ XT Extended Contact Varnish 1 - 10g Clicker™ Dispenser

To learn more about using Vanish XT for protecting enamel surfaces and for patients with orthodontic brackets, visit

www.3M.com/PreventiveCare



3M Oral Care 2510 Conway Avenue St. Paul, MN 55144-1000 USA

Phone 1-800-634-2249 Web 3M.com/dental 3M Canada Post Office Box 5757 London, Ontario N6A 4T1 Canada Phone 1-888-363-3685 3M, 3M Science. Applied to Life., ESPE, Clicker and Vanish are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. Printed in USA. © 3M 2016. All rights reserved. 70-2013-0148-1