

INSTRUCTIONS FOR USE

Preventives



Sparkle® Propy Paste with Xylitol, Sparkle® Free Propy Paste with Xylitol

Instructions for use:

Using a prophy angle, fill the cup by pressing the face of the cup into past while running the angle at low speed. Keep sufficient paste in the cup to avoid excessive dilution and swallowing of the paste. Saliva evacuation is recommended.

Caution: For professional use only. Federal Law (USA) prohibits dispensing without a prescription. Keep away from children. Dental prophylaxis should be performed by professionally trained individuals only. Do not swallow. Nausea can result from excessive ingestion of fluoride.



Ref.#'s UPCB, UPCBG, UPCCH, UPCM, UPCOV, UPCA, UPMB,
UPMBG, UPMCH, UPM, UPMOV, UPMA, UPFBG, UPSFCC, UPSFCF,
UPSFC, UPSFCWC, UPSFMC, UPSFMF, UPSFMS, UPSFMWC

Sparkle® is a registered trademark of Crosstex International, Inc.



CROSSTEX
A CANTEL MEDICAL COMPANY

Manufactured for: Crosstex International, Inc.

10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | Crosstex@crosstex.com

Made in USA

UP-IFU
Rev. 0 - 3/17