



**MYCO**  
**MEDICAL**

**TheSharpChoice®**

NMSDC Certified Diversity Supplier

DATASHEET

## QLICKSMART® BladeFLASK Scalpel Blade Removal System

The QlickSMART® BladeFLASK is the world's first single-handed scalpel blade remover.



- Holds 100 blades and is puncture proof
- OSHA compliant with single-handed activation when used with reusable QlickSMART® BladeFLASK Mounting Bracket (*sold separately*)
- Mount on a wall or desk with the reusable QlickSMART® BladeFLASK Mounting Bracket (*sold separately* — adhesive backing and screw mounts included)
- Audible click ensures blade removal
- Built-in counter with automatic shutoff when full
- Compatible with most brands of general surgery blades and flat handles



NO FINGERS



NO FORCEPS



NO RESHEATHING



NO INJURY

| MYCO CODE | DESCRIPTION   | UNIT OF MEASURE |
|-----------|---|-----------------|
| QFRGEN    | BladeFLASK — Non-Sterile  | 12 ea/case      |
| QBRGEN    | BladeFLASK Mounting Bracket — Non-Sterile<br>(Reusable, includes adhesive backing and screw mounts) | 24 ea/case      |

## How to Use:

### PARTS OF YOUR BLADEFLASK:

**INDICATOR:** SHOWS WHICH WAY SHARP EDGE MUST FACE.

THIS PART OF THE SCALPEL HANDLE IS CALLED THE TANG.

COUNTER

LANDING PAD

MOUNTING BRACKET

**CLICKSMART**  
Simply safer

### MOUNT TO WALL OR TROLLEY:

- If you haven't mounted your BladeFLASK before, prepare mounting bracket.
  - BRACKET HAS DOUBLE SIDED TAPE ON BOTTOM;
  - OR USE SCREWS.

\* See fitting template and instructions inside the bracket packaging.
- BladeFLASK can be mounted on a wall or on a trolley or bench.

**NOTE:** BladeFLASK is mounted on Mounting Bracket during use.

### REMOVE BLADE SINGLE HANDED ACTION:

- Place blade on landing pad with the sharp edge facing to the right (or 'tang' up).
- Push scalpel handle into BladeFLASK until you hear a click.
- Remove handle.

**NOTE:** BladeFLASK is disposed of as medical waste after 100 blades have been inserted.

As a leading NMSDC Diversity Certified Supplier, MYCO Medical is committed to “Bringing Reliability, Innovation and Value” to the Healthcare industry with an emphasis on “physician preference” surgical devices and disposables.

## To order, learn more or receive complimentary evaluation samples:

Please call MYCO Medical at (800) 454-6926 or (919) 460-2535 or email us at [info@mycomedical.com](mailto:info@mycomedical.com)



2015 Production Drive  
Apex, NC 27539 USA

[info@mycomedical.com](mailto:info@mycomedical.com)  
[www.mycomedical.com](http://www.mycomedical.com)

See the complete line of MYCO innovative disposable and reusable medical devices at  
[www.mycomedical.com](http://www.mycomedical.com)

Some medical devices are not licensed for sale in Canada.

© 2017 MYCO Medical, Inc. All rights reserved.  
LIT-00217 Rev 1 Eff 02.03.2017