



2017 INFECTION PROTECTION CATALOG

CROSSTEX®
A CANTEL MEDICAL COMPANY

For over 60 years Crosstex has been making vital contributions to the healthcare community by designing and manufacturing innovative infection prevention and control products. Crosstex serves the global healthcare community from strategically located manufacturing and distribution facilities all around the world and Crosstex is proud to manufacture the majority of its products in the U.S.A.



Focus on Innovation

Striving to continuously develop unique and innovative new products is the cornerstone of our healthcare mission. We are proud to serve the healthcare community as a leading global manufacturer providing solutions for better infection control, prevention and safety.

Among our best-selling patented, patent pending and innovative products:



• **Secure Fit® Technology** Face Masks are patent-pending masks that feature aluminum nose and chin pieces to provide healthcare workers with a mask that can be adjusted to fit any size or shaped face, significantly reducing gapping on the sides and bottom of the mask.

• **Sure-Check®** Sterilization Pouches are patent pending, with multi-parameter internal and external indicators that only change color when all three criteria for sterilization have been met.

• **STEAMplus™** Type 5 Integrators provide a distinct pass/fail result allowing you to release non-implant loads before receiving spore test results.



• **DentaPure®** iodinated resin bead cartridges are easily installed on any dental unit waterline (bottle or municipal) to kill bacteria and keep water safe for an entire year.

• **Sani-Tab®** Patient Towels with adhesive tabs, cost effectively eliminate bib clips and chains to avoid cross-contamination.

• **Comfort Plus®** Saliva Ejectors won't lock onto tissue thus protecting delicate mucosal tissue and are available in bubble gum, mint or unscented.

• **Sparkle®** Air Water Syringe Tips do not require any adapter and feature an exclusive Quad Core® air channel for air/water separation.



NEW! • **twist® plus Propphy Angles**, with 120° patented oscillating movement, provide more efficient and ergonomic cleaning.



• **Barrier Film** with 3/16" non-adhesive finger-lift edges allow for quick, easy placement and removal, without leaving behind surface residue.

• **Ambi-ance®** Ambidextrous Nitrile Gloves are powder free and provide maximum comfort and performance with improved dexterity and tactile sensitivity.

• **iBarrier®** Touch sensitive disposable sleeves are glove compatible for tablets and remote controls. Protect your equipment, patients and staff from cross-contamination.

Table of Contents



Personal Protection Page 1



Operatory Protection Page 15



Sterilization Page 20



Treatment Accessories Page 24



Surface Disinfectants Page 28



Cleaning Solutions Page 31



High-Level Disinfectant Page 32



Preventives Page 34



Imaging Comfort Page 37



Cotton/Non-Wovens Page 39



Surgical & Waste Accessories Page 42



Waterline Treatment Page 44

See page 46 for index to products.

Key to Symbols



All Crosstex products are 100% quality satisfaction guaranteed.



Look for this symbol throughout our catalog. It represents products unique to Crosstex offering special benefits for your practice and patients!



Crosstex is an industry leader in manufacturing environmentally friendly, biodegradable products. Look for this symbol next to Crosstex Green products!



Crosstex products bearing this symbol are proudly manufactured in the USA.



Crosstex products bearing this symbol are 100% non-latex (not made with natural rubber latex).



Single use product.



Crosstex products bearing this symbol are 100% gluten free.

WHY CROSSTEX®?

Enhanced Safety and Performance!

Crosstex offers a wide variety of mask designs, fit and filtration to match the protection needs for each procedure or risk level. The Crosstex MaskEnomics® filtration guide makes it easy to select the level of filtration required, including ASTM Level 1, 2 and 3 and N95 particulate respirator. From minimum performance masks for dry, short procedures, to our maximum filtration mask, Isolator Plus, Crosstex has the right mask for the task!



See page 5 for ordering information.



MAXIMUM FILTRATION

Indicated for use when treating patients with airborne diseases such as TB or influenza.

Meets EN 149: 2001 FFP2 NR

N95



Isolator® Plus N95 Particulate Respirator

- NIOSH approved N95 Respirator.
- Meets guidelines for TB exposure control
- Resistance to penetration by synthetic blood: 160mmHg.
- PFE = 99.9% @ 0.1 micron.
- Malleable nose and chin pieces.
- Inner/outer fluid resistant layers.
- Chin pocket construction.

Pictured: GPRN95



ASTM LEVEL 3

Ideal for procedures where heavy to moderate amounts of fluid, spray and/or aerosols are produced.

Meets EN14683 Rating – Type IIR Standard.

ASTM 3



Ultra® Sensitive Earloop w/ Secure Fit® Technology

- BFE = 99.9% @ 3 microns.
- PFE = 99.8% @ 0.1 micron.
- Patent pending design with aluminum strip under chin for custom fit.
- Fluid resistant outer layer.
- White hypoallergenic inner cellulose layer.
- Extra soft – will not lint, tear or shred.
- Free of chemicals, inks and dyes.
- Extra-long aluminum nose piece.

Pictured: GCFCXSSF



ASTM LEVEL 2

Ideal for procedures where moderate to light amounts of fluid, spray and/or aerosols are produced.

Meets EN14683 Rating – Type IIR Standard.

ASTM 2



Procedural Earloop w/ Secure Fit® Technology

- BFE = 99.9% @ 3 microns.
- PFE = 99.7% @ 0.1 micron.
- Patent pending design with aluminum strip under chin for custom fit.
- Fluid resistant outer layer.
- Fluid resistant white inner layer.
- Extra-long aluminum nose piece.

Pictured: GCPBLSF



ASTM LEVEL 1

Ideal for procedures where low amounts of fluid, spray and/or aerosols are produced.

Meets EN14683 Rating – Type II Standard.

ASTM 1



Isofluid® Earloop w/ Secure Fit® Technology

- BFE = 99.1% @ 3 microns.
- PFE = 98.9% @ 0.1 micron.
- Patent pending design with aluminum strip under chin for custom fit.
- Fluid resistant outer layer.
- White tissue inner layer.
- Extra-long aluminum nose piece.

Pictured: GCIPKSF



LOW PERFORMANCE

Ideal as a comfortable substitute for earloop face masks, this mask is a simple physical barrier ideal for exams and visitations or for dry, short procedures that do not produce fluid, spray or aerosols.



Surgical Molded

- BFE = 98.7% @ 4.2 microns.
- Splash resistant barrier.
- Flared edge prevents irritation.
- Soft, flexible nose piece.

Pictured: GCBL



MINIMUM PERFORMANCE

Ideal as a simple physical barrier for exams, isolation and visitations or for dry, short procedures that do not produce fluid, spray or aerosols.



Isolite® Earloop

- BFE = 98% @ 1.0 micron.
- Outer facial tissue layer.
- White facial tissue inner layer.
- Extra-long aluminum nose piece.

Pictured: GCLPK



Experience Secure Fit® for yourself.

Better Fit. Better Filtration. Better Protection.

3X's More Protection*

- ➔ Aluminum strips both above the nose and below the chin for a custom fit. Conforms to any size or shape face.
- ➔ Patent pending design significantly reduces gapping at top, bottom and sides.

*Study on File



Although the N95 closely resembles a surgical mask, it is a respirator and must be used in accordance with all OSHA regulations regarding respiratory protection.



FEATURE GUIDE TO CROSSTEX EARLOOP/TIE-ON FACE MASKS

	A	B	C	D	E	F	G	H	I	J	K	L	M
Ultra® Sensitive Earloop w/ Secure Fit® Technology	•	•				•	•	•	•				•
Ultra® Sensitive FogFree™ Earloop w/ Secure Fit® Technology	•	•		•	•	•	•	•	•				•
Ultra® Sensitive FogFree™ Earloop w/ Shield w/Secure Fit® Technology	•	•		•	•	•	•	•	•				•
Ultra® Earloop w/ Secure Fit® Technology	•	•				•	•	•				•	•
Ultra® Sensitive Earloop	•	•				•	•	•	•				•
Ultra® Sensitive FogFree™ Earloop	•	•		•	•	•	•	•	•				•
Ultra® Sensitive FogFree™ Earloop w/ Shield	•	•		•	•	•	•	•	•				•
Ultra® Earloop	•	•				•	•	•				•	•
Ultra® FogFree™ Earloop	•	•				•	•	•	•				•
Ultra® FogFree™ Earloop w/ Shield	•	•		•	•	•	•	•	•				•
Procedural Earloop w/ Secure Fit® Technology	•	•				•	•	•				•	•
Procedural Earloop	•	•				•	•	•				•	•
Surgical Tie-on	•	•						•	•			•	•
Isofluid® Earloop w/ Secure Fit® Technology	•	•				•	•	•					•
Isofluid FogFree® Earloop w/ Secure Fit® Technology	•	•				•	•	•					•
Isofluid FogFree® Earloop w/ Shield w/Secure Fit® Technology	•	•				•	•	•					•
Isofluid® Earloop	•	•				•	•	•					•
Isofluid FogFree® Earloop	•	•				•	•	•					•
Isofluid FogFree® Earloop w/ Shield	•	•				•	•	•					•
Isofluid® Plus Earloop	•	•				•	•	•					•
Isofluid® Plus Pure w/ Secure Fit® Technology	•	•				•	•	•					•
Isolite® Earloop	•	•	•			•	•	•					•



Ultra® Sensitive FogFree™ Earloop w/ Secure Fit® Technology



- BFE = 99.9% @ 3 microns.
- PFE = 99.8% @ 0.1 micron.
- Provides all the features of Ultra® Sensitive with Secure Fit® Technology with FogFree™ strip on inner layer.

Pictured: GCFCXSFSF  



Ultra® Sensitive FogFree™ Earloop w/ Shield w/ Secure Fit® Technology



- BFE = 99.9% @ 3 microns.
- PFE = 99.8% @ 0.1 micron.
- Provides all the features of Ultra® Sensitive FogFree™ with Secure Fit® Technology with attached optically clear, distortion-free shield.

Pictured: GCPWSSF  



Ultra® Earloop w/ Secure Fit® Technology

- BFE = 99.9% @ 3 microns.
- PFE = 99.8% @ 0.1 micron.
- Patent pending design with aluminum strip under chin for tight, custom fit.
- Fluid resistant white inner layer.
- Extra-long aluminum nose piece.
- Fluid resistant outer layer.

Pictured: GCFCXUSF  



Procedural Earloop


- BFE = 99.9% @ 3 microns.
- PFE = 99.7% @ 0.1 micron.
- Fluid resistant outer layer.
- Fluid resistant white inner layer.
- Extra-long aluminum nose piece.

Pictured: GCPLV  



Surgical Tie-on

- BFE = 99.8% @ 3 microns.
- PFE = 99.5% @ 0.1 micron.
- Fluid resistant outer layer.
- Fluid resistant white inner layer.
- Extra-large panel for surgical procedures.
- Extra-long aluminum nose piece.

Pictured: GCS 



Isofluid® Earloop



- BFE = 99.1% @ 3 microns.
- PFE = 98.9% @ 0.1 micron.
- Fluid resistant outer layer.
- White tissue inner layer.
- Extra-long aluminum nose piece.

Pictured: GCISA  



Isofluid FogFree® Earloop w/ Secure Fit® Technology



- BFE = 99.1% @ 3 microns.
- PFE = 98.9% @ 0.1 micron.
- Provides all features of Secure Fit® Isofluid with addition of FogFree® strip on inner layer.

Pictured: GCICXBSF  



Isofluid FogFree® Earloop w/ Shield w/ Secure Fit® Technology

- BFE = 99.1% @ 3 microns.
- PFE = 98.9% @ 0.1 micron.
- Provides all features of Secure Fit® Isofluid FogFree® with attached optically clear, distortion-free face shield.

Pictured: GCIPWBSF  

Also available:

Isofluid FogFree® Earloop
Isofluid FogFree® Earloop with Shield

Additional Face Protection Products



Face Shield

- Optically clear, distortion-free wrap-around face shield.
- 1.5" foam headband holds shield away from face; "floats" lightly on forehead, with no pressure on temples; vented for increased air flow.
- Protects mask and face from direct splatter; may prolong mask life.
- Sonically welded elastic headband for added strength.
- Anti-fog treatment on inside and outside of shield.

Pictured: GCSS  




Le Petit Masque

- Fluid Resistance - 80mmHg.
- BFE >98.4% @ 3 microns.
- PFE=98.5% @ 0.1 micron.
- Delta P (breathability) <4.0 H₂O/cm².
- Flame Spread Class 1.
- Measures 6 1/4" x 3 1/2" / 15.875 x 8.89cm.
- Fluid resistant pink outer layer.
- Soft white spunbond inner layer.
- ASTM Level 1

Pictured: GCAPTPK  

Mask performance levels indicated on pages 1, 2 & 3 were confirmed by tests performed for Crosstex by Nelson Laboratories in accordance with ASTM Standard F2100-11 and represent the most current performance test data (of these Crosstex masks).

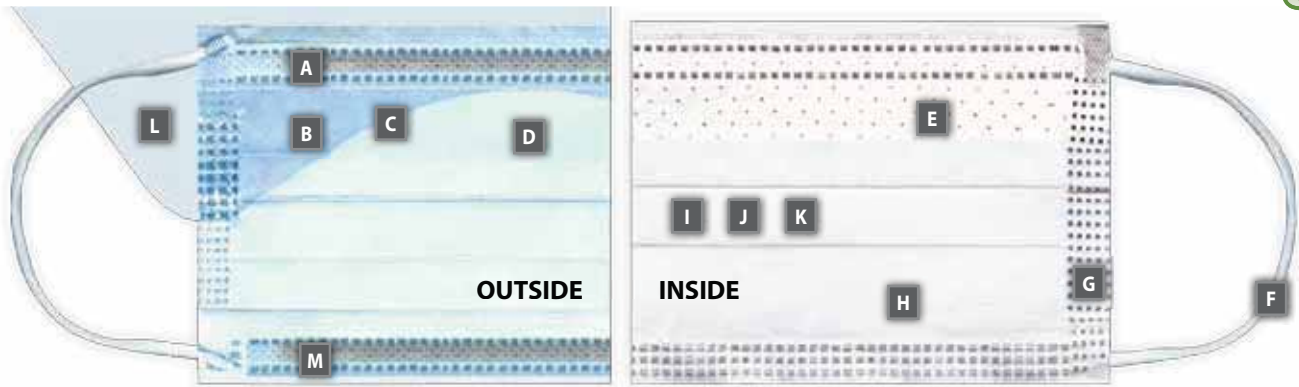

A Extra-long, enclosed aluminum nose piece forms a strong seal for maximum protection.

B Spunbond outer layer material for maximum protection against fluids.

C Facial tissue outer layer is ultrasoft and odorless.

D Unique vapor barrier placed on outside layer to prevent fogging.

E FogFree / No-Fog strip blocks and absorbs moisture; forms a strong seal to prevent fogging; cushions nose ridge.

F Secure Loop earloops are soft, strong, fit comfortably, and attached to outside of mask to eliminate irritation.

G SOF-BOND® ultrasonics ensures strongest construction, eliminating holes and defects.

H Highest quality filter media for best ASTM rating, filtration value and breathability.

I Cellulose inner layer for sensitive skin (white, extra soft, hypoallergenic) will not lint, tear or shred; void of dyes, inks and chemicals, which can cause irritation.

J Tissue inner layer is made of white, soft facial tissue.

K Spunbond inner layer made of white, fluid-resistant material.

L Attached wrap-around face shield is optically clear and distortion-free.

M Secure Fit® aluminum strip under chin significantly reduces gapping at the top, sides and bottom of the mask.

All Crosstex face masks are not made with natural rubber latex and are not reusable.



Ultra® Sensitive Earloop

- BFE = 99.9% @ 3 microns.
- PFE = 99.8% @ 0.1 micron.
- Fluid resistant outer layer.
- White hypoallergenic inner cellulose layer.
- Extra soft – will not lint, tear or shred.
- Free of chemicals, inks and dyes.
- Extra-long aluminum nose piece.

Pictured: GCFCXS



Also available:

Ultra® Sensitive FogFree™ Earloop

Ultra® Sensitive FogFree™ Earloop with Shield



Ultra® Earloop

- BFE = 99.9% @ 3 microns.
- PFE = 99.8% @ 0.1 micron.
- Fluid resistant outer layer.
- Fluid resistant white inner layer.
- Extra-long aluminum nose piece.

Pictured: GCFCXU



Ultra® FogFree™ Earloop w/ Shield

- BFE = 99.9% @ 3 microns.
- BFE = 99.8% @ 0.1 micron.
- Fluid resistant outer layer.
- Fluid resistant white inner layer.
- Unique FogFree™ shield with optically clear, distortion free wrap around shield.

Pictured: GCPW



Isofluid® Plus Earloop

- BFE = 99.8% @ 3 microns.
- PFE = 99.1% @ 0.1 micron.
- Fluid resistant outer layer.
- White hypoallergenic inner cellulose layer.
- Extra-long aluminum nose piece.
- Extra soft – will not lint, tear or shred.

Pictured: GPLUSPK



Isofluid® Plus Pure w/Secure Fit® Technology

- BFE = 99.8% @ 3 microns.
- PFE = 99.1% @ 0.1 micron.
- Fluid resistant outer layer.
- White hypoallergenic inner cellulose layer.
- Extra-long aluminum nose piece.
- Extra soft – will not lint, tear or shred.

Pictured: GPLUSWHF



Patient Protection

See-Clear® Eyeglass Wipes

- Effective cleaning for eyewear and face shields.
- 4" x 5.5" wipes are anti-fog, anti-static.
- Will not scratch, dries quickly.

Ref# SEEC



Patient Safety Mask with Shield

- U.S. Patent No's 6,185,740 and 6,523,170.
- Covers patient's eyes and nose.
- Protects fragile eye and mucosal tissue from flying particles, spray and splatter.
- Makes patients feel safe, but not "in the dark," with attached clear-vision shield.
- Form-fitting profile, flexible materials provide maximum access to mouth.
- White spunbond inner and outer layers.
- Fluid resistant.

Pictured: GCPAT



Award Winning Crosstex Masks



2008 2011
2009 2012
2010 2014



The Dental Advisor:
Top Masks:

► **Secure Fit®**

2012
2013
2014
2015



The Dental Advisor:
► Patient Safety Mask with Shield
++++



► Innovative Products

FACE PROTECTION

MASKENOMICS® 101

NEW ASTM F2100-11 STANDARDS


Understanding ASTM Face Mask Performance Levels

Selecting the appropriate mask for a particular procedure is a critical component of your Personal Protective Equipment (PPE) protocol. Although masks may look similar, each mask has notable differences affecting the quality and level of filtration. Understanding the ASTM performance level of each face mask can help make the selection

process easier and ensure your mask will provide the appropriate filtration.

Medical face mask materials covered under ASTM specifications are designated as one or more of the following performance levels based on the barrier performance properties of the materials used:

	ASTM Level 1	ASTM Level 2	ASTM Level 3
FLUID RESISTANCE, mmHg	80	120	160
BFE	≥ 95%	≥ 98%	≥ 98%
PFE, @ 0.1 micron	≥ 95%	≥ 98%	≥ 98%
DELTA P, mm H ₂ O/cm ²	< 4.0	< 5.0	< 5.0
FLAME SPREAD	Class 1	Class 1	Class 1

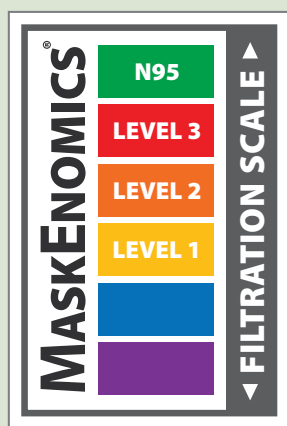

Crosstex makes the selection process easy with the MaskEnomics® rating system, designed to assist you in the decision making process by categorizing all Crosstex face masks by the level of filtration they provide... from minimum to maximum performance. Just look to the symbols displayed to find the appropriate level of filtration and protection for the procedure to be performed.

What the terms mean:

FLUID RESISTANCE* represents the mask's resistance to penetration by synthetic blood under pressure (mmHg). It measures the ability of a mask's material construction to minimize the saturation of fluids traveling through the material and potentially coming into contact with the wearer. The higher the fluid resistance (filtration), the better the protection.

BFE (Bacterial Filtration Efficiency) represents the percentage of aerosol particulates filtered at a size of 3 microns. It is the measure of the efficiency of the mask in filtering bacteria passing through it.

PFE (sub-micron Particulate Filtration Efficiency) represents the percentage of submicron particulates filtered at 0.1 micron. PFE is the measure of the efficiency of the mask in filtering particles passing through it. The size of the particles filtered is critical.

DELTA P (Differential Pressure) represents the pressure drop across the mask or resistance to air flow in mmH₂O/cm². This determines breathing resistance – the higher the Delta P, the less the breathability, but the better the filtration.

FLAME SPREAD measures flame spread of the mask material.

SOURCE: ASTM Standard specification for performance of materials used in medical face masks – F2100-11 Standard.

*As specified in Test Method F 1862.



FACE PROTECTION

Ordering Information

	Description	Quantity	Ref. #		Description	Quantity	Ref. #
	Isolite® Earloop			ASTM Level 2	Procedural Earloop (continued)		
	Blue	50/Box	GCLBL		Pink	50/Box	GCPPK
	Pink	50/Box	GCLPK		Yellow	50/Box	GCPYE
	Yellow	50/Box	GCLYE		Blue, Bulk Pack NEW!	100/Box	GCPBL100
	Surgical Molded				Surgical Tie-on		
	Blue	50/Box	GCBL		Blue	50/Box	GCS
	Pink	50/Box	GCPK				
ASTM Level 1	Isofluid® Earloop w/ Secure Fit® Technology			ASTM Level 3	Ultra® Sensitive Earloop w/ Secure Fit® Technology		
	Blue	50/Box	GCIBLSF		White	50/Box	GCFCXSSF
	Lavender	50/Box	GCILVSF		Ultra® Sensitive FogFree™ Earloop w/ Secure Fit® Technology		
	Pink	50/Box	GCIKPSF		White	40/Box	GCFCXSFSF
	Isofluid FogFree® Earloop w/ Secure Fit® Technology				Ultra® Sensitive FogFree™ Earloop w/ Shield w/ Secure Fit® Technology		
	Blue	40/Box	GCICXBSF		White	25/Box	GCPWSSF
	Isofluid FogFree® Earloop w/ Shield w/ Secure Fit® Technology				Ultra® Earloop w/ Secure Fit® Technology		
	Blue	25/Box	GCIPWBSF		Blue	50/Box	GCFCXUSF
	Isofluid® Earloop				Ultra® Sensitive Earloop		
	Blue	50/Box	GCIBL		White	50/Box	GCFCXS
	Green	50/Box	GCIGR		Ultra® Sensitive FogFree™ Earloop		
	Lavender	50/Box	GCILV		White	40/Box	GCFCXS
	Pink	50/Box	GCIPIK		Ultra® Sensitive FogFree™ Earloop w/ Shield		
	Sapphire	50/Box	GCISA		White	25/Box	GCPWS
	Teal	50/Box	GCITE		Ultra® Earloop		
	Topaz	50/Box	GCITZ		Blue	40/Box	GCFCXU
	Turquoise	50/Box	GCITQ		Ultra® FogFree™ Earloop		
	Blue, Bulk Pack NEW!	100/Box	GCIBL100		Blue	40/Box	GCFCX
	Isofluid FogFree® Earloop			N95	Ultra® FogFree™ Earloop w/ Shield		
	Blue	40/Box	GCICXB		Blue	25/Box	GCPW
	Sapphire	40/Box	GCICXS		Isolator Plus N95 Particulate Respirator		
	Topaz	40/Box	GCICXZ		Blue/White Stripe	28/Box	GPRN95
	Turquoise	40/Box	GCICXT		Face Shield		
	Isofluid FogFree® Earloop w/ Shield				¾ Length(7") Clear Face Shield	24/Box	GCSS
	Blue	25/Box	GCIPWB		¾ Length(7") Clear Face Shield, bulk	72/Box	GCSSB
	Sapphire	25/Box	GCIPWS		Full Length(9") Clear Face Shield	18/Box NEW!	GCSS9
	Topaz	25/Box	GCIPWZ		Full Length(9") Clear Face Shield, bulk	72/Box NEW!	GCSS9B
	Turquoise	25/Box	GCIPWT		Le Petit Masque		
	Isofluid® Plus Earloop				Pink	50/Box	GCAPTPK
	Blue	50/Box	GPLUSBL		Patient Safety Mask w/ Shield		
	Kaleidoscope	50/Box	GPLUSKA		White	25/Box	GCPAT
	Pink	50/Box	GPLUSPK		See-Clear® Eyeglass Wipes		
ASTM Level 2	Isofluid® Plus Pure w/ Secure Fit® Technology					120/Box	SEEC
	White NEW!	50/Box	GPLUSWHSF				
	Procedural Earloop w/ Secure Fit® Technology						
	Blue	50/Box	GCPBLSF				
	Lavender	50/Box	GCPLVSF				
	Pink	50/Box	GCPPKSF				
	Procedural Earloop						
	Blue	50/Box	GCPBL				
	Lavender	50/Box	GCPLV				


Crosstex can make no warranties or representations, either expressed or implied, that these products will fully protect the user from exposure to blood or bodily fluids or risk of contracting infectious diseases. OSHA requires the employer to evaluate the anticipated exposure and select the appropriate protective masks to prevent contamination of skin, eyes and respiratory passages.

PATIENT TOWELS | BIBS | LINERS

The Crosstex success story began with the development of our first product, the disposable patient towel. For 60 years we have been the benchmark of quality in patient towels, bibs and liners.

Our unique manufacturing process:


- 1 Provides the often imitated, but never duplicated, Crosstex embossing pattern, which allows moisture to flow horizontally across the towel, protecting patient's clothing.
- 2 Offers our exclusive reinforced edge embossing design for added strength and durability where towel clips are placed.
- 3 Bonds the layers of tissue (and poly, when applicable) securely together, eliminating separation of the layers.
- 4 Creates the strength of the towel horizontally through the combination of our proprietary manufacturing equipment and tissue formulation. This prevents tearing when a towel clip is applied.

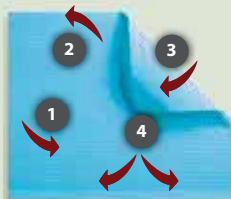

COLOR GUIDE



EMBOSSING PATTERNS

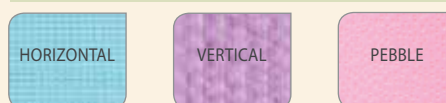

FOLD



Towel Dispenser

Description Ref. #

Clear Lucite PTC
5" x 6" x 7" /
13 x 15 x 18 cm
Holds towels up to
19" x 13" / 48 x 33 cm

Sani-Tab® Chain-Free® Patient Towel


Don't let infection spread at a dangerous clip!



Dental bib clips and chains have been shown to pick up dangerous microorganisms from skin, saliva and plaque, presenting a potential source of cross-contamination in the dental operator. Reduce the risk with Sani-Tab – the only chain-free patient towel with patented adhesive tabs eliminating the need for bib chains and clips. Sani-Tab maximizes infection control and patient comfort while saving time and money!



- Easy to use – simply position towel anywhere on the patient and apply pressure to the tab.
- Bib stays where placed to maximize coverage, unlike bib clips and other disposable towel holders.
- Can be placed high on patient to protect clothing.

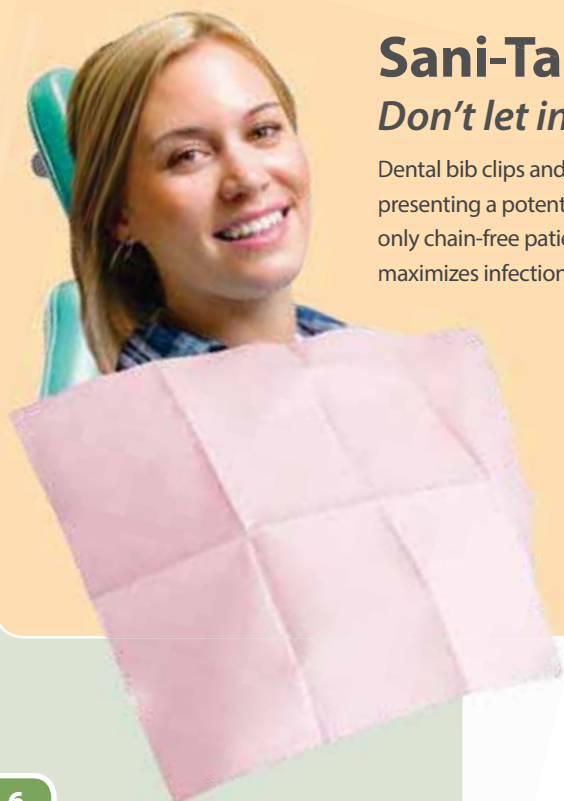PATENTED





PATIENT TOWELS | BIBS | LINERS

 		SIZE	FOLD C-Crosstex V-Vantage	EMBOSSING PATTERN	REF. # / COLOR					
3-PLY TISSUE W/1-PLY POLY										
POLYBACK® 500/Case	19" X 13" 48 X 33 cm	C	HORIZONTAL	WPXAQ WPXDR WPXPE WPXYE	AQUA DUSTY ROSE PEACH YELLOW	WPXBG WPXGR WPXSI WPXFP	BEIGE GREEN SILVER GREY HAPPY FEET	WPXBL WPXLV WPXWH WPXTB	BLUE LAVENDER WHITE TEDDY BEAR	
POLY-GARD® 500/Case	19" X 16" 48 X 41 cm	C	HORIZONTAL	WGXBL	BLUE	WGXWH	WHITE			
POLY-GARD® DELUXE 300/Case	18" X 30" 46 X 76 cm	V	PEBBLE	W3D1930BL	BLUE					
POLYBACK® CONTOUR BIB 400/Case	18" X 22" 46 X 56 cm	V	PEBBLE	WERUECBL WERUECGR	BLUE GREEN	WERUECDR WERUEC	DUSTY ROSE WHITE			
ADVANTAGE PLUS® 500/Case	18" X 13" 46 X 33 cm	V	VERTICAL	WPXAAQ WPXADR WPXAPE WPXAYE	AQUA DUSTY ROSE PEACH YELLOW	WPXABG WPXAGR WPXASI	BEIGE GREEN SILVER GREY	WPXABL WPXALV WPXAWH	BLUE LAVENDER WHITE	
SANI-TAB® CHAIN-FREE® 400/Case	19" X 13" 48 X 33 cm	C	HORIZONTAL	WPXBLT	BLUE	WPXDRT	DUSTY ROSE	WPXWHT	WHITE	
2-PLY TISSUE W/1-PLY POLY										
PINK WITH A PURPOSE ECONOBACK® 500/Case	19" X 13" 48 X 33 cm	C	HORIZONTAL	WEXPP	PINK RIBBON					
ECONOBACK® 500/Case	19" X 13" 48 X 33 cm	C	HORIZONTAL	WEXAQ WEXDR WEXPE WEXYE	AQUA DUSTY ROSE PEACH YELLOW	WEXBG WEXGR WEXSI WEXFP	BEIGE GREEN SILVER GREY HAPPY FEET	WEXBL WEXLV WEXWH WEXTB	BLUE LAVENDER WHITE TEDDY BEAR	
ULTRAGARD® 500/Case	19" X 16" 48 X 41cm	C	HORIZONTAL	WUXBL	BLUE	WUXWH	WHITE	WUXYE	YELLOW	
ADVANTAGE™ 500/Case	18" X 13" 46 X 33 cm	V	VERTICAL	WEXAAQ WEXADR WEXAPE WEXAYE	AQUA DUSTY ROSE PEACH YELLOW	WEXABG WEXAGR WEXASI	BEIGE GREEN SILVER GREY	WEXABL WEXALV WEXAWH	BLUE LAVENDER WHITE	
SANI-TAB® CHAIN-FREE® 400/Case	19" X 13" 48 X 33 cm	C	HORIZONTAL	WEXBLT WEXLVT	BLUE LAVENDER	WEXDRT	DUSTY ROSE	WEXWHT	WHITE	
EXTRA HEAVY TISSUE W/1-PLY POLY										
PROBACK® 500/Case	19" X 13" 48 X 33 cm	C	HORIZONTAL	WEPBAQ WEPBGR WEPBSI WEPBTB	AQUA GREEN SILVER GREY TEDDY BEAR	WEPBBL WEPBLV WEPBWH	BLUE LAVENDER WHITE	WEPBDR WEPBPE WEPBYE WTXSI	DUSTY ROSE PEACH YELLOW SILVER GREY	
3-PLY TISSUE										
PROFESSIONAL® REGULAR 500/Case	19" X 13" 48 X 33 cm	C	HORIZONTAL	WTXAQ WTXDR WTXYE	AQUA DUSTY ROSE YELLOW	WTXBG WTXGR WTXFP	BEIGE GREEN HAPPY FEET	WTXBL WTXWH	BLUE WHITE	
2-PLY TISSUE										
PROFESSIONAL® THRIFT 500/Case	19" X 13" 48 X 33 cm	C	HORIZONTAL	WT5BL	BLUE	WT5WH	WHITE			

APRONS

WAIST LENGTH: 27" x 30" / 69 x 76 cm

Description	Ref. #
Blue Nylon	NN3BL
Blue Plastic	NP3BL
White Plastic	NP3WH
Yellow Plastic	NP3YE

KNEE LENGTH: 27" x 44" / 69 x 112 cm

Description	Ref. #
Blue Nylon	NN4BL
Blue Plastic	NP4BL
Yellow Plastic	NP4YE
White Plastic	NP4WH

ANKLE LENGTH: 36" x 54" / 91 x 137 cm

Description	Ref. #
Blue Plastic	NP5BL
White Plastic	NP5WH



Patient Barrier Aprons Plastic & Nylon

- Impervious to liquids to protect the patient.
- Features two alligator clips attached to the front of the neck band to hold a disposable patient towel/bib and a touch-tab Velcro® neck closure for fast and easy fastening.
- Available in three convenient sizes, barrier aprons may be wiped clean quickly and easily with any antibacterial product or warm soap and water.

Nylon

- Lighter in weight.
- Greater patient comfort.



GOWNS

Description	Quantity	Ref. #
One Size Fits All	10/Poly Bag	NBG



Impervious Barrier Gowns

- Touch-tab Velcro® closures at neckline.
- Hand sewn ties at waist.
- Elastic cuffs to fit over gloves.
- Open back keeps wearer cooler.
- Made of 100% non-woven spun bonded polypropylene with a polyethylene coating.
- Impervious to most liquids and aerosols.

Disposable barrier gowns are not intended for use as a general chemical barrier.





The What, When and Where of Proper Hand Hygiene

Always wash your hands:

- At the beginning and end of the day or shift.
- When hands are visibly soiled or contaminated.
- Before and after eating or using the rest room.

Wash hands:

To remove physical matter and bioburden:

- Plain soaps or antimicrobial soaps may be used.
- Lather for at least 15 seconds to lift contamination off hands.

- Rinse completely to remove soap and contamination.
- Dry completely to avoid chapping.
- Use lotions and protectants to moisturize and heal skin.

Sanitize hands:

To kill unseen microorganisms on otherwise clean, dry hands:

- Dispense SaniTyze™ waterless gel or wipe with Sani-Hands®. Use enough gel for hands to remain wet for at least 15 seconds.
- Rub hands for at least 15 seconds, until hands are dry.

Always perform hand hygiene (wash or sanitize):

- Before/after you come in contact with a patient.
- Before leaving the operatory or lab.
- After contact with inanimate objects that may be contaminated.
- Before donning gloves.
- After removing gloves.

HAND CLEANSERS



SaniSept® Antimicrobial Soap with 0.3% Triclosan

- Proven effective against microbes hidden in skin layers as well as transient microbes.
- Contains Aloe Vera, Vitamin E, Liposomes and Keratin to moisturize skin.
- Effective against harmful bacteria.
- Petroleum free.
- Latex compatible – no lanolin or petroleum-based emollients.
- Nut and gluten free.



Description

18 oz Pump Bottle / 532 ml
Gallon / 3.785L

Ref.

JSOD
JSOG



SaniClenz® Antimicrobial Skin Cleanser with 2% CHG (Chlorhexidine Gluconate)

- As effective as 4% surgical scrubs, yet is gentle to the skin, has a pleasant scent and does not stain.
- Provides persistent, residual protection to the skin and is effective against a broad spectrum of bacteria, fungi and yeast.
- Contains Aloe Vera, Vitamin E, Liposomes and Keratin to moisturize skin.
- Petroleum free.
- Latex compatible – no lanolin or petroleum-based emollients.
- Nut and gluten free.



Description

18 oz Pump Bottle / 532 ml
Gallon / 3.785 L

Ref.

JSCP
JSCG

CHG is more effective than other antimicrobials – It reacts strongly with negatively charged bacterial cells, achieving high Log 10 reduction of skin microbes. When used as a 2% antimicrobial hand wash, CHG is as effective as 4%, but is gentle to skin, has a pleasant odor and does not stain. CHG is long acting. It chemically binds to epithelium, where it remains effective, even in presence of blood (organic material). At the same time, CHG is non-toxic because it has negligible potential for absorption through intact skin.

HAND SANITIZERS

Description

220 Wipe Dispenser Tub
6" x 7.5" / 15 x 19 cm

Ref.

SAND



Sani-Hands® ALC Antimicrobial Alcohol Hand Wipes

- Demonstrated 99.99% effective against 28 bacteria and 5 viruses as well as gram positive and negative bacteria, fungi and yeast.
- Contains 65.9% Ethyl Alcohol meeting CDC recommended guidelines.
- Clinically proven to remove bacteria and soil from hands.
- Contains Aloe Vera and Vitamin E to moisturize, condition, and soften skin.



Description

2 oz Bottle / 60 ml
4 oz Bottle / 118 ml
8 oz Pump Bottle / 237 ml
18 oz Pump Bottle / 532 ml

Ref.

JWG2
JWG
JWG8
JWGP

SaniTyze™ Waterless Moisturizing Antimicrobial Gel

- Won't irritate or dry out skin like soap and water and other hand gels.
- Contains Aloe Vera, Vitamin E, Liposomes and Keratin to moisturize skin.
- Latex compatible—no lanolin or petroleum-based emollients.
- Contains 62% Ethyl Alcohol meeting CDC recommended guidelines.
- Nut and gluten free.



Description

Dispenser
1000 ml Pouch
Promo Pack: 2 Dispensers,
plus 6 - 1000 ml Pouches

Ref.

PGEL1
JWGEL1
JWGPRO1

SaniTyze™ Wall-Mounted Dispenser

- Convenient, wall mounted one-touch dispenser.
- Provides a pre-measured application to minimize waste.
- Easy to use and economical.
- Nut and gluten free.





HAND PROTECTION



AloeCare Plus 3[®] Skin Care Lotion

- Keeps your hands soft and moisturized throughout the day.
- Special, non-greasy formula absorbs quickly into the skin.
- Contains Aloe Vera, Vitamin E, Liposomes and Keratin to keep skin soft and smooth.
- Petroleum free.
- Latex compatible – no lanolin or petroleum-based emollients.
- Nut and gluten free.



Description

18 oz Pump Bottle / 532 ml

Ref.

JLD



OSAP infection control experts recommend using latex compatible lotions and skin protectants throughout the day to protect your most important personal barrier: YOUR INTACT SKIN. Crosstex Hand Care products are latex compatible, nourish skin and promote healing. Each ingredient has a specific purpose, all of which work together as a system. Aloe vera promotes healing. Keratin strengthens nails. Vitamin E promotes the growth of healthy skin tissue. Liposomes enhance moisture retention.

Methods	Crosstex Product	Purpose	Area	Minimum Duration
Routine Hand Wash	SaniSept	Remove debris and harmful microorganisms	All surfaces of the hands and fingers	Lather for 15 seconds
Antiseptic Hand Wash	SaniSept SaniClenz	Remove debris and destroy/reduce harmful microorganisms	All surfaces of the hands and fingers	Lather for 15 seconds
Antiseptic Hand Rub	SaniTyze Sani-Hands	Remove debris and destroy/reduce harmful microorganisms	All surfaces of the hands and fingers	Rub for 15 seconds until hands are dry
Surgical Antisepsis	SaniSept SaniClenz	Remove debris and destroy/reduce harmful microorganisms	Hands	Lather for 2-6 minutes

Modified from Centers for Disease Control and Prevention, "Frequently Asked Questions, Hand Hygiene."
Available at www.cdc.gov/oralhealth/infectioncontrol/faq/hand.htm. February 2006.

WARNING: These products do not eliminate the risk of contracting any disease or infection. For external use only. Follow all instructions and warnings on the use of these products. Must be used in accordance with applicable federal, state or local regulations and guidelines as may be issued from time to time.

LATEX EXAM GLOVES-AMBIDEXTROUS

Gloves are the most important personal barrier to protect against blood-borne pathogen exposure. Every office should have a selection of gloves to meet the needs of each procedure and staff member.

Quality of materials and manufacturing:

All Crosstex gloves meet FDA, ASTM and ISO physical and performance regulations and specifications for medical grade gloves, including:

- Physical dimensions, thickness, tensile strength and modulus.
- Major defects (pinholes, tears), minor defects (visual).

Comfort & size selection, and performance features:

- Crosstex gloves come in a variety of materials, proportions, finishes, textures, and sizes.
- Polymer is added to soothe skin and aid donning of powder-free gloves.
- Low-modulus (high-stretch) materials are balanced with strength.

Units of measure for gloves:

Protein and powder levels are stated per glove, per gram of glove, or per square decimeter (dm²) of surface area. Measurements are calculated using estimates of glove size:

- 1 glove = estimated 7.8 grams.
- 1 glove = estimated 8.9 dm².

Latex vs. non-latex:

Latex allergy and sensitivity is increasing among healthcare workers. Antigenic latex should be limited in latex gloves, and non-latex gloves must be available for sensitive individuals, including patients.

Warning: These products do not eliminate the risk of contracting any disease or infection. Follow all instructions and warnings on the use of these products. Protective gloves must be used in accordance with applicable federal regulations and guidelines as may be issued from time to time. Components used in making gloves may cause allergic reactions in some users. If reactions are suspected, cease using the product and consult a physician. Not intended for use as a general chemical or absolute biological barrier.

Caution: Safe use of latex gloves by or on latex sensitized individuals has not been established. Certain gloves contain natural rubber latex which may cause allergic reactions.

Description	Quantity	Ref. #
Extra Small	100/Box	19080
Small	100/Box	19081
Small/Medium	100/Box	19082
Medium	100/Box	19083
Medium/Large	100/Box	19084
Large	100/Box	19085



Omega-Lightly Powdered

- Textured, natural/white.
- Powder: USP modified corn starch inside, USP calcium carbonate outside.



Description	Quantity	Ref. #
Small	100/Box	MSC
Medium	100/Box	MMC
Large	100/Box	MLC



Classic®-Lightly Powdered

- Smooth, bisque finish.
- Allotex – latex with extremely low antigenic protein.*
- Low modulus – enhanced comfort.
- Powder: USP modified cornstarch.



* 8.8 µg/dm² antigenic protein: ASTM 8.9 and Annex A2 limit = <10 µg/dm²



LATEX EXAM GLOVES-AMBIDEXTROUS



Ultra Plus® -Powder-Free

- Smooth, bisque-finish, no pigment.
- Polymer lined, non-chlorinated, double washed inside & out.



Description	Quantity	Ref. #
Small	100/Box	MSCP
Medium	100/Box	MMCP
Large	100/Box	MLCP



Friction Grip-Powder-Free

- Textured, natural/white.
- Double polymer coated for donning.



Description	Quantity	Ref. #
Extra Small	100/Box	19043
Small	100/Box	19040
Medium	100/Box	19041
Large	100/Box	19042

	AMBIDEXTROUS LATEX EXAM GLOVES				FITTED LATEX GLOVES		NON-LATEX GLOVES (CONTAINS NO LATEX PROTEIN)		
GLOVE	OMEGA	CLASSIC	ULTRA PLUS	FRICTION GRIP	FITTED LATEX	ULTIMATE FITTED	NITRILE FITTED	AMBIDEXTROUS NITRILE	COPOLYMER PLASTIC
POWDERED	●	●			●				
POWDER-FREE			●	●		●	●	●	●
MEETS ASTM D5712 MAX. TOTAL PROTEIN OF < 200 µg/dm ²	●	●	●	●	●	●	N/A	N/A	N/A
POLYMER-COATED		●	●	Inside & Out	Inside & Out	Inside & Out	Inside & Out		
NON-CHLORINATED	●	●	●	●	●	●	●	N/A	N/A
TEXTURE	Textured	Smooth	Smooth	Textured	Textured	Textured	Textured	Textured Fingers	Smooth
COLOR	Natural White	Natural White	Natural White	Natural White	Natural White	Natural White	Blue	Blue	Clear
BEADED CUFF	●	●	●	●	●	●	●	●	N/A

Crosstex Non-Latex Gloves:

- Nitrile fitted gloves have superior strength, fit, and better protection from chemical penetration than latex gloves.
- Copolymer overgloves are clear plastic.

Crosstex Latex Gloves:

- All are natural rubber latex, natural or white color.
- Low antigenic protein (Allotex) reduces latex exposure.
- < 200 mg per dm² for powdered gloves and <50 mg per dm² for powder free gloves.

Powdered vs. Powder-Free:

Powder is used as a donning agent, may be an irritant to some users. Sensitive users are advised to use alternative gloves.

- Crosstex uses minimal USP cornstarch.
- "Powdered" gloves have < 10 mg/dm² powder residue.
- "Powder-free" gloves have < 2 mg powder residue /glove, meeting ASTM-D 6124-06 standard.

Non-Powder Donning Agents/Processes:

- Chlorinated Gloves - processed with the use of chlorine to smooth the surface of gloves to provide non sticky, easy and improved donning.
- Non Chlorinated Gloves - polymer coated for smoother donning.

Ambidextrous (Exam) Gloves:

- Provide economy and efficiency.
- Can stretch to fit hand contours (Crosstex gloves have low modulus - low resistance to stretching).



FITTED LATEX EXAM GLOVES–NON-STERILE

Description	Quantity	Ref. #
6.5	50 Pair/Box	19031
7	50 Pair/Box	19032
7.5	50 Pair/Box	19033
8	50 Pair/Box	19034
8.5	50 Pair/Box	19035
9	50 Pair/Box	19036



Left/Right Fitted Lightly Powdered

- Textured, no color pigment.
- Powder: USP modified corn starch.



Description	Quantity	Ref. #
6.5	50 Pair/Box	19071
7	50 Pair/Box	19072
7.5	50 Pair/Box	19073
8	50 Pair/Box	19074
8.5	50 Pair/Box	19075
9	50 Pair/Box	19076



Ultimate Left/Right Powder-Free

- Textured, natural/white.
- Double polymer coated for donning.



NON-LATEX GLOVES

Description	Quantity	Ref. #
6.5	50 Pair/Box	19093
7	50 Pair/Box	19094
7.5	50 Pair/Box	19095
8	50 Pair/Box	19096
8.5	50 Pair/Box	19097



Nitrile Left/Right Fitted Powder-Free

- Textured, blue 100% synthetic nitrile.
- Double polymer coated for donning.



Description	Quantity	Ref. #
Extra Small	200/Box	19010
Small	200/Box	19011
Medium	200/Box	19012
Large	200/Box	19013
Extra Large	200/Box	19014



NEW! *Ambi-ance*®

Ambidextrous Nitrile Glove Powder-Free

- Soft Stretch Nitrile.
- Puncture and tear resistant barrier.
- Textured fingertips.



Description	Quantity	Ref. #
Small	100/Box	MSCC
Medium	100/Box	MMCC
Large	100/Box	MLCC



Copolymer Plastic Powder-Free

- Smooth, 100% clear synthetic polymer.
- Overglove, not for patient treatment.
- Dispensed in individual sheets.





BARRIER FILM

Barrier Film with Finger-Lift Edge



- Unique 3/16" non-adhesive **Finger Lift Edge** allows quick, easy placement/removal.
- Disposable film for areas/surfaces that are hard to reach, clean or disinfect.
- Light tack adhesive easily attaches to surfaces without leaving residue.



SLEEVES (See Digital Sensor Sleeves on Page 36)



Chair Sleeves

- Universal clear plastic sleeve for patient chairs.
- Covers chair back and chair controls.
- Reduces risk of cross-contamination from airborne aerosols.
- Easily slides on and off.



T-Handle Light Sleeves

- Patented clear plastic sleeve for T-Style handles.
- White indicator arrow for quick location of die-cut.
- Will not fall off during use.



Syringe Sleeves

Syringe Sleeve

- Clear plastic sleeve easily fits over three-way syringe, high or low speed handpieces, and/or air abrasion handpieces.
- Reduces risk of cross-contamination from airborne aerosols.
- **Without opening:** simply place over syringe/handpiece and push through.
- **With opening:** convenient pre-cut opening for easy insertion.

Syringe Sleeve w/Adhesive

- Convenient, pre-cut opening for easy insertion with **clear adhesive strip** that attaches to tubing to prevent sliding of sleeve, creating a closed barrier.



Description	Quantity	Ref. #
4" x 6" / 10 x 15 cm	1200 Sheets/Roll	BFBL (Blue) BFCL (Clear) BFPP (Pink w/ a Purpose)

Description	Quantity	Ref. #
27.5" x 24" / 70 x 61 cm	225 Sleeves/Roll	BCCS (Clear)

Description	Quantity	Ref. #
4" x 5.75" / 10 x 14.5 cm	500/Box	BCLST (Clear with white arrow)

Description	Quantity	Ref. #
Without Opening		
2.5" x 10" / 6 x 25 cm	500/Box	BCSS
With Opening		
2.5" x 10" / 6 x 25 cm	500/Box	BCSSO (Clear)
2.5" x 10" / 6 x 25 cm	500/Box	BCSSBO (Blue)
With Opening and Clear Adhesive Strip		
2.5" x 11.5" / 6 x 29 cm	500/Box	BCSSA (Clear)

SLEEVES

Description	Quantity	Ref. #
ADEC/Size B-Ritter		
10.5" x 14" / 27 x 36 cm	500/Box	BCTS1014 (Clear) 
Mini Trays		
7.5" x 10.5" / 19 x 27 cm	500/Box	BCTS710 (Clear)
Chayes/ Size B Ritter		
11.625" x 14.5" / 30 x 37 cm	500/Box	BCTS1114 (Clear)
Midwest/Weber		
11.5" x 16" / 29 x 41 cm	500/Box	BCTS1116 (Clear)



Tray Sleeves

- Clear plastic sleeves with end fold for secure fit.
- Reduces risk of cross-contamination from airborne aerosols.
- Simply remove after each patient and discard.



Description	Quantity	Ref. #
15" x 26" / 38 x 66 cm	250/Box	BCXR (Clear)



X-Ray Sleeve

- Universal clear plastic sleeve fits most x-ray machines.
- Reduces risk of cross-contamination from airborne aerosols.
- Will not adversely affect x-ray performance.



Surface barriers reduce the risk of cross-contamination caused by airborne aerosols, splash and spatter.

TOUCH SURFACES

Usually contaminated by staff during a procedure.

EXAMPLES

Air/water syringe / light handles / headrests / unit controls / chair switches / x-ray exposure button / handpiece holders / evacuator tip holders / saliva ejector tip holders / handpieces - high /low speed, ultrasonic scaler and air abrasion.

TRANSFER SURFACES

Not touched directly by the dental staff, but usually are contacted by contaminated instruments.

EXAMPLES

Bracket trays / instrument trays.

SPLASH, SPATTER & AEROSOL SURFACES

Comprise all surfaces in the operatory not classified as touch or transfer surfaces.

EXAMPLES

Dental chairs / x-ray heads / view boxes / unused countertop areas.

Barriers reduce operatory "down time," are easy to apply and remove, and are visible to patients. Barriers communicate "clean and safe."



TRAY COVERS




Bracket Tray Covers

- Manufactured from a unique, heavy weight paper stock.
- Impedes flow of moisture.
- Minimizes seep-through.

Liquid-Proof Bracket Tray Covers

- Polyethylene-backed paper prevents seep-through.
- *Dual purpose:*
Poly-side down for moisture barrier.
Poly-side up for quick mixing.

BRACKET TRAY COVERS		SIZE	REF. # / COLOR				MADE IN USA	LATEX
SIZE A	WEBER/CHAYES 1,000/Carton	9.5" x 12.25" 24 x 31 cm	FABL FAWH	BLUE WHITE	FADR DUSTY ROSE	FAGR GREEN		
SIZE B	RITTER 1,000/Carton	8.5" x 12.25" 22 x 31 cm	FBAQ FBDR FBPE FBYE	AQUA DUSTY ROSE PEACH YELLOW	FBBG FBGR FBSI	FBBL FBLV FBWH	BLUE LAVENDER WHITE	
SIZE C	WEBER HI-LO 1,000/Carton	11" x 17.5" 28 x 44 cm	FCBL	BLUE	FCGR GREEN	FCWH WHITE		
SIZE D	SS WHITE #20 1,000/Carton	10.25" x 15.75" 26 x 40 cm	FDBL	BLUE	FDGR GREEN	FDWH WHITE		
SIZE E	MIDWEST 1,000/Carton	9" x 13.5" 26 x 34 cm	FEBL FEWH	BLUE WHITE				
SIZE M	MINI 1,000/Carton	5" x 8" 13 x 20 cm	FMINI	WHITE				
SIZE O	STANDARD ROUND 1,000/Carton	13.5" Diameter 34 cm Diameter	FOBL	BLUE	FOWH WHITE			
SIZE V	VIKING 1,000/Carton	8.5" x 11" 22 x 28 cm	FVWH	WHITE				
SIZE X	COX 1,000/Carton	11" x 11" 28 x 28 cm	FCOX	WHITE				
								
				<div><div>AQUA</div><div>BEIGE</div><div>BLUE</div><div>DUSTY ROSE</div><div>GREEN</div><div>LAVENDER</div><div>PEACH</div><div>SILVER GREY</div><div>WHITE</div><div>YELLOW</div></div>				
LIQUID-PROOF BRACKET TRAY COVERS								
SIZE B	RITTER 500/Carton	8.5" x 12.25" 22 x 31 cm	FBC	WHITE				
SIZE F	ADEC 500/Carton	8.5" x 11" 22 x 28 cm	FF	WHITE				

HEADREST COVERS



Polycoated Headrest Covers

- Soft facial tissue combined with poly.
- Fluid resistant barrier.
- Strong and absorbent.



Plastic Headrest Covers

- Clear, soft 1 mil thick polyethylene.
- 3/8"/0.95cm gusset – lays flat on headrest.
- Impervious barrier between patient and headrest.



Non-Woven Headrest Covers

- Soft, breathable non-woven fabric.
- Quiet – does not rustle.
- Provides maximum patient comfort.



HEADREST COVERS POLYCOATED	SIZE	REF. # / COLOR					
STANDARD 500/Case	10" x 10" 25 x 25 cm	L0BL L0DR	BLUE DUSTY ROSE	L0GR L0LV	GREEN LAVENDER	L0WH	WHITE
EXTRA LARGE 500/Case	10" x 13" 25 x 33 cm	L3BL L3DR	BLUE DUSTY ROSE	L3GR L3LV	GREEN LAVENDER	L3WH	WHITE
JUMBO 500/Case	13" x 13" 33 x 33 cm	L13BL L13WH	BLUE WHITE				
EXTRA JUMBO 500/Case	11.5" x 15" 29 x 38 cm	L15WH	WHITE				
PLASTIC							
REGULAR 250/Dispenser Box	9.5" x 11" 24 x 28 cm	L0CP L0CPW	CLEAR WHITE				
JUMBO 250/Dispenser Box	9.5" x 14" 24 x 36 cm	L3CP L3CPW	CLEAR WHITE				
NON-WOVEN							
STANDARD 500/Case	10" x 10" 25 x 25 cm	L0CB L0CL	BLUE LAVENDER	L0C	WHITE		
EXTRA LARGE 500/Case	10" x 13" 25 x 33 cm	L3CB L3CL	BLUE LAVENDER	L3C	WHITE		

BLUE
DUSTY ROSE
GREEN
LAVENDER
WHITE



EQUIPMENT | TRANSPORTATION PROTECTION



iBarrier®

Protect your valuable equipment from cross-contamination!

New! iBarrier® Touch Sensitive Sleeves for Tablets are universal and compatible with most tablets.

Reduce the risk of cross-contamination with iBarrier, the only full coverage, disposable sleeve that is both easy and inexpensive enough for single use to help ensure proper infection prevention for tablet computers.



Description	Quantity	Ref. #
iBarrier® for Tablets 8.25" x 10" + 1.5" Lip/ 21 x 25 + 4 cm Lip	100 Sleeves/Box	BCIB



Lab Transport Bags

- Meets OSHA requirements for transport of medical specimens/dental lab cases.
- Convenient 2 pouch system keeps Rx/lab slip separate from specimens/lab work as required by OSHA.
- Easy to understand instructions.
- Biohazard symbol for easy identification and OSHA compliance.
- Permanent adhesive closure does not contact gloved hand – tamper evident.
- White front allows easy recording of information.



Lab Transport Bags provide added protection for dental and medical laboratory workers as required by OSHA and recommended by the NCCLS. (National Committee for Clinical Laboratory Standards.)



Description	Quantity	Ref. #
6" x 10" / 15 x 25 cm	250/Box	BAGS



Exam Table Paper

Smooth

- Lightweight, treated paper resists moisture penetration.
- Smooth, soft finish for patient comfort.
- Rolls are individually wrapped.

Crepe

- Lightweight, specially treated paper absorbs moisture.
- Soft, supple linen-like texture for added patient comfort.
- Rolls are individually wrapped.



INDIVIDUALLY WRAPPED ROLLS

Description	Size	Length	Ref. #
White-Smooth	18" / 46 cm	225 ft / 68.58 m	ETPS18
White-Smooth	21" / 53 cm	225 ft / 68.58 m	ETPS21
White-Crepe	18" / 46 cm	125 ft / 38.1 m	ETPC18
White-Crepe	21" / 53 cm	125 ft / 38.1 m	ETPC21



Pillow Cases

- Facial tissue combined with poly for patient comfort and protection.
- Fluid resistant barrier between patient and pillow.
- Soft, strong and absorbent.



Description	Size	Ref. #
White 21" x 30" / 53 x 67 cm	25/Bag	PC2130

STERILIZATION POUCHES

AVAILABLE IN 14 SIZES!

Description	Quantity	Ref. #
2.25" x 4" / 6 x 10 cm	200/Box	SCXX2
2.75" x 9" / 7 x 23 cm	200/Box	SCX2
3.5" x 5.25" / 9 x 13 cm	200/Box	SCXS2
3.5" x 9" / 9 x 23 cm	200/Box	SCS2
3.5" x 9" / 9 x 23 cm	500/Box	SCS2S
3.5" x 22" / 9 x 56 cm	100/Box	SCL3222 NEW!
5.25" x 6.5" / 13 x 17 cm	200/Box	SCW2
5.25" x 10" / 13 x 25 cm	200/Box	SCM2
5.25" x 15" / 13 x 38 cm	200/Box	SCL5152
7.5" x 13" / 19 x 33 cm	200/Box	SCL2
8" x 16" / 20 x 41 cm	200/Box	SCL8162
10" x 15" / 25 x 38 cm	100/Box	SCL10152
10.5" x 17" / 27 x 43 cm	100/Box	SCL10172 NEW!
12" x 15" / 30 x 38 cm	100/Box	SCL12152
12" x 18" / 30 x 46 cm	100/Box	SCL12182

* Steam chemical indicator classes defined:

Single process indicators are intended for use with individual units to indicate that the unit has been directly exposed to the sterilization process and unprocessed units. Designed to react to one or more of the critical process variables.

Multi-Variable Indicators: a multi-variable indicator is designed to react to two or more of the critical variables and is intended to indicate exposure to a sterilization cycle at a stated value of the chosen variable.

Description	Quantity	Ref. #
2" x 8" / 5 x 20 cm	200/Box	SCZ
2.25" x 4" / 6 x 10 cm	200/Box	SCXX
2.75" x 9" / 7 x 23 cm	200/Box	SCX
3.25" x 9" / 8 x 23 cm	200/Box	SCQ
3.5" x 5.25" / 9 x 13 cm	200/Box	SCXS
3.5" x 9" / 9 x 23 cm	200/Box	SCS
3.5" x 9" / 9 x 23 cm	500/Box	SCS5
3.5" x 22" / 9 x 56 cm	100/Box	SCL322 NEW!
4.25" x 11" / 11 x 28 cm	200/Box	SCB1X
5.25" x 10" / 13 x 25 cm	200/Box	SCM
5.25" x 15" / 13 x 38 cm	200/Box	SCL515 NEW!
7.5" x 13" / 19 x 33 cm	200/Box	SCL
8" x 16" / 20 x 41 cm	200/Box	SCL816 NEW!
10" x 15" / 25 x 38 cm	100/Box	SCL1015
12" x 15" / 30 x 38 cm	100/Box	SCL1215
12" x 18" / 30 x 46 cm	100/Box	SCL1218

Pink with a Purpose® External Process Indicator

Description	Quantity	Ref. #
3.5" x 9" / 9 x 23 cm	200/Box	SCSPP
5.25" x 10" / 13 x 25 cm	200/Box	SCMPP

Sure-Check and Duo-Check Sterilization Pouches are not considered hazardous waste and can be disposed of in regular trash (40 CFR 261.24 tested using the TCLP, EPA Publication SW-846).

Sure-Check® Sterilization Pouches

Triple your confidence with the world's most advanced sterilization pouches with multi-parameter internal/external indicators.

CDC infection control guidelines make it clear: to ensure effective sterilization, items must be exposed to **all 3** sterilization criteria – **time, temperature and pressure (steam)**. Advanced patent pending Sure-Check® Sterilization Pouches feature:

- **Highest sterility assurance** – Built-in internal/external multi-parameter indicators only change color when all 3 criteria for sterilization have been met – unlike traditional sterilization pouches with a single process indicator that measures only temperature.
- **Simplified process** – Eliminates the extra step of adding separate multi-parameter indicators to each pouch.
- **Meets CDC Guidelines** for Disinfection and Sterilization in Healthcare Facilities, 2008.
- **Meets ANSI/AAMI ST79** Comprehensive Guide to Steam Sterilization and Assurance in Health Care Facilities.
- **Tested and certified** by an independent laboratory.
- **For steam and EtO only.**

**Time
Temperature
Steam**



Sure-Check pouches, when combined with daily or weekly testing using ConForm biological monitoring systems, provide the highest level of sterility assurance available.



Duo-Check® Sterilization Pouches



- Secure Seal**
Duo-Check® offers extra wide chevron protection against instrument breaching, while Sure-Check offers advanced triple seal protection.
- Medical Grade Paper**
Wet strength treated, virgin medical grade paper that meets or exceeds medical pouch standards.
- Blue Tinted Transparent Film**
Facilitates both easy instrument viewing and identification of punctures or tears in film.
- Wide Self-Seal Adhesive Strip**
Easily and securely seals pouch, providing reliable seal until pouch is opened.
- Perforated Fold**
Facilitates easy, accurate folding of adhesive strip to form airtight seal until pouch is opened.



- Complete Paper-Film Separation**
Consistently clean separation (peeling) of paper and film is due to high grade materials and adhesives.
- Thumb Notch**
Assures easy opening of pouch.
- Six Corner Tack Seals**
Prevents corners from curling after sterilization. Tacks resist collection of dust and contaminants near seal.
- Process Indicators**
Internal and external indicators change color with sterilization heat, are easily visible, and are compatible with steam (indicators turn light brown) and EtO (indicators turn dark brown) sterilization agents.

For steam and EtO only.



BIOLOGICAL MONITORING

NEW!

ConFirm® *The leader in biological monitoring.*



ConFirm® 10 In-Office Biological Monitoring System

Biological monitoring with final readout in just 10 hours!



Now you can meet biological monitoring guidelines for steam sterilizers in **less than half the time**... with ConFirm® 10, the first in-office system that delivers results in **just 10 hours**.

- **Easy to use** – Simply process indicator in a normal steam sterilization cycle followed by a 10 hour incubation period for both the test and control vials.
- **Easy to read results** – Following incubation, if the media turns yellow this indicates a failed test.
- **Fast** – Final readout in just 10 hours.
- **A 24-hour vial CANNOT** be processed in a ConFirm 10 incubator.
- **Built-in Ampoule Crusher.**



BECAUSE YOU CAN'T SEE STERILE.®

Follow CDC Compliance Guidelines* for Optimal Protection:

*Visit CDC.gov for additional information

www.CrosstexBMS.com

Description	Quantity	Ref. #
Steam Biological Indicators	25/Box	C10BI25
ConFirm® 10 Digital Incubator (Record Keeper, Manual)	EACH	C10SK
Specify 110 or 220 voltage		
ConFirm® Record Keeper Booklet	EACH	CRK02
ConFirm® 10 Standard* Incubator 110 VOLT (Standard Incubator Record Keeper, Manual)	EACH	NDB-601
ConFirm® 10 Standard* Incubator 220 VOLT (Standard Incubator Record Keeper, Manual)	EACH	NDB-601E

*Made offshore

ConFirm® 10 digital incubator shown, in addition, a standard (non-LED display) model also available. ConFirm® 10 Incubators are equipped for 110 and 220 volts (specify when ordering).

ConFirm® 24 In-Office Biological Monitoring System

Easy to use and interpret... results in just 24 hours!

Simply process indicator in a normal sterilization cycle. Activate indicator, incubate and record readout. If any spores survive the process, the indicator medium will turn yellow signifying a failed test.

- **Readout** in just 24 hours.
- **Easy to use, easy to read results.**
- **One of the most cost-effective** methods of biological monitoring.
- **Complete Offering** - Everything you need for effective biological monitoring of steam sterilizers, including incubator, indicators, record book and manual.
- **Universal BI** - ConFirm® 24 biological indicators can be processed in most incubators, including the current 3M Attest incubator.
- **A 24-hour vial CANNOT** be processed in a ConFirm 10 incubator.
- **Built-in Ampoule Crusher**



CAUTION: Visual surveillance of sterilizer gauges and chemical monitoring via process indicators and chemical integrators is only effective in identifying gross sterilizer malfunction. Biological monitoring using spore suspensions in strips or vials remains the ultimate criterion for verifying a sterilizer's effectiveness. Weekly testing is recommended by the CDC, ADA and OSAP. Use a biological indicator that is appropriate for the method of sterilization: *Geobacillus stearothermophilus* for steam sterilizers and chemical vapor sterilizers; *Bacillus atrophaeus* for dry heat and ethylene oxide units.

Description	Quantity	Ref. #
Steam Biological Indicators	25/Box	CSBI25
Starter Kit (25 Indicators, Incubator, Record Keeper, Manual)	EACH	CBMS10
ConFirm Record Keeper Booklet	EACH	CRK02
Ampoule Crusher/Activator for In-Office System (Only Required for Standard 1401 Incubator)	EACH	CCRUSH

ConFirm® 24 Incubators are equipped for 110 and 220 volts (specify when ordering).

CHEMICAL MONITORING



STEAMPlus™ Type 5 Integrators



Used with steam sterilization cycles to **provide a definitive pass or fail result** – allowing for the release of all non-implant loads before the results of the spore test are received.

- **Moving-front technology** provides an immediate readout for use in gravity, prevacuum, and flash cycles.
- Enhance sterility assurance by **monitoring sterilizer performance.**



Description	Quantity	Ref. #
STEAMPlus™ Type 5 Integrators 4" x .75" Strip / 10 x 2 cm	100/pack	SSI-100

Does not replace weekly use of a biological indicator because it does not contain spores.

BIOLOGICAL MONITORING

ConForm Premium Test Service

Description	Ref. #
6 Tests (3 Strip Test, 2 Tests & 1 Control Strip)	CST060
12 Tests (3 Strip Test, 2 Tests & 1 Control Strip)	CST120
48 Tests (3 Strip Test, 2 Tests & 1 Control Strip)	CST480

Includes pre-paid postage.

Test strips can not be split between two sterilizers.

ConForm® Value Test Service

Description	Ref. #
12 Tests (2 Strip Test, 1 Test & 1 Control)	CVT120
52 Tests (2 Strip Test, 1 Test & 1 Control)	CVT520

Does not include postage.

ConForm® Mail-in Sterilizer Monitoring Service

The most comprehensive mail-in service in healthcare.

We offer everything you need to verify sterilizer performance including our complimentary reminder service.

- **Compatible** with dry heat, chemical, EtO and steam sterilizers.
- **Test results in as little as 24 hours** for steam, 72 hours for chemical, 7 days for dry heat and EtO upon receipt of spore strips in our laboratory. Your choice of immediate test failure notification by fax or email with a follow-up phone call the next day from one of our clinical consultants. Sterilization results are faxed or emailed quarterly and annually.
- **Check results online 24 hours a day** on our secure, confidential web site.
- **Complimentary reminder service** provides either a fax or email alert notification of a missed test date.
- Third party documentation of test results provides a tangible record when proof of testing must be provided.
- **One of the most cost-effective methods of biological monitoring.**
- Highly resistant dual-species bacterial spores provide the most reliable method of testing for an effective sterilization cycle.



INSTRUMENT PACKAGING & MONITORING

Description	Quantity	Ref. #
12" x 12" / 30 x 30 cm	1,000/Case	SW12
15" x 15" / 38 x 38 cm	500/Case	SW15
18" x 18" / 46 x 46 cm	500/Case	SW18
20" x 20" / 51 x 51 cm	500/Case	SW20
24" x 24" / 61 x 61 cm	500/Case	SW24
30" x 30" / 76 x 76 cm	250/Case	SW30
36" x 36" / 91 x 91 cm	250/Case	SW36
40" x 40" / 102 x 102 cm	250/Case	SW40
45" x 45" / 114 x 114 cm	250/Case	SW45
48" x 48" / 122 x 122 cm	100/Case	SW48
54" x 54" / 137 x 137 cm	100/Case	SW54



PETROLEUM FREE— manufactured from environmentally friendly cellulose material made from wood pulp. Eliminates skin sensitivity issues associated with oily residues inherent in plastic based wrappers.

CSR Sterilization Wrap

- **Highest bacterial filtration** efficiency (BFE) available.*
- **Highly permeable** – no wet packs.
- **Bacterial barrier** to both airborne and waterborne contamination.
- **Outstanding liquid repellency** – water, alcohol, iodine.
- Use with EtO, steam, gamma irradiation or E-beam sterilization technology.
- Even texture to permit immediate visual detection of small pinholes/tears.
- **Exceptional tensile strength** – wet or dry.
- **Meets Flammable Fabrics Act (FFA) requirements.**

*Data on File



Description	Quantity	Ref. #
With Process Indicator		
2" / 5 cm	100ft/30m Roll	SI2
3" / 8 cm	100ft/30m Roll	SI3
4" / 10 cm	100ft/30m Roll	SI4
6" / 15 cm	100ft/30m Roll	SI6
Without Process Indicator		
2" / 5 cm	100ft/30m Roll	SN2
3" / 8 cm	100ft/30m Roll	SN3
4" / 10 cm	100ft/30m Roll	SN4
6" / 15 cm	100ft/30m Roll	SN6



Sani-Tube® Sterilization Tubing (Nylon)

- **Manufactured from clear, puncture-resistant, 1 ½ mil nylon.**
- **Use in steam only.**
- Seal with process indicator tape or heat sealer.



All products on this page are not considered hazardous waste and can be disposed of in regular trash, with the exception of Steam Process Indicator Tape. (40 CFR 261.24 tested using the TCLP, EPA Publication SW-846).



INSTRUMENT PACKAGING & MONITORING



Sani-Roll® Sterilization Tubing (Paper/Plastic)

- **Convenient, economical** way to package instruments/items – use only length and width required.
- **Manufactured from medical grade paper.**
- **Blue tinted transparent film easily identifies compromise** in film due to puncturing or tearing.
- **Complete paper and film separation.**
- **Color process indicators - steam, EtO.**



Description	Length	Ref. #
2" / 5 cm	100 ft/30 m roll	SCT2
3" / 8 cm	100 ft/30 m roll	SCT3
4" / 10 cm	100 ft/30 m roll	SCT4
6" / 15 cm	100 ft/30 m roll	SCT6

Description	Length	Ref. #
2" Roll / 5 cm	200 m	SCT2200
3" Roll / 8 cm	200 m	SCT3200
4" Roll / 10 cm	200 m	SCT4200
6" Roll / 15 cm	200 m	SCT6200
8" Roll / 20 cm	200 m	SCT8200
12" Roll / 30 cm	200 m	SCT12200



Sure-Check® Strips

- **Multi-parameter indicator** for use in pouches, tubing, autoclave bags, CSR wraps.
- **Indicates all conditions for sterilization have been achieved** – time, temperature, presence of steam.
- **Meets OSHA & CDC guidelines** for sterilization monitoring.
- **Simple, convenient to use.**
- **Works in both steam and chemiclave.**



Description	Quantity	Ref. #
White 3.75" x .625" / 10 x 2 cm	100/Bag	SCK



Non-Latex, Lead-Free Steam Indicator Tape

- **Non-Latex** – safer for you, your staff and patients.
- **Lead-free** – safer for the environment.
- **Effectively seals tubing, autoclave bags, CSR wraps.**
- **Indicates specific temperature has been achieved during sterilization cycle.**



Description	Ref. #
.75" x 60 yards / 1.9 cm x 55 m	STM



Steam Process Indicator Tape

- **Effectively seals tubing, autoclave bags, CSR wraps.**
- **Indicates specific temperature has been achieved during sterilization cycle.**



Description	Ref. #
Small (.5" x 60 Yards / 1.3 cm x 55 m)	STS
Medium (.75" x 60 Yards / 1.9 cm x 55 m)	STM
Large (1" x 60 Yards / 2.5 cm x 55 m)	STL



Self-Sealing Autoclave Bags (Paper)

- **Unique hospital double-fold seal with pull-off adhesive tab.**
- **Gusset feature provides 50% more inner space.**
- **Color process indicators – steam.**



Description	Quantity	Ref. #
2.5" x 10.5" + 1.5" Gusset / 6 x 27 + 4 cm Gusset	1,000/Carton	OC
2.5" x 8.75" + 1.5" Gusset / 6 x 22 + 4 cm Gusset	1,000/Carton	OCS



AirView™ II Bowie-Dick Test Pack

- Specifically designed to easily detect the presence of air in a pre-vacuum steam sterilizer.
- Smaller pack size reduces the amount of waste generated on a daily basis.
- Easy-to-read test sheet clearly identifies failures when used as instructed and can be filed for record keeping.
- Lead free ink chemistry reduces the amount of potentially hazardous waste entering the waste stream.
- Packaged in a case quantity of 30 test packs, perfect for your monthly use.

Description	Quantity	Ref. #
5" x 3.75" x 0.625" / 13 x 10 x 2 cm	30 Test Packs Per Case	MBD030

SALIVA EJECTORS | TONGUE DEPRESSORS

Description	Quantity	Ref. #
White/White	100/Bag	ZWWCP
Clear/White	100/Bag	ZCWCP
White/Pink Bubblegum Scented	100/Bag	ZWPCPBG
White/Green Mint Scented	100/Bag	ZWGCPM

Comfort Plus® Premium Saliva Ejectors

The kinder, gentler saliva ejector...
takes in fluid, not tissue!

- Patented tip **reduces patient discomfort.**
- **Optimizes fluid removal.**
- **Minimizes trauma** to mucosal tissue.
- **Facilitates uninterrupted fluid and light debris removal.**
- **Prevents tissue from blocking suction.**

Now available
in Mint & Bubblegum scented!



Description	Quantity	Ref. #
White	100/Bag	ZFF
Adapters	10/Bag	ZFFAD

Hygoformic Saliva Ejectors

- **Combination saliva ejector/tongue retractor.**
- **Unique coil design** is adjustable for maximum patient comfort.
- **Smooth edges eliminate irritation** - ideal for patients with sensitive tissue.
- **Biodegradable.**



Description	Quantity	Ref. #
Premium Plus		
Clear/Blue	100/Bag	ZCB
Clear/White	100/Bag	ZCW
White/White	100/Bag	ZWW
Pink/Pink	100/Bag	ZPP
Multi Color	100/Bag	ZCMC
Advantage		
Blue/Blue	100/Bag	ZBBI
Clear/Blue	100/Bag	ZCBI
Clear/Clear	100/Bag	ZCCI
White/White	100/Bag	ZWWI

Premium Plus® & Advantage Saliva Ejectors

- Wire-reinforced tube forms and **maintains any desired shape.**
- **Provides optimal suction** without aspirating tissue.
- **Soft, non-removable tip** - bonded to the tube for maximum patient safety.




EVACUATOR TIPS | ARTICULATING PAPER

Description	Quantity	Ref. #
Blue-Small Tip .0625" / .16 cm	25/Bag	ZSABL
White-Standard Tip .125" / .32 cm	25/Bag	ZSAWH
Green-Large Tip .25" / .64 cm	25/Bag	ZSAGR

Surgical Aspirator Tips


- **Made of durable, high quality polyethylene with a 30° angle.**
- Tips are designed to **fit all high-volume evacuation systems.**
- **May be used for cheek retraction during procedure** - will not bend.

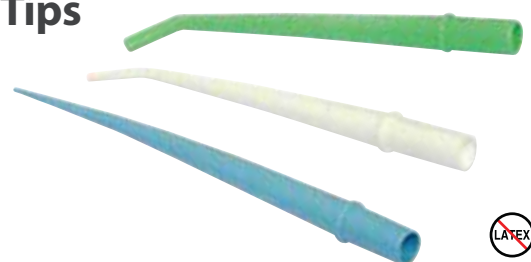

Description	Quantity	Ref. #
Combo Tip®, White	100/Bag	ZET
Non-Vented, White	100/Bag	ZED1
Slotted, White	50/Bag	ZES50

HVE Tips

- **Two tips in one** - vented end on one side and nonvented end on the other.
- **Unique "S" tip design with NO sharp edges** for maximum patient comfort.

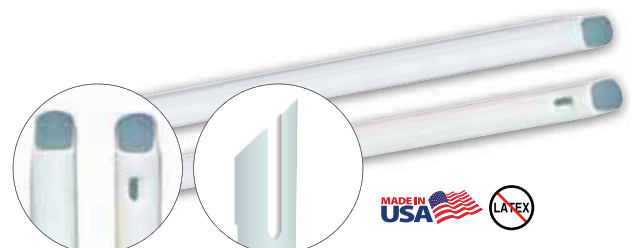

Combo Tip®

Slotted





EVACUATOR TIPS | ARTICULATING PAPER



MaxVac Plus® Combo Tip®

- Identical to the Combo Tip in every way but color.
- Available in 5 colors - blue, green, pink, purple, yellow.



Description	Quantity	Ref. #
Blue	100/Bag	ZETMVB
Green	100/Bag	ZETMVGR
Pink	100/Bag	ZETMVPK
Purple	100/Bag	ZETMVLV
Yellow	100/Bag	ZETMVYE
Assorted (2 bags per color per case)	Case10/Bags	ZETMVP

Sparkle® Air Water Syringe Tips Disposable

NEW!

- **No adapter required.** Locking groove ensures tip is securely installed.
- **Quad Core® Air Channel** ensures air/water separation.
- Beveled edge provides **increased patient comfort.**
- Rounded non-scratch end **prevents O-ring damage.**
- Single-use **eliminates cleaning and potential cross-contamination risks.**
- **Translucent** colors allow for visualization of water flow/blockage.





Description	Quantity	Ref. #
Clear	250/Bag	BCSAWSCL
White	250/Bag	BCSAWSWH
Assorted	250/Bag	BCSAWS
Assorted	1500/Bag	BCSAWSB
Assorted Includes: Blue, Purple, Yellow, Amber		

Articulating and Occlusion Testing Materials

- The combined use of articulating paper, film, and foil to mark high and low occlusion points provides the most accurate way to mark every point of articulation.
- Superior manufacturing process combined with high quality control guarantees clear, positive marking without smudging on wet or dry fields.
- Color coded packaging and product descriptions make product identification easy.
- Articulating paper available in straight, curved, curved with tab and horseshoe shaped.
- Articulating film and foil available in straight.

NEW!

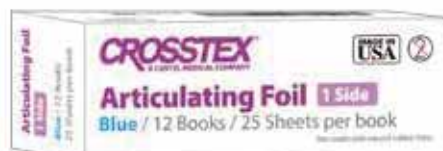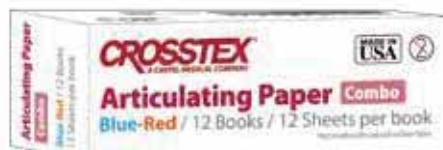

ARTICULATING PAPER

12 sheets per book/12 books per box

Description	Ref. #
Extra Thin, Blue 0.0015"/38 µm	TPXT
XX Thin, Blue 0.00125"/32 µm	TPXXT
Thick Blue 0.005"/127 µm	TPTH
Thin Blue 0.0028"/71 µm	TPT
Blue Red Combo, 0.0028"/71 µm	TPBR
Blue Red Combo, Thin, Curved 0.003"/71 µm	TPCBRT
Blue Red Combo, with tab, Curved 0.004"/89 µm	TPCBR
Horseshoe Articulating Paper, 12 sheets per book/6 books per box Blue Red Combo 0.0035"/89 µm	TPH

ARTICULATING FOIL

25 sheets per book/12 books per box

Description	Ref. #
Blue Red Combo, 0.0005"/12 µm	TFOBR
Red, 2 side 0.0005"/12 µm	TFOR2
Blue, 2 side 0.0005"/12 µm	TFOB2
Blue, 1 side 0.0005"/12 µm	TFOB

ARTICULATING FILM

25 sheets per book/12 books per box

Description	Ref. #
Black Red Combo, Precisely True 0.0005"/13µm	TFMBKR
Blue Thin Precisely True, 0.0005"/13µm	TPP
Red Precisely True, 0.0005"/13µm	TFMR
Blue Red Combo, Precisely True 0.0005"/13µm	TFMBR

CUPS | CUP DISPENSER | MATRIX STRIPS | MIXING PADS

Description Quantity Ref.

5 oz /148 ml		
Beige	1,000/Case	CXBG
Blue	1,000/Case	CXBL
Dusty Rose	1,000/Case	CXDR
Green	1,000/Case	CXGR
Lavender	1,000/Case	CXLV
Silver Grey	1,000/Case	CXSI
White	1,000/Case	CXWH
Yellow	1,000/Case	CXYE
Clear/Translucent	2,500/Case	CXCL



Plastic Cups

- Double rolled lip for easy separation.
- Unique embossing for greater strength and easy dispensing.
- Double coated for lustrous, high gloss finish.



Description Quantity Ref.

1oz / 3 cc	100/Sleeve	CX1
------------	------------	-----

Medicine/
Mixing Cups

- Convenient for dispensing or mixing.
- Features ounce, cc, ml, dram & teaspoon calibrations.
- Unbreakable plastic; can be squeezed to form pour spout.



Description Quantity Ref.

White base with clear top	1	PCC
------------------------------	---	-----

Cup Dispenser

- Contemporary design for every office decor.
- Includes self-adhesive strips for easy mounting.
- Dispenses 5 oz/148 ml plastic and wax coated paper cups.
- 50 cups fit into dispenser.



Description Quantity Ref.

6" / 15 cm	500 Blades/Box	IC
------------	----------------	----



Tongue Depressors

- Standard 6"/15 cm.
- Smooth and splinter-free.
- Box is poly wrapped and self-dispensing.
- Medical grade.



Description Quantity Ref.

4" / 10 cm long	500 Strips/Tube	KSTRIP500
.375" / .95 cm wide		
.002" / 60 μ thick	1,000 Strips/Tube	KSTRIP1000

Matrix Strips

- Strong and economical.
- Made of mylar polyester film.
- Resists solvents and will not stretch, curl or break.



Description Quantity Ref.

3" x 3" / 8 x 8 cm	100 Sheets/Pad	KPAD33
3" x 6" / 8 x 15 cm	100 Sheets/Pad	KPAD36
6" x 6" / 15 x 15 cm	100 Sheets/Pad	KPAD66



Poly Coated Mixing Pads

- Extra heavy with a high poly finish.
- Use for mixing cements, composites, rubber base material and silicates.



Non-skid base.



DENTAL DAMS

Creates uncontaminated working fields and eliminates visual obstructions.

Dental Dam – Latex, Powder Free / Low Protein

- Manufactured from medium modulus natural latex.
- Powder-free – less than 5mg/sheet (ASTM D6124:2001).
- Low protein – extractable protein content is below FDA allowable claim limit of 50µg/g per sheet (ASTM D5712:1 995).
- 100% non-chlorinated.

Significantly reduces the risk of latex allergy reactions.

Most latex allergies are caused by plant protein in natural rubber latex. Research conducted by the Royal College of Physicians (UK) has found that, by removing the protein and powder in latex products, incidence of latex allergies is significantly reduced.



Mint



Unflavored



Peppermint

Dental Dam Non-Latex



- Non-Latex product drastically reduces the risk of allergic reactions for both patients and staff.
- Peppermint Flavored



SIZE	QUANTITY	GAUGE	COLOR	FLAVOR	REF. #	
DENTAL DAM Latex– Powder Free/Low Protein						
5″x 5″/13 x 13 cm 5″x 5″/13 x 13 cm 5″x 5″/13 x 13 cm	52 Sheets/Box	Thin Medium Heavy	Green Green Green	Mint Mint Mint	19202 19200 19201	
6″x 6″/15 x 15 cm 6″x 6″/15 x 15 cm 6″x 6″/15 x 15 cm		Thin Medium Heavy	Green Green Green	Mint Mint Mint	19402 19400 19401	
5″x 5″/13 x 13 cm 5″x 5″/13 x 13 cm 5″x 5″/13 x 13 cm		Thin Medium Heavy	Blue Blue Blue	Unflavored Unflavored Unflavored	19102 19100 19101	
6″x 6″/15 x 15 cm 6″x 6″/15 x 15 cm 6″x 6″/15 x 15 cm	36 Sheets/Box	Thin Medium Heavy	Blue Blue Blue	Unflavored Unflavored Unflavored	19302 19300 19301	
DENTAL DAM Non-Latex						
6″x 6″/15 x 15 cm		15 Sheets/Box	Medium	Purple	Peppermint	19500

GAUGE MEASUREMENTS: THIN 0.125-0.160 / Avg 0.14mm | MEDIUM 0.165-0.200 / Avg 0.18mm | HEAVY 0.205-0.240 / Avg 0.22mm

WIPES | SPRAYS

Description	Quantity	Ref. #
Large 6" x 10.5" / 15.2 x 26.6 cm	75/Canister	SANLB2
Extra Large 7.5" x 15" / 19 x 38 cm	65/Canister	SANXLB2
Large 5" x 7" / 12.7 x 17.78 cm	40/Box	SANLB
Extra Large Individual 11.5" x 11.75" / 29.2 x 29.85 cm	40/Box	SANXLB2I

Sani-Cloth® Bleach Germicidal Disposable Wipe

Bactericidal, fungicidal, tuberculocidal, virucidal

Sani-Cloth® Bleach is ideal for disinfecting high-risk areas endemic with *Clostridium difficile* spores and Norovirus. The stabilized dilution of sodium hypochlorite is tested effective against 50 microorganisms in 4 minutes (including 14 Multi-Drug Resistant Organisms (MDROS)). Sani-Cloth Bleach is now available in an Extra Large (XL) Canister which features an unfolded, ready-to-use 7.5 x 15-inch wipe.



Description	Quantity	Ref. #
Large 6" x 6.75" / 15 x 17 cm	160/Canister	SANLC
Extra Large 7.5" x 15" / 19 x 38 cm	65/Canister	SANXLC2

Sanitex® Plus Germicidal Disposable Wipe

EPA reg. intermediate level disinfectant: bactericidal, virucidal

- Effective against MRSA in 3 minutes, HBV and HVC in 2 minutes, and HIV-1 in 1 minute.
- Use for disinfecting on hard, non-porous surfaces and equipment.



Description	Quantity	Ref. #
Large 6" x 6.75" / 15 x 17 cm	160/Canister	SANL
Extra Large 7.5" x 15" / 19 x 38 cm	65/Canister	SANXL2

Sani-Cloth® Plus Germicidal Disposable Cloth

EPA reg. low level disinfectant: bactericidal, virucidal

Sani-Cloth® Plus Germicidal Disposable Cloth is ideal for use in alternate care settings including dental offices. Sani-Cloth® Plus is compatible with a broad range of hard, non-porous surfaces and equipment commonly found in healthcare environments. It meets CDC, OSHA and CMS Tag F441 guidelines and is bactericidal, virucidal. Effective against 16 microorganisms in 5 minutes – including bloodborne pathogens, viruses and bacteria.



Description	Ref. #
Quart / .946 Liter	JSSDP
Gallon / 3.785 Liter	JSSGP



Sanitex Plus® Spray Ready-to-Use Disinfectant/Cleaner

EPA reg. intermediate level disinfectant: tuberculocidal, bactericidal, virucidal

- Kills TB in 6 min., HBV in 3 min., HIV in 30 sec.
- Active ingredients: Quaternary Ammonium and alcohol.
- Low alcohol content (21%) provides excellent cleaning with rapid TB kill.
- Use for both cleaning and disinfecting steps on hard, non-porous surfaces and patient care equipment.





WIPES

Super Sani-Cloth® Germicidal Disposable Wipe



**EPA reg. intermediate level disinfectant:
tuberculocidal, bactericidal, virucidal**

Super Sani-Cloth® Germicidal Disposable Wipes are ideal for daily use in fast-paced environments that require short contact times and broad coverage of microorganisms. Its fast contact time allows for a quick room turnover, and it is compatible with a broad range of surfaces and equipment in healthcare. Super Sani-Cloth® meets CDC, OSHA and CMS Tag F441 guidelines and is bactericidal, tuberculocidal and virucidal. Effective against 27 microorganisms in 2 minutes, including: Multi-Drug Resistant Organisms (MDROs), bloodborne pathogens, TB, and viruses.



Description	Quantity	Ref. #
Large 6" x 6.75" / 15 x 17 cm	160/Canister	SANLS
Extra Large 7.5" x 15" / 19 x 38 cm	65/Canister	SANXLS2
Large Individual 5" x 8" / 12.7 x 20 cm	50/Box	SANLSI
Extra Large Individual 11.5" x 11.75" / 29.2 x 29.8 cm	50/Box	SANXLSI

Sani-Cloth® AF3 Germicidal Disposable Wipe



**EPA reg. intermediate level disinfectant:
tuberculocidal, bactericidal, virucidal**

Alcohol and fragrance free

Sani-Cloth® AF3 Germicidal Disposable Wipes are designed to make surface disinfection more efficient for staff and pleasant for patients. Sani-Cloth AF3 is both alcohol and fragrance free. The alcohol free formula makes it ideal for use on sensitive dental equipment and fragrance free makes it ideal for use around patients and staff, even those with respiratory sensitivities. Sani-Cloth AF3 is tested effective against 44 clinically relevant microorganisms in 3 minutes, including: Multi-Drug Resistant Organisms (MDROs), bloodborne pathogens, TB and viruses.



Description	Quantity	Ref. #
Large 6" x 6.75" / 15 x 17 cm	160/Canister	SANAF3
Extra Large 7.5" x 15" / 19 x 38 cm	65/Canister	SANAF3L23
Large Individual 5" x 8" / 12.7 x 20.3 cm	50/Box	SANAF3I
Extra Large Individual 11.5" x 11.75" / 29.2 x 29.8 cm	50/box	SANAF3L2I3

Sani-Cloth® HB Germicidal Disposable Wipe

**EPA reg. hospital disinfectant:
bactericidal, virucidal, fungicidal**

Alcohol free/quaternary formula is ideal for disinfecting sensitive hard, non-porous surfaces and equipment. Bactericidal, fungicidal and virucidal. Effective against 100+ organisms in 10 minutes, including: HBV, influenza A/Hong Kong (H3N2), Acinetobacter baumannii, Aspergillus niger and Trichophyton mentagrophytes.



Description	Quantity	Ref. #
Large 6" x 6.75" / 15 x 17 cm	160/Canister	SANHBL
Extra Large 7.5" x 15" / 19.1 x 38.1 cm	65/Canister	SANHBXL2

Please refer to the chart on Page 30 for microorganism kill time.

WARNING: The products on pages 28, 29 and 30 do not eliminate the risk of contracting any disease or infection. Follow all instructions and warnings on the use of these products. Must be used in accordance with applicable federal, state or local regulations and guidelines as may be issued from time to time.



SPRAYS | WIPES

When using surface disinfectants:

The best way to avoid unwanted damage, discoloration, or unpredictable health effects is to use professional EPA approved products according to label directions. If more than one product is used on a surface, use only products with compatible ingredients. **CAUTION:** Using products with different active ingredients on the same surfaces may lead to unpredictable damage and toxicity. Sanitex and Sani-Cloth products with Quaternary Ammonium may be used in combination with each other.

General guidelines for use of disinfectant wipes:

- 1 Wear appropriate gloves and other (PPE) personal protective equipment when cleaning blood and body fluids (heavy soil). Heavy cleaning gloves, when compared to patient treatment gloves, provide superior protection against chemical and sharps penetration.
- 2 Follow CDC and OSAP recommendations: first clean and then disinfect clinical contact surfaces, using EPA approved intermediate-level disinfectant, or at a minimum, hospital level disinfectant.
- 3 Pull wipe out of canister and close lid tightly.
- 4 In the presence of heavy, visible, or non-visible soil:
 - CLEAN** soil from surface with first wipe.
 - DISCARD** first wipe. Do not reuse.
 - DISINFECT** with second wipe: thoroughly wet surface. Treated surface must remain visibly wet. Allow to air dry.
- 5 In the absence of soil, on a pre-cleaned surface, use a clean wipe to thoroughly wet and disinfect the surface. Allow to air dry.
- 6 Dispose of used wipes in trash, according to local regulations.

Pre-moistened wipes help avoid over-spray, drips, reducing aerosolization and excess saturation. All Sani-Cloth and Sanitex Plus wipes have a "Deep Well" lid on the canister providing a secure seal to prevent moisture loss.



MICROORGANISM	Sanitex Plus [®]		Sani-Cloth [®] Plus	Super Sani-Cloth [®]	Sani-Cloth [®] HB	Sani-Cloth [®] AF3
	Spray	Wipes				
MYCOBACTERIUM BOVIS (TB)	6 MIN	–	–	1 MIN	–	3 MIN
HEPATITIS B VIRUS	3 MIN	2 MIN	2 MIN	2 MIN	10 MIN	3 MIN
HEPATITIS C VIRUS	3 MIN	2 MIN	2 MIN	2 MIN	–	3 MIN
ACINETOBACTER BAUMONNII	5 MIN	–	–	2 MIN	–	3 MIN
ADENOVIRUS TYPE 5	–	–	–	2 MIN	–	–
ASPERGILLUS NIGER	5 MIN	–	–	–	10 MIN	–
CAMPYLOBACTER JEJUNI	–	3 MIN	3 MIN	2 MIN	10 MIN	3 MIN
CANDIDA ALBICANS	5 MIN	–	–	2 MIN	10 MIN	3 MIN
CANINE DISTEMPER	–	–	–	–	10 MIN	–
CANINE PARVOVIRUS	5 MIN	–	–	–	–	–
ESCHERICHIA COLI (E. COLI) 0157:H7	–	3 MIN	3 MIN	2 MIN	10 MIN	3 MIN
HERPES SIMPLEX VIRUS, TYPE I	–	–	–	–	10 MIN	–
HERPES SIMPLEX VIRUS, TYPE II	30 SEC	1 MIN	1 MIN	2 MIN	10 MIN	3 MIN
HIV-1	30 SEC	1 MIN	1 MIN	30 SEC	30 SEC	30 SEC
INFLUENZA STRAIN A/HK	30 SEC	1 MIN	1 MIN	2 MIN	10 MIN	3 MIN
INFLUENZA STRAIN B VIRUS (ALLEN STRAIN)	–	–	–	–	30 SEC	3 MIN
KLEBSIELLA PNEUMONIAE	10 MIN	–	–	2 MIN	10 MIN	3 MIN
LISTERIA MONOCYTOGENES	–	–	–	–	10 MIN	3 MIN
METHICILLIN RESISTANT STAPH AUREUS (MRSA)	3 MIN	3 MIN	3 MIN	2 MIN	10 MIN	3 MIN
PSEUDOMONAS AERUGINOSA ATCC 15442	3 MIN	3 MIN	3 MIN	2 MIN	10 MIN	3 MIN
POLIO I VIRUS	3 MIN	–	–	–	–	–
RESPIRATORY SYNCYTIAL VIRUS (RSV)	–	1 MIN	1 MIN	1 MIN	–	3 MIN
RHINOVIRUS	3 MIN	–	–	2 MIN	–	–
SALMONELLA ENTERICA	3 MIN	5 MIN	5 MIN	2 MIN	10 MIN	3 MIN
STAPHYLOCOCCUS AUREUS ATCC 6538	3 MIN	3 MIN	3 MIN	2 MIN	10 MIN	3 MIN
STREPTOCOCCUS PYOGENES	–	–	–	–	10 MIN	3 MIN
TRICHOPHYTON MENTAGROPHYTES	5 MIN	–	–	–	10 MIN	–
VACCINIA VIRUS	–	–	–	2 MIN	10 MIN	–
VANCOMYCIN-RESISTANT ENTEROCOCCUS FAECALIS (VRE)	5 MIN	3 MIN	3 MIN	2 MIN	–	3 MIN



LIQUIDS | TABLETS

EfferZyme® Effervescent Cleaning Tablets



- **Contains Enzymes** – Biodegradable, non-chlorine formulation contains fast acting protease enzymes. Enhances removal of blood and other soils that may be resistant to chemical detergents.
- **Neutral pH Formula** – Will not damage or harm instruments/endoscopes. Contains no harsh alkaline components and is non-corrosive.
- **Dilution Levels** – 2 tablets = 1 gallon / 1 tablet = ½ gallon. Simply place tablets in warm water.
- **Cost Effective** – Reduced shipping/handling/storage costs.
- **Pleasant Scent** – Spearmint.



Quantity	Ref. #
----------	--------

52 Tablets/Box	JET
----------------	-----

Product can be used with ultrasonic cleaners, for instrument presoak, for evacuation system cleaning, and for slow/high speed hand pieces. See package for usage instructions.

CrossZyme® Concentrated Enzyme Ultrasonic Cleaner

- **Enzymatic** – Biodegradable, phosphate and chloride free component to enhance removal of blood and proteinaceous fluids.
- **Neutral pH Formula** – Will not damage or harm instruments/endoscopes. Contains no harsh alkaline components and is non-corrosive. Safe for use on plastics, stainless steel, glass and aluminum. Contains rust inhibitors.
- **Balanced Detergent System** – Effective cleaning and easy rinsing over a wide range of concentrations and temperatures.
- **Powerfully Chelated** – Improves performance in hard water conditions.
- **Pleasant Scent** – Citrus.
- **CrossZyme Dilution Ratios:** 20 oz. water to 1 oz. for general use, 10 oz. water to 1 oz. for heavy use.



Quantity	Ref. #
----------	--------

1 Gallon Bottle / 3.785L	JEZ
--------------------------	-----

Product can be used with ultrasonic cleaners, for instrument cleaning or presoak, and for slow/high speed hand pieces. See package for usage instructions.



Tartar and Stain Remover

- **Powerful** – Ultrasonic acidic detergent removes calculus, tobacco, food stains and permanent cement from bridges, dentures and orthodontic appliances.
- **Environmentally Friendly** – Biodegradable and phosphate free.
- **Ready to use.**



Quantity	Ref. #
----------	--------

1 Gallon Bottle / 3.785L	JEZTS
--------------------------	-------

General Purpose Non-Ammoniated Ultrasonic Cleaner

- **Multi-Purpose Formula** – Use for removal of buffing compounds, pumice, rouge, tripoli, food particles, foreign matter and soils of all type
- **Safe** – Use on dentures, burs, bridges, crowns, orthodontic appliances, instruments, reamers, diamonds, aspirator tips and glass slides.
- **Concentrated** – 10:1 dilution.
- **Non-Ammoniated** – Ammonia-free - no dangerous fumes to irritate nasal passages or eyes.
- **Environmentally Friendly** – Biodegradable and phosphate-free.
- **Free Rinsing** – Leaves instruments spotless.



Quantity	Ref. #
----------	--------

1 Gallon Bottle / 3.785L	JEZNA
--------------------------	-------

SANI ProZyme™ Enzymatic Detergent



SANI ProZyme™ Enzymatic Detergent is designed to remove organic and inorganic soils such as blood, tissue and protein-rich fluids from flexible and delicate instruments. The enzyme breaks down (actually digests) the organic soils, and the detergent removes them, holding them in suspension and avoiding redepositing on the cleaned surfaces. SANI ProZyme™ Enzymatic Detergent is highly concentrated, low foaming, phosphate-free, has rust-inhibiting properties and a neutral pH.

- **Environmentally Friendly** - biodegradable, phosphate-free, non-polluting formula.
- Rinsing agents leave instruments **spot and residue-free**.
- **Highly concentrated** - low cost per use.



Quantity	Ref. #
----------	--------

NEW! 1 Gallon Bottle	JED
-----------------------------	-----

½ -2 oz	Per Gallon of Water
---------	---------------------

2-3 min oz	Manual Soak
------------	-------------

15- 20 min	For Heavy Debris
------------	------------------

Quantity	Ref. #
1 Gallon Bottle	JGLUT NEW!

SANI Glut™ Glutaraldehyde 3%

SANI Glut™ Glutaraldehyde 3% is a reusable manual liquid chemical sterilant and high-level disinfectant. SANI Glut™ Glutaraldehyde 3% is recommended for all immersible and heat sensitive instruments. Activation is required. Solution needs to be tested with the appropriate test strip prior to each use to verify effective disinfection concentration.

- **Fast acting** – 25 minute high level disinfection at 25°C.
- **Environmentally friendly** - biodegradable and formaldehyde-free.
- **Excellent performance** - performs more effectively than products that use formaldehyde, glyoxal or other corrosive materials.
- **Economical** - 28-day reuse life.

10 min. High-Level Disinfection*

10 Hour Sterilization*

28 day Reuse Period

*at 25°C



Quantity	Ref. #
1 Gallon Bottle	ML02-0127

Rapicide® OPA/28 High Level Disinfectant

Rapicide® OPA/28 provides a superior way to reprocess heat-sensitive, semi-critical instruments that are unsuitable for sterilization. With Rapicide® OPA/28, you have a powerful, ready-to-use disinfectant carrying an active ingredient of 0.575% ortho-phthalaldehyde (OPA). Only Rapicide® OPA/28 test strips should be used to verify MRC before each use.

- Rapicide® OPA/28 has a **reuse period of up to 28 days**, twice the reuse period of 14 day OPA brands.
- **High-level disinfection in just 10 minutes** at room temperature (20°C minimum).
- Guaranteed materials compatibility.
- The ultimate combination of **safety, convenience and value.**

Visit OPA28.com for valuable resources about OPA/28 and high-level disinfection and free 1-hour CE on HLD.

10 min. High-Level Disinfection

28 day Reuse Period



Saves TIME



Saves MONEY



Lasts LONGER



SANI HLD Holder

The Sani HLD Holder is a simple accessory to make the process of high-level disinfection easier. The Sani HLD Holder can be used with cleansers (enzymes and detergents), disinfectants (phenols) and cold sterilants (glutaraldehyde, OPA and chlorines). Dimensions: 13 7/8" x 9 5/8" x 5 1/8".

- **High capacity** - Hold up to 1 gallon of solution.
- **Easy to use** - Inner basket is easily removed and allows for convenient draining of the disinfectant.
- **Convenient** - Hinged lid contains a calendar to remind staff of when the solution needs to be changed.



Quantity	Ref. #
Each	JHOLD



Rapicide® OPA/28 Test Strips

Test strips provide MRC verification of Rapicide® OPA/28 high-level disinfectant. Rapicide OPA/28 Test strips are designed specifically for specific use with our disinfectant.


- **User friendly** - Large, easy to read indicator pad.
- **Intuitive** - Pass/fail reference color chart on bottle.
- **Fast** - 3-second dip time and 90-second read time.



Quantity	Ref. #
Rapicide® OPA/28 Test Strips 50 Strips/Bottle	ML02-0137

FLUORIDES | TRAYS

Description	Quantity	Ref. #
Mint	50/Box	UFVPM NEW!
Melon	50/Box	UFVPM NEW!

 = NOT MADE WITH NATURAL RUBBER LATEX

ProGuard™ 5% Sodium Fluoride Varnish

ProGuard™ is the ideal fluoride varnish for a simple, hassle-free procedure that can be performed in a few minutes. ProGuard's convenient single-dose packaging, smooth consistency, and excellent flow characteristics make application a breeze. Patients appreciate ProGuard's pleasant flavors, clear-setting formula, quick application process, and efficient, highly effective desensitization.

- Available in two pleasant flavors: mint and melon.
- Dries to a clear finish.
- Sets fast, even in the presence of saliva.
- Quickly and easily applied with minimal set up and clean up time.
- Convenient, easy-to-use single-dose packs.
- Smooth consistency with good flow characteristics.
- Efficient fluoride release with high substantivity.
- Sweetened with Xylitol, gluten-free.
- Free of: soy, milk, eggs, gluten, aspartame and saccharin



Description	Size	Ref. #
Bubble Gum	16 oz/ 480 ml	UFGABG
Cherry	16 oz/ 480 ml	UFGAC
Cotton Candy	16 oz/ 480 ml	UFGACC
Grape	16 oz/ 480 ml	UFGAG
Marshmallow	16 oz/ 480 ml	UFGAMAR
Orange-Vanilla	16 oz/ 480 ml	UFGAOV
Strawberry	16 oz/ 480 ml	UFGAS
Watermelon	16 oz/ 480 ml	UFGAW
Dye-Free Mint	16 oz/ 480 ml	UFGADM



Zap® Fluoride Gel

- **Smooth, creamy gel with 1.23% APF Fluoride ion.**
- **Advanced thixotropic formula** flows under pressure for greater coverage.
- **Unique formula** for longer-lasting flavor retention.
- **Soft dispensing bottle** with convenient side window.
- **Nine patient-pleasing flavors!**
- **Comes in convenient 16oz. dispensing bottles.**
- Free of: Tree nuts, nuts, soy, milk, eggs, gluten, aspartame, sulfite, and parabens. **Contains Xylitol.**

MADE IN USA 

Description	Size	Ref. #
Mint	16oz/ 480 ml	UFGNM



Zap® Neutral pH Fluoride Gel


- **0.90% fluoride ion.**
- **Special antibacterial formulation.**
- **Unique formula for long-lasting flavor** retention – increased patient acceptance.
- **Four minute treatment.**
- **Soft dispensing bottle** with side window.
- **Comes in convenient 16oz. dispensing bottles.**
- **Gluten and paraben free. Contains Xylitol.**

MADE IN USA 

Description	Quantity	Ref. #
Small - Yellow	50/Box	YFTHS
Medium - White	50/Box	YFTHM
Large - Blue	50/Box	YFTHL

Zap® Dual Arch Fluoride Gel Trays

- **Soft, anatomical design** for comfortable, natural fit.
- **Buckle ridges** improve fluoride access to teeth.
- **Three color-coded sizes** for easy identification.
- **Long, interlocking handles.**
- **Unique “space-saver” package** for easy storing and dispensing.

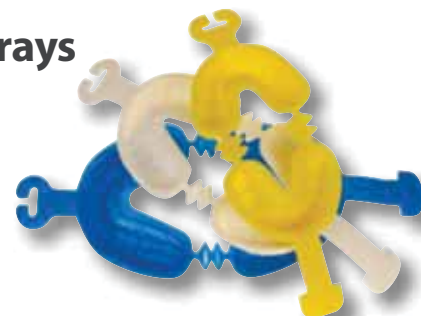

 = NOT MADE WITH NATURAL RUBBER LATEX



TOPICAL GEL | PROPHY PASTE

GumNumb® Topical Anesthetic Gel

- **Contains 20% benzocaine.**
- **Rapid onset** within 30 seconds and a short duration with virtually no systemic absorption.
- **Special formulation minimizes salivation.**
- **Provides temporary relief of pain from local anesthetic injections**, periodontal curettage, impression taking and intra-oral radiographs.
- **Available in 6 flavors.**
- Free of: Nuts, soy, milk, eggs, gluten, and aspartame.



MADE IN USA

Description	Size	Ref. #
Bubble Gum	1oz/29.6 ml Jar	UTBG
Cherry	1oz/29.6 ml Jar	UTCH
Grape	1oz/29.6 ml Jar	UTG
Mint	1oz/29.6 ml Jar	UTM
Pina Colada	1oz/29.6 ml Jar	UTPC
Strawberry	1oz/29.6 ml Jar	UTS

Sparkle® Prophylaxis Paste with Xylitol

- **Patients can be choosy!** Now you can offer your patients better tasting fun choices for tooth polishing...with Patient's Choice Sparkle® Prophylaxis Paste.
- **Enhance patient satisfaction:** Your patients will enjoy a splatter-free experience with our advanced formulation to improve paste consistency, designed to spread easily and completely rinse out, leaving no residual grit.
- **Faster results with better protection:** Designed for maximum stain removal with minimal enamel loss, contains 1.23% Fluoride ion.
- Free of: Nuts, soy, milk, eggs, gluten, aspartame, and corn/corn products. **Contains Xylitol.**



Assorted Flavors Includes:

Cherry, Mint, Orange Vanilla

Berrylicious Assorted Flavors Includes:

Raspberry, Blueberry, Strawberry



Description	Quantity	Ref. #
Coarse Grit		
Mint	12oz/340gr Jar	UPC12
Berrylicious Asst.	200/Box	UPCB
Bubble Gum	200/Box	UPCBG
Cherry	200/Box	UPCCH
Mint	200/Box	UPCM
Orange Vanilla	200/Box	UPCOV
Assorted Flavors	200/Box	UPCA
Medium Grit		
Berrylicious Asst.	200/Box	UPMB
Bubble Gum	200/Box	UPMBG
Cherry	200/Box	UPMCH
Mint	200/Box	UPMM
Orange Vanilla	200/Box	UPMOV
Assorted Flavors	200/Box	UPMA
Fine Grit		
Bubble Gum	200/Box	UPFBG

Sparkle® FREE Prophylaxis Paste with Xylitol

- **The choice for patients during or after whitening/bleaching of teeth.**
- **Free of dyes, fluorides and gluten.**
- **Splatter-free formula** preferred by patients.
- **Four Sparkle-licious flavors and 2 grits:** Cinnamon, Fruity, Spearmint and White Chocolate in either medium or coarse grit.
- **200 single-use cups per box.**
- Free of: Nuts, soy, milk, eggs, gluten, aspartame, sulfite, and corn/corn products. **Contains Xylitol.**



MADE IN USA

Description	Quantity	Ref. #
Coarse Grit		
Cinnamon	200/Box	UPSFCC
Fruity	200/Box	UPSFCE
Spearmint	200/Box	UPSFCS
White Chocolate	200/Box	UPSFWC
Medium Grit		
Cinnamon	200/Box	UPSFMC
Fruity	200/Box	UPSFME
Spearmint	200/Box	UPSFMS
White Chocolate	200/Box	UPSFMC



PROPHY ANGLES

Description	Quantity	Ref. #
Soft Cup, Gray	100/Bag	TPLUSPASC NEW!
Firm Cup, White	100/Bag	TPLUSPAFC NEW!
Tapered Brush	100/Bag	TPLUSPATB NEW!

twist^{plus} Oscillating Disposable Propphy Angles

The new twist[®] plus provides more efficient and ergonomic cleaning.

- 120° reciprocating motion allows for continuous contact polishing and reduces splatter.
- Small head design improves visibility and increases maneuverability.
- The enhanced cup design with fluting improves pick-up and interproximal polishing. Cup seals very tightly to the head, reducing the chance of catching soft tissue in the corners of the mouth.
- Robust gear mechanism provides a smooth, continuous motion and the recessed grip design makes the modified pen grasp comfortable and ensures a secure grasp – both improving the overall ergonomics of the angle.




Description	Quantity	Ref. #
Soft Cup, Purple	100/Box	TPASSC1
Firm Cup, White	100/Box	TPASFC1
Soft Cup, Purple	500/Box	TPASSC5
Firm Cup, White	500/Box	TPASFC5
Tapered Brush	100/Box	TPATBC1

Sparkle[®] Propphy Angle

Created with hygienists' feedback, Sparkle's patented second-generation gear design runs vibration-free and noise-free.

- Beveled gear ensures smoother running angle that won't lock up or overheat.
- Improved soft cup design: dark color for greater visibility, flared design to improve subgingival access and reduce splatter to minimize cross-contamination.

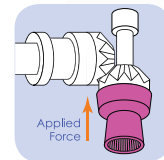

Description	Quantity	Ref. #
Soft Cup, Purple	100/Box	TPASEZ
Firm Cup, White	100/Box	TPAFEZ



Sparkle[®] EZ Contra Angle

- Advanced ergonomic design reduces hand fatigue and micro-traumas, allows for greater access.
- Provides all additional benefits of Sparkle Propphy Angle.



Sparkle V[®] 5% Sodium Fluoride Varnish with Xylitol

- Dries to a natural tooth shade on contact.
- No need for prophy prior to use.
- Single-use package with included bendable brush applicator.
- Sets rapidly in the presence of saliva – no need to isolate teeth.
- Free of: Tree nuts, nuts, soy, milk, eggs, gluten, aspartame and saccharin.





COMFORT CUSHIONS

Crosstex Imaging Comfort Cushions include a variety of cushioning and protective devices that improve patient comfort and maintain proper infection control throughout film based and digital radiography procedures. Patient comfort in the imaging process is critical to accuracy. Discomfort can prevent patients from fully occluding on the bite block and compromise the quality of the resulting digital X-ray.

Edge-Ease®

- Provide a soft comfortable edge to protect the patient's soft tissue from x-ray film and sensor abrasions.
- Allow the patient to bite harder on positioning devices, for better periapical positioning and more accurate diagnostic X-rays.
- Easily adhere to film packets, phosphor plates and digital sensor barrier sleeves.
- Save time and money while decreasing the need for retakes.



Description	Quantity	Ref. #
Large (Blue) Fits long edge of size 2 film and sensors	300/Box	BWEEB
Small (White) Fits short edge of size 1-2 film size and all 0-1 sensors	300/Box	BWEEW



Bite Wing-Ease®

- A seamless, cushioned barrier sleeve that adjusts snugly to fit all types of sensors and folds to create a bite tab.
- Enable easy sensor positioning for more accurate images.
- All-in-one seamless barrier sleeve, comfort cushion and bite tab.
- Bite tab thickness protects sensor cord.
- Can be used with sensors and Rinn® XCP® system components.



Description	Quantity	Ref. #
Large (White) Adjustable to fit all size 2 sensors	50/Box	BWEL
Small (Blue) Adjustable to fit all size 0-1 sensors	50/Box	BWES




Wrap-Ease®

- A fully adjustable, all-in-one seamless barrier sleeve, comfort cushion and positioning aid that fits all brands of sensors.
- Cushioned sensors provide maximum patient comfort.
- Converts autoclavable film holders into sensor holders.
- Great for vertical bite wings.
- Eliminates the need for disposable sensor holders.



Description	Quantity	Ref. #
Large (White) Adjustable to fit all size 2 sensors	50/Box	BWWEL
Small (Blue) Adjustable to fit all size 0-1 sensors	50/Box	BWWES

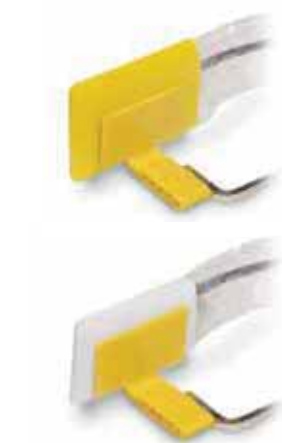

It's-A-Wrap®

- Seamless, cushioned barrier sleeves designed for use with Schick® CDR® disposable bite holder tabs and Rinn®-style positioners.
- Available in two sizes: yellow fits size 2 sensors; white fits size 0-1 sensors.
- Fully adjustable to all sensor types and brands.
- Seamless and cushioned for maximum patient comfort.
- Protects valuable sensors.
- Quick, easy and economical.



Description	Quantity	Ref. #
Large (Yellow) Adjustable to fit all size 2 sensors	50/Box	BWRAPL
Small (White) Adjustable to fit all size 0-1 sensors	50/Box	BWRAPS



SENSOR SLEEVES | BITEWING TABS

Description	Quantity	Ref. #
Large (Yellow) Fits size 2 sensor	50/Box	BWSSL




Sensor Slippers®

- Cushioned sensor sleeve built for use with gripping-style holders, such as Rinn® Uni-Grip® and Eezee-Grip®.
- Foam cushion stabilizes sensor in the holder and prevents slippage.
- Creates a softer edge that won't abrade corners of the mouth.
- Seamless sleeve won't split open.



Description	Quantity	Ref. #
Large (Pink) Fits size 2 film and sensors	100/Box	BWSEL
Small (Blue) Fits size 0-1 film and sensors	100/Box	BWSES

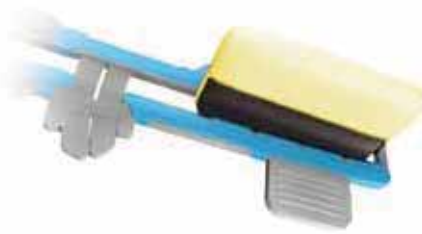

Slip-Ease®

- Unique protective cover slips over all brands of x-ray film, plates, sensors and positioning devices.
- Works best with gripping-style sensor holders.
- Smoothes edges and corners of devices to add comfort.
- Adheres with patented Safety Strip to prevent accidental aspiration.
- Requires only one for full mouth series using phosphor plates or wireless sensors.
- Helps protect phosphor plates from being scratched.



Description	Quantity	Ref. #
Size 1 Large (Clear) 1.625" x 8.5"	500/Box	BCXSL
Size 2 Small (Clear) 1.375" x 8.25"	500/Box	BCXSS

Digital Sensor Sleeves

Traditional sensor sleeves have rough seams that can irritate delicate mucosal tissue during radiographic procedures. Crosstex Digital Sensor Sleeves have smooth seams – eliminating the patient discomfort that rough seams can cause.

- Manufactured from soft polyethylene.
- Provide reliable protection against cross-contamination.
- Convenient sizing to fit all common digital imaging sensors.
- Covers and protects valuable sensors.



Rough seams of traditional sensor sleeve



Smooth seams of the Crosstex Digital Sensor Sleeve



Description	Quantity	Ref. #
Bite Wing Tabs	500/Box	YBT



Bite Wing Tabs®

- Self-adhesive, latex-free tabs ensure accurate, distortion-free bite wing radiographs and patient comfort.
- Attaches easily to both adult and pediatric x-ray films.
- Tabs hold securely when patients close their mouth to ensure precise positioning.
- Pop-out dispenser packaging saves time.
- Convenient, sanitary and economical.





SPONGES

UltraPure® Non-Woven Sponges 4 Ply, Non-Sterile

A great new way to reduce your carbon footprint:

Eco-friendly UltraPure Non-Woven Sponges, the newest member of our non-woven product family. They're soft, lint-free and absorb faster than synthetics, rayon and cotton blends, thanks to their superior basis weight material. And, unlike typical synthetic non-woven sponges, they're biodegradable. So you can provide improved treatment for your patients . . . and the environment!

- **Made with 100% USA-grown cotton** for increased comfort. Soft and abrasion resistant. 4-ply, non-sterile. Superior, faster absorbency (40-45 GSM) compared to synthetics, rayon and cotton blends.
- **Eco-friendly** – sponges biodegrade.
- **No toxic chemicals used in manufacturing process** – all-natural whitening process (H₂O₂ whitening); no bleach or chlorine.
- **Improved infection control** – exclusive automatic manufacturing process reduces cross-contamination risks and provides lowest bioburden levels.
- **Safe for all** – hypoallergenic, non-toxic.
- **Multi-purpose** – use for cleaning, prepping, placement on incision/extraction sites.
- **Won't adhere to instruments** – lint free.



100% cotton grown in USA



Ultra Gauze® Non-Woven Sponges 4 Ply, Non-Sterile

- **More absorbent than traditional sponges** – exclusive 70/30 rayon/polyester formed non-woven material replicates the look of woven gauze.
- **Releases easily** from incision/extraction sites.
- **Extremely soft**; produces little or no lint.
- **Multi-purpose** – can be used for cleaning, prepping and placement on incision/extraction sites.



* Gram Weight: Weight per square meter of rayon/poly fabric.



Description	Quantity	Ref. #
2" x 2" / 5 x 5 cm 45 GR WT*	5,000/Case	ENC100
4" x 4" / 10 x 10 cm 40 GR WT*	2,000/Case	ENC4100

Description	Quantity	Ref. #
2" x 2" / 5 x 5 cm 40 GR WT*	5,000/Case	ENCNWU
2" x 2" / 5 x 5 cm 40 GR WT*	8,000/Case	ENCNWUB
3" x 3" / 7.62 x 7.62 cm 40 GR WT*	4,000/Case	ENC3NWU
4" x 4" / 10 x 10 cm 40 GR WT*	2,000/Case	ENC4NWU

SPONGES

PREMIUM 4 Ply Non-Sterile

Description	Quantity	Ref. #
2" x 2" / 5 x 5 cm 50 GR WT*	5,000/Case	ENCNW
3" x 3" / 7.5 x 7.5 cm 40 GR WT*	4,000/Case	ENC3NW
2" x 2" / 5 x 5 cm 50 GR WT*	4,000/Case	ENCNW4K
4" x 4" / 10 x 10 cm 40 GR WT*	2,000/Case	ENC4NW

*Gram Weight: Weight per square meter of rayon/poly fabric

ADVANTAGE PLUS® 4 Ply Non-Sterile

Description	Quantity	Ref. #
2" x 2" / 5 x 5 cm 35 GR WT*	5,000/Case	ENCNWL

ADVANTAGE 4 Ply Non-Sterile

Description	Quantity	Ref. #
2" x 2" / 5 x 5 cm 30 GR WT*	5,000/Case	ENCNWLA
3" x 3" / 7.5 x 7.5 cm 30 GR WT*	4,000/Case	ENC3NWLA
4" x 4" / 10 x 10 cm 30 GR WT*	2,000/Case	ENC4NWLA

PREMIUM Non-Sterile

Description	Quantity	Ref. #
2" x 2" / 5 x 5 cm 125 GR WT*	5,000/Case	ENCC
3" x 3" / 7.5 x 7.5 cm 125 GR WT*	4,000/Case	ENCC3
4" x 4" / 10 x 10 cm 125 GR WT*	2,000/Case	ENCC4

STERILE

Description	Quantity	Ref. #
2" x 2" / 5 x 5 cm 125 GR WT*	5,000/Case	ESCC

ADVANTAGE PLUS® Non-Sterile

Description	Quantity	Ref. #
2" x 2" / 5 x 5 cm 90 GR WT*	5,000/Case	ENACC

*Gram Weight: Weight per sleeve.

Description	Quantity	Ref. #
2" x 2" / 5 x 5 cm 12 ply	8,000/Case	ENC212
2" x 2" / 5 x 5 cm 8 ply	5,000/Case	ENC2
3" x 3" / 7.5 x 7.5 cm 12 ply	4,000/Case	ENC312
3" x 3" / 7.5 x 7.5 cm 8 ply	4,000/Case	ENC38
4" x 4" / 10 x 10 cm 16 ply	2,000/Case	ENC416
4" x 4" / 10 x 10 cm 12 ply	2,000/Case	ENC412
4" x 4" / 10 x 10 cm 8 ply	4,000/Case	ENC48

Non-Wovens 4 Ply, Non-Sterile

- Unique rayon/polyester blend.
- Provides maximum absorbency.
- Less adhesion to wounds for faster healing.
- Soft, smooth, strong, virtually lint free.
- Ideal for sustained patient use, disinfecting, general cleaning.

**Cotton-Filled Sponges Exodontia**

- Made from USP cotton gauze.
- Pure, virgin white cotton with 100% cotton-fiber filling.
- Highly absorbent.

**All-Gauze Sponges Premium, Non-Sterile**

- Made from USP 100% gauze.
- Available in 8 ply, 12 ply or 16 ply.
- Highly absorbent.



United States Pharmacopeia (USP) cotton/cotton gauze defines specific standards based on thread count, length, width, weight, absorbency, ignited residue, fatty matter and alcohol-soluble dyes. Manufacturers of this material must conduct ongoing tests that assure the product meets and/or exceeds the standards established by the USP. All Crosstex sponges are manufactured in facilities having an in-house laboratory that continually monitors for USP compliance.



COTTON ROLLS | ACCESSORIES

Cotton Rolls Non-Sterile

- Made from 100% cotton.
- Contain NO cellulose or rayon fibers.
- Soft, pliable, non-linting.
- Will NOT adhere to mucous membrane.
- Maintain shape when wet.
- Absorb up to 10 times their weight.



PREMIUM Description	Quantity	Ref. #
---------------------	----------	--------

#2 Medium 1.5" x .375" / 4 x 1 cm	2,000/Box	DNC
---	-----------	-----

ADVANTAGE PLUS® Description	Quantity	Ref. #
-----------------------------	----------	--------

#2 Medium 1.5" x .375" / 4 x 1 cm	2,000/Box	DNAC
--------------------------------------	-----------	------

Braided Cotton Rolls Non-Sterile

- Made from 100%, short fiber cotton.
- Bound by thin strands of polyester filament.
- Starch-free for maximum softness.
- Tightly braided to maintain shape when wet.
- Highly absorbent.
- Will NOT adhere to mucous membrane.



Description	Quantity	Ref. #
-------------	----------	--------

1.5" x .375" / 4 x 1 cm	2,000/Box	DNCB
-------------------------	-----------	------



Cotton Roll Dispenser Deluxe

- Flip-up top for easy loading.
- Holds up to 50 cotton rolls.
- Slide out drawer dispenses up to 4 rolls.
- Non-skid base for easy dispensing.
- Heavy duty, positive aseptic barrier.



Description	Size	Ref. #
-------------	------	--------

Blue	4" x 3.3" x 2" / 10 x 9 x 5 cm	PDCBL
White	4" x 3.3" x 2" / 10 x 9 x 5 cm	PDCWH

Cotton Tipped Applicators

- 100% cotton tip.
- All wood dowels.
- Extremely tight bud minimizes linting.
- Applicators are sealed in autoclavable bags of 100.



PREMIUM Description	Quantity	Ref. #
---------------------	----------	--------

3" / 8 cm	1,000/Box	H3C
6" / 15 cm	1,000/Box	H6C

ADVANTAGE PLUS® Description	Quantity	Ref. #
-----------------------------	----------	--------

3" / 8 cm	1,000/Box	H3AC
6" / 15 cm	1,000/Box	H6AC



Amalgam Squeeze Cloths

















- Made from fine, closely woven white cotton.
- Minimal linting.



PREMIUM Description	Quantity	Ref. #
---------------------	----------	--------

3" / 8 cm	500/Box	S5C
-----------	---------	-----

Bard-Parker® BLADES | SCALPELS

		STERILE				NON-STERILE
		CARBON STEEL		STAINLESS STEEL		CARBON STEEL
	SIZE	RIB-BACK®	SAFETYLOCK™ RIB-BACK®	CONVENTIONAL	PROTECTED	RIB-BACK®
	10	371110	371150	371210	373910	371310
	11	371111	371151	371211	373911	371311
	12	371112	371152	371212		371312
	15	371115	371153	371215	373915	371315
	20	371120	371154	371220	373920	371320
	21	371121	371155	371221	373921	371321
	22	371122	371156	371222	373922	371322
	23	371123	371157	371223	373923	371323
	24					371324
	25					371325
	60					371340
	10A			371700		
	12B			371712		
	15C			371716	374305	
	11					

DISPOSABLE SCALPELS				
STERILE			NON-STERILE	
CONVENTIONAL	PROTECTED	PROTECTED (LONG HANDLE)	CONVENTIONAL	PROTECTED
371610	372610	372710	371630	373810
371611	372611	372711	371631	373811
371612	372612			
371615	372615	372715	371633	373815
371620	372620		371634	373820
371621	372621		371635	373821
371622	372622			
371623	372623			373823
371613	372613			
			371641	

Individually packaged, 50 blades per box.
3 boxes per case Non-Sterile, Carbon Steel, Rib-Back® Individually packaged, 25 strips of 6 blades per case, 150 blades per case

10 per box, 10 boxes per case
* Mini Scalpels 100 per bag, 10 bags per case
Protected Non-Sterile Scalpels: 10 per Strip, 50 strips per case




Bard-Parker® HANDLES



MADE IN USA



LATEX

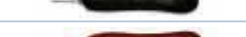CONVENTIONAL HANDLES - NON-STERILE			
	REF. #	BLADE SIZES	SIZE
	371030	10-15	3
	371031	10	3L
	371032	10	3LA
	371040	20-25	4
	371041	20-25	4L
	371050	10-15	5
	371060	20-25	6
	371070	10-15	7
	371080	60	8
	371090	10-15	9
5 per case			

PROTECTED HANDLES - NON-STERILE			
	REF. #	BLADE SIZES	SIZE
	374030	10-15	3
	374031	10-15	3L
	374040	10	4
	374041	20-23	4L
5 per case			



NEEDLES | SYRINGES



All BD products are available exclusively through Crosstex in the US dental market.



Luer-Lok® Needles

- Available in a variety of gauges and lengths.
- All gauges and lengths come in boxes of 100.

CAUTION; do not bend or alter needle shaft prior to use or insert needle shaft all the way to the hub as needle breakage and possible injury may result. To insure safety, always handle needles with care. If bent or damaged, no attempt should be made to straighten needles or use product. Do not recap needles except as permitted by OSHA, CDC or local policy.



Syringe / Needle Combinations / Luer-Lok® Tip

- Available in a variety of gauges, sizes, and lengths.
- All gauges and lengths come in boxes of 100.



Syringes

- Available in a variety of sizes.



Tuberculin Syringe / Detachable Needle, Slip Tip

- Detachable needles.
- Intradermal bevel.



SHARPS CONTAINERS



Other sizes and styles available upon request



Phlebotomy (Dual Access)

- Clear top provides visualization of collector fill-levels.



Multi-Use, One Piece

- Convenient, vertical "point-first" drop.
- One-way funnel valve to prevent needlesticks.



Wallmate Sharps

- 5.4 quart side entry pearl sharps container.
- Attached glove dispenser cabinet.
- Locking wall bracket for added security.



Gauge	Length	Ref. #
23G Short	1" / 2 cm	305145
25G Extra Short	.625" / 1 cm	305122
25G Short	1" / 2 cm	305125
25G Long	1.5" / 4 cm	305127
27G Extra Short	.5" / 1 cm	305109
27G Long	1.25" / 3 cm	305136
30G Extra Short	.5" / 1 cm	305106
30G Long	1" / 2 cm	305128

Gauge	Size	Length	Ref. #
21G	3 ml	1" / 2 cm	309575
21G	3 ml	1.5" / 4 cm	309577
22G	3 ml	1" / 2 cm	309572
22G	5 ml	1.5" / 4 cm	309631
25G	3 ml	.375" / 1 cm	309570

Size	Quantity	Ref. #
3 ml	100/Box	309657
5 ml	100/Box	309646
10 ml	100/Box	309604
20 ml	48/Box	302830

Gauge	Size	Quantity	Ref. #
25G	1 ml	Each	309626

Size / Feature	Color	Ref. #
1.5 Quart, Dual Access	Red	305487

Size / Feature	Color	Ref. #
3.3 Qt., Reg. Funnel	Red	305488
6.9 Qt., Reg. Funnel	Red	305489
8.2 Qt., Reg. Funnel	Red	305490
5 Gal., Lg. Funnel	Red	305477

Size / Feature	Color	Ref. #
5.4 Qt., Side Entry Collector, Glove Cabinet	Pearl	305550

Waterline Purification System DentaPure®

Take the guesswork out of water compliance!

DentaPure's award-winning technology offers a safe, simple and economical dental unit water purification solution. No tablets, no shocking, no complicated monitoring protocols.

DentaPure utilizes the same proven NASA-developed technology originally developed to provide failsafe, guaranteed potable water in space. DentaPure utilizes a non-allergenic isotope of the germicide iodine to deliver clean, pure water that meets or exceeds all microbiological standards (less than 500 CFU/mL).

- Registered as an antimicrobial product.
- Available for municipal and bottle systems.
- Eliminates human error and hidden labor costs.
- Extremely cost-effective as compared to traditional purification systems.
- No tablets and no distilled water required.
- No hazardous silver. No special trash disposal requirements.



ACTIVE INGREDIENT:

Iodine*	46%
Other Ingredients	54%

*Bonded to strong base anion exchange resin

TOTAL 100%

Unit	DP365M	DP365B	DP40B
Iodinated Resin	38.0 cc	16.0 cc	4.0 cc

The elemental iodine used in DentaPure contains NO allergenic proteins.

BOTTLE SYSTEMS

Description	Quantity	Ref. #
365 Day Cartridge	1 Ea.	DP365B

DentaPure® 365-Day Independent Water Bottle Cartridge

A point-of-source cartridge that retrofits your unit's existing water bottle pickup tube.

- Perfect for long-term use.
- Installs directly into dental unit's water supply bottle (see diagram).
- Provides continual water quality to the unit for the life of the cartridge.
- Change every 365 days.
- Complimentary reminders are provided with product registration.
- Simple installation via included quick-connect fittings.



MADE IN USA 

Description	Quantity	Ref. #
60 Day Cartridge	1 Ea.	DP40B

DentaPure® 60-Day Independent Water Bottle Cartridge

A point-of-source cartridge that retrofits your unit's existing water bottle pickup tube.

- Perfect for short-term use.
- Installs directly into dental unit's water supply bottle.
- Provides continual water quality to the unit for the life of the cartridge.
- Change every 60 days.
- Complimentary reminders are provided with product registration.
- Simple installation via included quick-connect fittings.



MADE IN USA 



MUNICIPAL SYSTEMS

DentaPure® 365-Day Independent Water Municipal Cartridge

A point-of-source purification cartridge that retrofits your unit's existing municipal water line.

- Perfect for long-term use.
- Installs directly into the chair's municipal system (J-Box).
- Provides continual water quality to the unit for the life of the cartridge.
- Change every 365 days.
- Complimentary reminders are provided with product registration.
- Installation fittings included.



Description	Quantity	Ref. #
365 Day Municipal (City) Cartridge	1 Ea.	DP365M

TESTING STRIPS



Iodine Test Strips

- Test strips may be used to verify adequate (higher than 0.5 ppm) levels of iodine release.
- Use after installation & for occasional assurance testing.
- Bottle includes 50 test strips.



Description	Quantity	Ref. #
50 Test Strips	1 Ea. Bottle	TEST

SUGGESTED USE TABLE:

DENTAPURE® UNIT SIZE	IF WATER USAGE RECORDS ARE KEPT	IF WATER USAGE RECORDS ARE NOT KEPT	IF IODINE CONTENT IS MONITORED
DP40B	40 Liters	60 Calendar Days	Change when iodine output falls to less than 0.5 ppm
DP365M & DP365B	240 Liters	365 Calendar Days	

Adherence to CDC/ADA guidelines for aseptic procedures including a 2 minute morning flush, a 20-30 second flush between patients, and hand piece sterilization must be continued.


Studies show that water heaters can be a source of increased biological contamination. For optimal biological purity Crosstex recommends not using water heaters.

Easy Install Diagram

DentaPure is a simple point-of-source iodinated resin bead cartridge that retrofits your unit's existing water bottle pickup tube (DP365B) or municipal waterline (DP365M).



Visit
DentaPure.com
for clinical articles
and installation
videos

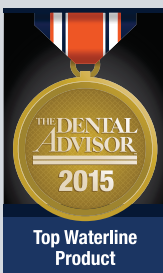

The Crosstex Family of Educational Resources

As a global leader in infection prevention and control, we are passionate about producing high quality, innovative products. But we don't stop there. Crosstex has an ongoing commitment to educate clinicians on proper infection control products, and protocol, in order to safeguard individuals and practices across the globe.

As you consider our products, please familiarize yourself with our suite of educational resources. Watch product tutorials and demonstrations, access CE accredited courses, share online training videos, sterility assurance work tools, compliance information and much more.



CROSSTEXLEARNING.COM

Register for CE accredited courses on a broad spectrum of topics. Topics range from face mask selection and ASTM levels to instrument reprocessing best practices.



**STERILIZATION CLASSROOM
SPSMEDICAL.COM/EDUCATION**

Crosstex/SPSmedical offers a series of educational CD-ROMs on a variety of infection control topics; accredited audio seminars; and video training that can be used for group or individual training.



CROSSTEXBMS.COM

Crosstex Biological Monitoring Services Website: customers can access spore test results online 24/7 and access a host of instrument reprocessing information and work tools.



CHECKYOURTEST.COM

Crosstex/SPSmedical sterilization products and spore testing results website, receive online results and educational information.



CROSSTEX.COM/SECUREFIT

Experience the custom fit of a Secure Fit® Technology face mask. 3X's more protection than a standard earloop mask (data on file). Better Fit = Reduced Risk!



OPA28.COM

The fastest and longest lasting high-level disinfectant on the market today. Site includes directions for use, wall posters and work logs.



DENTAPURE.COM

Infection Control Helpline

TOLL FREE (1-855-878-3745)
1-8558-STERILE
8am - 5pm M-F MST

Contact a clinical consultant today.

Crosstex is committed to your practices' success and safety.



Facebook.com/CrosstexProtects



YouTube.com/CrosstexProtects



Personal Protection

Page 1



Operatory Protection

Page 15



Sterilization

Page 20



Treatment Accessories

Page 24



Surface Disinfectants

Page 28



Cleaning Solutions

Page 31



High-Level Disinfectant

Page 32



Preventives

Page 34



Imaging Comfort

Page 37



Cotton/Non-Wovens

Page 39



Surgical & Waste Accessories

Page 42



Waterline Treatment

Page 44

CROSSTEX | PROTECTS®
A CANTEL MEDICAL COMPANY

10 Ranick Road
Hauppauge, NY 11788-4209 USA

Tel: 631-582-6777 | 888-276-7783

Fax: 631-582-1726 | E-mail: crosstex@crosstex.com

crosstex.com

California • Colorado • Georgia • New York • Pennsylvania

Argentina • Japan • Singapore • The Netherlands



Proud member of the following associations:



PERSONAL PROTECTION

FACE PROTECTION

Masks | Shields

Isolator® Plus N95 Particulate Respirator	1
Isolite® Earloop	1
Surgical Molded	1
Isofluid® Earloop w/ Secure Fit® Technology	1
Isofluid® Plus Pure w/ Secure Fit® Technology	3
Isofluid FogFree® Earloop w/ Secure Fit® Technology	2
Isofluid FogFree® Earloop w/Shield w/ Secure Fit® Technology	2
Isofluid® Earloop	2
Isofluid FogFree® Earloop	2
Isofluid FogFree® Earloop w/Shield	2
Isofluid® Plus Earloop	3
Procedural Earloop w/Secure Fit® Technology	1
Procedural Earloop	2
Surgical Tie-on	2
Ultra® Sensitive Earloop w/ Secure Fit® Technology	1
Ultra® Sensitive FogFree™ Earloop w/Secure Fit® Technology	2
Ultra® Sensitive FogFree™ Earloop w/Shield w/Secure Fit® Technology	2
Ultra® Earloop w/Secure Fit® Technology	2
Ultra® Sensitive Earloop	3
Ultra® Sensitive FogFree™ Earloop	3
Ultra® Sensitive FogFree™ Earloop w/Shield	3
Ultra® Earloop	3
Ultra® FogFree™ Earloop	3
Ultra® FogFree™ Earloop w/Shield	3
Face Shield	2
Le Petit Masque	2
Patient Safety Mask with Shield	3
See-Clear® Eyeglass Wipes	3
Masks At-A-Glance Order Info	5

PATIENT TOWELS | BIBS | LINERS

Towel Dispenser	6
Polyback®, Poly-Gard®, Poly-Gard Deluxe	7
Polyback® Contour Bib	7
Advantage Plus®	7
Sani-Tab® Chain-Free® – 2-ply, 3-ply	7
Pink With A Purpose Econoback®	7
Econoback®	7
Ultragard®	7
Advantage™	7
Proback®	7
Professional® – Regular, Thrift	7

APRONS | GOWNS

Patient Barrier Aprons	8
Impervious Barrier Gowns	8

HAND CARE

Cleansers

SaniSept® Antimicrobial Soap	9
SaniClenz® Antimicrobial Skin Cleanser	9

Sanitizers

Sani-Hands® ALC Alcohol Wipes	10
SaniTyz™ Waterless Moisturizing Antimicrobial Gel	10

SaniTyz™ Wall-Mounted Dispenser	10
---------------------------------	----

Moisturizers

AloeCare Plus 3® Skin Care Lotion	11
-----------------------------------	----

GLOVES

Ambidextrous Latex, Powdered

Omega, Lightly Powdered	12
Classic, Lightly Powdered	12

Ambidextrous Latex, Powder-Free

Ultra Plus®, Friction Grip	13
----------------------------	----

Fitted Latex, Powdered

Fitted Left/Right	14
-------------------	----

Fitted Latex, Powder-Free

Ultimate Left/Right	14
---------------------	----

Non-Latex Nitrile, Powder-Free

Nitrile Fitted Left/Right	14
---------------------------	----

Copolymer Plastic Powder-Free

Ambidextrous Nitrile, Powder-Free	14
-----------------------------------	----

Ambi-ance® Nitrile

OPERATORY PROTECTION

SURFACE BARRIERS

Barrier Film–Finger–Lift Edge	15
Chair Sleeves	15
T-Handle Light Sleeves	15
Syringe Sleeves	15
Tray Sleeves	16
X-Ray Sleeve	16

EQUIPMENT PROTECTION

Bracket Tray Covers	17
Liquid-Proof Bracket Tray Covers	17
iBarrier® Touch Sensitive Sleeves	19
Headrest Covers	
Polycoated, Plastic, Non-Woven	18
Exam Table Paper	
Crepe, Smooth	19
Pillow Cases	19

TRANSPORTATION PROTECTION

Lab Transport Bags	19
--------------------	----

STERILIZATION

STERILIZATION POUCHES

Sure-Check® Pouches	20
Duo-Check® Pouches	20
Pink with a Purpose®	20

BIOLOGICAL MONITORING

ConFirm® 10 In-Office System	21
ConFirm® 24 In-Office System	21
Mail-In Monitoring Service	22

CHEMICAL MONITORING

STEAMPlus™ Type 5 Integrators	21
-------------------------------	----

INSTRUMENT PACKAGING & MONITORING

CSR Sterilization Wrap	22
Self-Sealing Autoclave Bags	23
AirView™ II Bowie-Dick Test Pack	23

Sani-Tube® Sterilization Tubing	22
Sani-Roll® Sterilization Tubing	23
Sure-Check® Strips	23
Non-Latex, Lead-Free Steam Indicator Tape	23
Steam Process Indicator Tape	23

TREATMENT ACCESSORIES

Saliva Ejectors

Comfort Plus®, Premium Plus®, Advantage, Hygoformic	24
---	----

Evacuator Tips

Surgical Aspirator Tips	24
HVE Tips	24

MaxVac Plus® Combo Tip®

Single-Use Patient Disposables	25
--------------------------------	----

Sparkle® Air Water Syringe Tips

Articulating Paper, Foil, Film	25
Tongue Depressors	26

Matrix Strips

Poly Coated Mixing Pads	26
-------------------------	----

Cups

Plastic, Medicine/Mixing	26
Cup Dispenser	26

Dental Dams

Latex, Powder-Free/Low Protein	27
Latex-Free	27

SURFACE DISINFECTANTS

Wipes

Sani-Cloth® Bleach Wipe	28
Sani-Cloth® Plus Wipe	28
Sanitex® Plus Wipe	28
Super Sani-Cloth® Wipe	29
Sani-Cloth® AF3 Wipe	29
Sani-Cloth® HB Wipe	29

Spray

Sanitex Plus® Spray	28
---------------------	----

CLEANING SOLUTIONS

Liquids & Tablets

EfferZyme® Cleaning Tablets	31
CrossZyme® Enzyme Ultrasonic Cleaner	31
Tartar and Stain Remover	31
General Purpose Non-Ammoniated Cleaner	31
SANI ProZyme™ Enzymatic Detergent	31

HIGH-LEVEL DISINFECTANT

Rapicide® OPA/28 High-Level Disinfectant	32
SANI HLD Holder	33
SANI Glut™ Glutaraldehyde 3%	32
Rapicide® OPA/28 Test Strips	33

PREVENTIVES

PATIENT'S CHOICE®

FLUORIDES | TRAYS

ProGuard™ 5% Sodium Fluoride Varnish	34
Sparkle V® 5% Sodium Fluoride Varnish	36
Zap® 1.23% APF Fluoride Gel	34
Zap® Neutral pH Fluoride Gel	34

Zap® Dual Arch Fluoride Trays	34
-------------------------------	----

TOPICAL ANESTHETIC

GumNumb® Topical Gel	35
----------------------	----

PROPHY PASTE

Sparkle® Prophyl Paste	35
Sparkle® FREE Prophyl Paste	35

DISPOSABLE PROPHY ANGLES

twist® plus Oscillating Angle	36
Sparkle® Angle	36
Sparkle® EZ Contra Angle	36

IMAGING COMFORT

Edge-Ease®	37
Bite Wing-Ease®	37
Wrap-Ease®	37
It's-A-Wrap®	37
Sensor Slippers®	38
Slip-Ease®	38
Digital Sensor Sleeves	38
Bite Wing Tabs®	38

COTTON | NON-WOVENS

SPONGES

UltraPure® Biodegradable	39
Ultra Gauze®	39
All-Gauze	40

Non-Wovens

Premium, Advantage Plus®, Advantage	40
-------------------------------------	----

Cotton-Filled (Exodontia)

Premium, Sterile, Advantage Plus®	40
-----------------------------------	----

COTTON ROLLS

Premium, Advantage Plus®	41
Braided	41

ACCESSORIES

Cotton Roll Dispenser	41
Cotton Tipped Applicators	41
Amalgam Squeeze Cloths	41

SURGICAL & WASTE ACCESSORIES

Aspen Surgical Blades	42
Aspen Surgical Scalpels	42
Aspen Surgical Handles	42
BD Luer-Lok® Needles & Syringes	43
BD Syringe/Needle Combo	43
BD Tuberculin Syringe	43
BD Sharps Containers	43

WATERLINE TREATMENT

DentaPure® Waterline Purification System	44
--	----

The information contained in this catalog is provided for informational purposes only, without any representation or warranty of accuracy or completeness of information or warranty of any kind. In particular, we can make no warranties, either expressed or implied, that the products in this catalog will fully protect the user from exposure to blood or body fluids or risk of contracting infectious diseases. OSHA requires the employer to evaluate the anticipated exposure and select the appropriate protective products that prevent contamination of clothing or the body and that comply with federal, state and/or local regulations or guidelines as may be issued from time to time. Regulations, specifications and/or standards referred to in this catalog are the appropriate regulations, specifications and/or standards in effect as of the date of printing of this catalog, and are subject to change. In addition, we may make improvements or changes in our products at any time, without notice.

Crosstex Locations:

North America

Corporate Headquarters

10 Ranick Road
Hauppauge, NY 11788
Tel: 631-582-6777
Toll Free: 888-276-7783
Fax: 631-582-1726
Email: Crosstex@crosstex.com

California Facility

14059 Stage Road
Santa Fe Springs, CA 90670
Tel: 562-921-3343
Toll Free: 800-707-2737
Fax: 562-921-8070
Email: Crosstexca@crosstex.com

Colorado Facility

109 Inverness Drive East, Unit F
Englewood, Colorado 80112-5105
Tel: 303-699-3356
Toll Free: 800-819-3336
Fax: 303-699-8255
Email: Crosstexco@crosstex.com

International

European Facility

Sourethweg 11, 6422 PC
Heerlen, The Netherlands
Tel: 31-455-471-451
Fax: 31-455-429-695
Email: Crosstexeurope@crosstex.com

Japan Facility

Kouwa Central Business Bldg #303
3-9-12 Tarumi-Cho Suitea
Osaka, Japan
Tel: 816-6190-2570
Fax: 816-6190-2580
Email: Crosstex@mtb.biglobe.ne.jp

Georgia Facility

621 Hurricane Shoals Road NW Suite G
Lawrenceville, GA 30046
Tel: 770-682-3369
Toll Free: 800-743-3490
Fax: 770-682-3174
Email: Crosstexga@crosstex.com

Pennsylvania Facility

534 Vine Street
Sharon, PA 16146
Tel: 724-347-0400
Toll Free: 800-426-3898
Fax: 724-347-0444
Email: Crosstexpa@crosstex.com

Crosstex/SPSmedical

6789 W. Henrietta Road
Rush, NY 14543
Tel: 585-359-0130
Toll Free: 800-722-1529
Fax: 585-359-0167
Email: info@spsmedical.com
www.spsmedical.com

South American Facility

Dora Fleitas 960
Parque Industrial Luis Guillon
CP-1838, Buenos Aires, Argentina
Tel: 54-11-4284-1700
Fax: 54-11-4284-1799
Email: diego@crosstexargentina.com

Asia Pacific Facility

1A International Business Park #05-01
Singapore 609933
Tel: 65-6227-9698
Fax: 65-6225-6848
Email: Crosstexasia@crosstex.com



Personal Protection



Operatory Protection



Sterilization



Treatment Accessories



Surface Disinfectants



Cleaning Solutions



High-Level Disinfectant



Preventives



Imaging Comfort



Cotton/Non-Wovens



Surgical & Waste Accessories



Waterline Treatment

CROSSTEX® | **PROTECTS®**

© 2015 Crosstex - 5/18/15 - CTX2719

