Pulpdent® Paste

The Original Pre-Mixed Calcium Hydroxide Dressing

Pulpdent Paste has pH>12 and is radiopaque. It is applied directly to the exposed pulp in all pulp capping, pulpotomy and pulpal curettage procedures. Pulpdent Paste stimulates the pulp to form a new dentin bridge which is visible clinically after 2-3 months.

Pulpdent Paste is used routinely in root canal therapy as an intracanal dressing between office visits and to treat periapical lesions, abscesses, root resorption, traumatic injuries and other complicated cases.

INSTRUCTIONS FOR USE

For Vital Pulp Therapy

1. Remove plastic cap. Securely place blunt needle on syringe. You may bend needle to facilitate access to cavity. Alternatively, dispense paste on a pad and place it into the cavity with an appropriate instrument.
2. For vital pulpotomy, place 2–3mm of Pulpdent Paste onto pulp. For pulp capping, place 1–2mm of Pulpdent Paste at exposure site.
3. Dry with stream of air to create a “crust”.
4. Gently tamp Pulpdent Paste with a pledget of cotton to ensure contact with the pulp. Again dry with stream of air.
5. Flow thin mix of hard setting cement over the Pulpdent Paste. Place final restoration.


For Root Repair

1. Fill root canal with Pulpdent Paste using the needles provided, the Pulpdent Pressure Syringe, a lentulo spiral or blunt paper point.
2. Cover with a pledget of cotton and a temporary filling material.
3. Pulpdent Paste can be removed by minimal filing and irrigation and replaced as necessary until the desired results have been achieved. The root canal can then be obturated by an appropriate method.

NOTE: To extend shelf life, place a few drops of water in the cap and recap syringe promptly after each use.

NOTE: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

CLINICAL SITUATIONS WHERE PULPDENT PASTE CAN BE USED

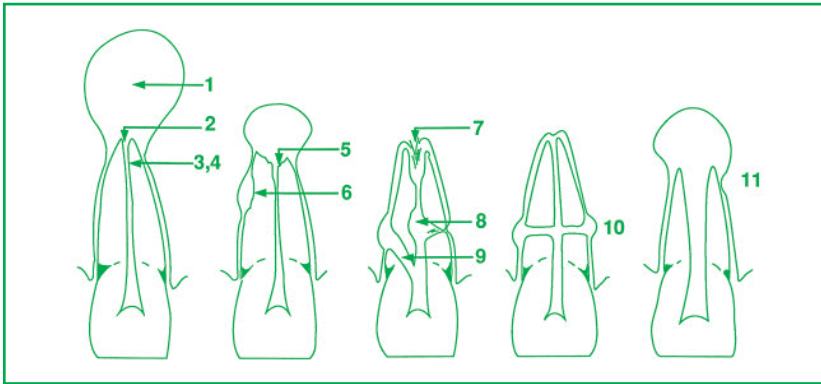- | | |
|---|---|
| 1. exudation control | 8. internal-external root resorption |
| 2. large periapical lesions | 9. root perforations |
| 3. antibacterial dressing | 10. transverse root fractures |
| 4. temporary root filling | 11. incompletely developed pulpless teeth |
| 5. apical inflammatory resorption | (Heithersay, G.S., Calcium Hydroxide in the Treatment of Pulpless Teeth with Associated Pathology, J. Brit Endo Society, 8[2]: 74-93 July 1975) |
| 6. inflammatory resorption following trauma | |
| 7. apical internal resorption | |

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Pulpdent Corporation		Revision Date: October 1, 2016	
SAFETY DATA SHEET			
Trade Name:		Pulpdent Paste	
1.0	Commercial Product Name and Supplier		
1.1	Commercial product name	Pulpdent Paste	
1.2	Application / Use	Dental material for use by dental professionals.	
1.2.2	SIC	851 Human health activity	
1.2.3	Use Category	55	
1.3.	Manufacturer	Pulpdent Corporation 80 Oakland Street, P.O. Box 780, Watertown, MA 02472 USA Telephone: 1 617 926-6666 / Fax: 1 617 926-6262 Email: Pulpdent@pulpdent.com 1-800-535-5053 (24 Hour / USA) Advena Ltd. Pure Offices, Plato Close Warwick CV34 6WE UK	
1.4	Emergency Telephone Number		
1.5	Authorized European Representative		

2.0	Hazards Identification			
2.1	Classification			
2.1.1	Classification according to Regulation (EC) No. 1272/2008 [CLP]	Hazard Class	Hazard Category	Hazard Statement
		Skin irritation	2	H315
		Eye irritation	2	H319
2.1.2	Classification according to Directive 67/548/EEC (See SECTION 16 for full text of risk phrases)			
2.2	GHS Label Elements Hazard Pictograms Signal Word: WARNING Restricted to use by dental professional only. Hazard Statements H319: Eye irritation. 2. May cause eye irritation. H315: Skin irritation. 2. May cause skin irritation. Precautionary Statements P280: Wear protective gloves and eye protection P305+P351: If in eyes, rinse cautiously with water for several minutes. P337+P313: If eye irritation persists, get medical advice/attention. P302+P352: If on skin, wash with plenty of soap and water.			

3.0	Composition			
3.1	Chemical characterization of the preparation			
3.2	Hazardous ingredients			
	CAS Number	Name of the Ingredient	Concentration	Classification according to 67/548/EEC
	1305-62-0	Calcium Hydroxide	39%	Classification according to Regulation (EC) No.1278/2008 (CLP)
				Irritant (Xi) R 36 / 38 Eye irritation; 2

4.0	First Aid Measures	
4.1	Special Instructions	Show this safety data sheet to medical personnel. Get medical attention in case of uncertainty. Calcium hydroxide-based preparations present minimal hazard in the quantities provided. May cause irritation on contact with eyes. Prolonged exposure may cause irritation to mucous membranes, skin, gastrointestinal or respiratory tract.
4.2	Eye Contact	Keep eyelids apart and flush with running water for 15+ minutes. Get medical attention.
4.3	Skin Contact	Remove any contaminated clothing. Wipe off excess from skin. Immediately flush skin with water. Get medical attention if irritation persists.
4.4	Ingestion	Rinse mouth with water. Do not induce vomiting. Give large amounts of water to dilute. Get immediate medical attention for ingestion of large amount. Never give anything by mouth to an unconscious person.
4.5	Inhalation	Move to fresh air. If necessary, administer artificial respiration and seek medical attention.
4.6	Precautions for first responders	Wear safety glasses and gloves to prevent contact.
4.7	Information for physician	
	Symptoms	Irritation, pain or redness in eyes, on skin, on mucous membranes.
	Hazards	May be irritating to eyes, skin, respiratory system, gastrointestinal tract or mucous membranes especially with prolonged exposure. There is a risk of serious damage to eyes.
	Treatment	Same as First Aid above.

5.0	Fire Fighting Measures	
5.1	Suitable extinguishing media	Not a fire hazard. Extinguish fire with agent suitable for surrounding fire.
5.2	Extinguishing media to avoid	None
5.3	Special exposure hazards in a fire	None
5.4	Special protective equipment for fire-fighters	Self-contained breathing apparatus

6.0	Accidental Release Measures	
6.1	Personal precautions	Wear gloves, lab coat and chemical splash goggles.
6.2	Environmental precautions	None
6.3	Method for clean up	Wipe up/absorb with paper towels. Wash area of spill with water-saturated paper towels/cloths. Place spilled material and absorbent towels in container for disposal.

7.0	Handling and Storage	
7.1	Handling	For use by dental professionals only. Wear gloves, safety glasses; wash hands after use. Avoid unnecessary exposure. Follow good hygiene practices. Do not smoke, eat or drink. Wash hands after use.
7.2	Storage	Remove applicator tip immediately after use. Keep tightly capped in original container. Store at cool room temperature. Avoid extremes of temperature (>27°C/80°F, <5°C/40°F) and acids.
7.3	Specific uses	Dental preparation

8.0	Exposure Controls / Personal Protection	
8.1	Exposure limit values	Not available
8.2	Exposure controls	Wear safety glasses, lab coat and gloves. No other equipment required under normal conditions of use of this product in the quantity provided.
8.2.1	Occupational exposure controls	Wear safety glasses, lab coat and gloves.
8.2.1.1	Respiratory protection	Good general ventilation is sufficient to control any airborne vapors.
8.2.1.2	Hand protection	No special requirements other than the usual surgical gloves.
8.2.1.3	Eye protection	No special requirements other than the usual safety glasses.
8.2.1.4	Skin protection	No special requirements. Good personal hygiene, safety practices and wearing a lab coat should protect from unnecessary exposure.

8.2.1.5	Other controls	Emergency eye wash fountain. Wash hands after use.
8.2.2	Environmental exposure controls	Avoid release into environment as may cause pH variation.
9.0	Physical and Chemical Properties	
9.1	Appearance / Color	
9.1.1	Color	White paste
9.1.2	Odor	Odorless
9.2	Important health, safety and environmental information	
9.2.1	pH	pH > 12
9.2.2	Boiling point	Decomposes
9.2.3	Flash point	Not combustible
9.2.4	Flammability (solid, gas)	Not combustible
9.2.5	Explosive properties	Not applicable
9.2.6	Oxidizing properties	Not determined
9.2.7	Vapor pressure	< 1 mm Hg (Id: A)
9.2.8	Specific gravity	Not determined
9.2.9	Solubility in water	Slight
9.2.10	Partition coefficient	Not determined
9.2.11	Viscosity	Not determined
9.2.12	Vapor density	Not determined
9.2.13	Evaporation rate	Not determined

10.0	Stability and reactivity	
10.1	Conditions to avoid	Extremes of temperature (>27°C/80°F, <5°C/40°F).
10.2	Materials to avoid	Calcium hydroxide reacts violently with maleic anhydride, nitro-ethane, nitro-methane, nitro-paraffin, nitro-propane, phosphorus. As a strong alkali, it is incompatible with acids.
10.3	Hazardous decomposition products	Caustic fumes of calcium oxide when heated to decomposition.
10.4	Further information	Stable under normal conditions.

11.0	Toxicological information	
11.1	Acute toxicity	Not toxic. Minimal health hazard under normal conditions of use.
11.2	Irritation and corrosiveness	Minimal health hazard under normal conditions of dental practice. Prolonged exposure may cause irritation of mucous membranes, eyes, skin, gastrointestinal tract or respiratory tract
11.3	Sensitization	Not applicable.
11.4	Sub-acute, sub-chronic and prolonged toxicity	Narrowing of the esophagus may occur weeks to years after ingestion of large quantities, making swallowing difficult. Persons with pre-existing skin conditions may be more susceptible to the irritating effects of calcium hydroxide.
11.5	Carcinogenicity, Mutagenicity, Reproductive Toxicity	Not considered a carcinogen, mutagen or teratogen. Not toxic to the reproductive system.
11.6	Empirical data	Not available
11.7	Clinical Experience	Pulpdent calcium hydroxide preparations have been marketed to and used by dental professionals since 1947. Dentists' experience with thousands of successful procedures is evidence of safe and effective dental treatment.

12.0	Ecological Information	
12.1	Ecotoxicity	Not considered detrimental to water or soil. Avoid release into environment as calcium hydroxide may cause pH variation. Follow government regulations.

13.0	Disposal Considerations	
13.1	Regulations	Follow all local and national government regulations in disposing material or contaminated packaging.
14.0	Transport Information	
14.1	Restrictions	None. Not regulated.
14.2	UN Number	None
14.3	Technical name	Calcium hydroxide preparation
14.4	Packing group	Not applicable
	IATA class	Not applicable
15.0	Regulatory Information	
15.1	EU Regulations	Class IIA medical device under MDD 93/42/EEC.
15.2	US FDA	Class II medical device
15.3	Health Canada	Class III medical device
16.0	Other information	
16.1	List of relevant R phrases	R 36 / 38: Irritating to eyes and skin.
16.2	Hazard Statements	H319: Eye irritation. 2. May cause eye irritation. H315: Skin irritation. 2. May cause skin irritation.
16.3	Precautionary Statements	P280: Wear protective gloves and eye protection P305+P351: If in eyes, rinse cautiously with water for several minutes. P337+P313: If eye irritation persists, get medical advice/attention. P302+P352: If on skin, wash with plenty of soap and water.
16.4	Restrictions on use	Calcium hydroxide preparations are to be used only by dental professionals who have been trained in their use.
16.5	Further information	The information presented herein is believed to be factual as it has been derived from the works of persons believed to be qualified experts. However, nothing contained in this information is to be taken as a warranty or representation for which Pulpdent Corporation bears legal responsibility. The user should review any recommendations in the specific context of the intended use to determine whether they are appropriate.
16.6	Sources of key data	National Institute for Occupational Safety (NIOSH) Occupational Safety and Health Administration (OSHA) Eur-Lex European Union Law: Regulation (EC) No. 1272/2008 (CLP) and Regulation (EC) No. 1907/2006 (REACH). Guidance on the compilation of safety data sheets. Version 1.1; December 2011. European Chemicals Agency
16.7	Information which has been added, deleted or revised	This Safety Data Sheet has been revised to meet the requirements of the GHS SDS format, Regulation (EC) No. 1272/2008 (CLP) and Regulation (EC) No. 1907/2006 (REACH). Specifically, Sections 2.1, 2.2, 3.2, 16.2, 16.3 have been modified.

PULPDENT Corporation

80 Oakland Street • Watertown, MA 02472 • U.S.A.
(617) 926-6666 or (800) 343-4342 • Fax (617) 926-6262
www.pulpdent.com • www.activabioactive.com • pulpdent@pulpdent.com

XP-PSYK-IN-09
REV: 10/2016