

Ketac-Cem® Radiopaque Step-by-step


Consult instructions
prior to use.




**Remove temporary
crown**




**Clean abutment with
pumice and water**

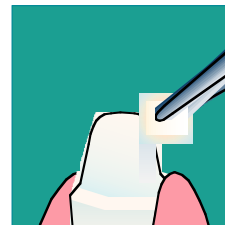

**Dry, moderately and
do not desiccate**

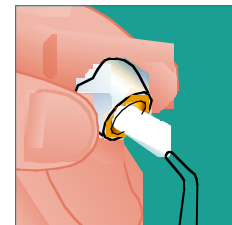


**Clean the crown
with air**

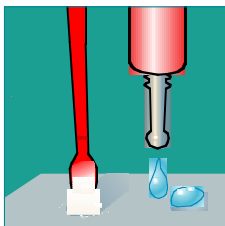


**Ratio: 1 spoon powder
+ 2 drops liquid**




**Take powder with the
spoon. Scrap off excess**

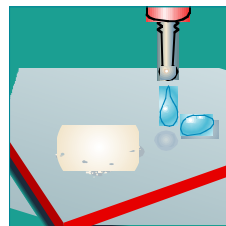


**Put the drops on the
mixing pad**

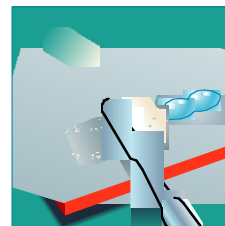

**Mix powder and
liquid**

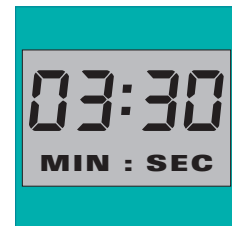


**Mixing and working
time: 3:30 min**

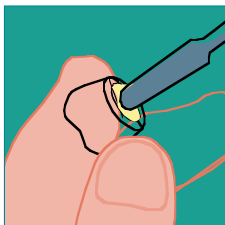


**Brush a thin coat of
cement**




**Place the crown
without pressure**

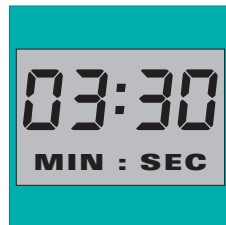

Setting: 3:30 min

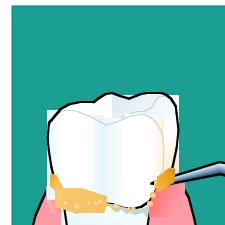

Remove excess

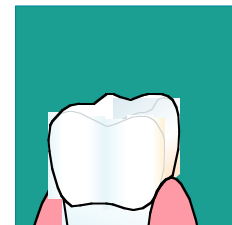

Result