

INSTRUCTIONS FOR USE

Imaging Comfort




BiteWing-Ease® X-Ray Comfort Barriers

How to use your Crosstex® BiteWing-Ease®:



Step 1:

Insert sensor image-side up in BiteWing-Ease.

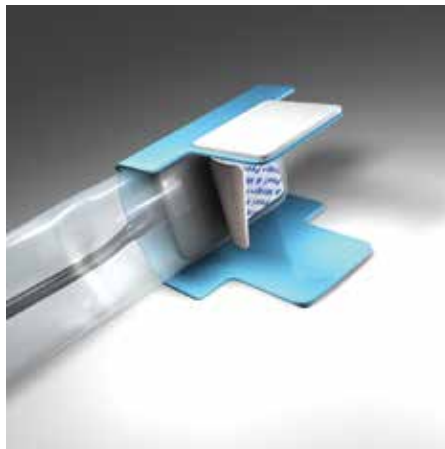

Step 2:

Remove "Peel & Wrap" liner and fold to center to create bite tab.



Step 3:

Take images and discard BiteWing-Ease after use.

Ref.#'s BWEL, BWES

BiteWing-Ease® is a registered trademark of Crosstex International, Inc.



Not made with natural rubber latex.



CROSSTEX®
A CANTEL MEDICAL COMPANY

Manufactured for: Crosstex International, Inc.

10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | Crosstex@crosstex.com

Made in China

BWE-IFU
Rev. 0 – 11/16