


INSTRUCTIONS FOR USE

Imaging Comfort



Slip-Ease® X-Ray Comfort Cushions

How to use your Crosstex® Slip-Ease®:

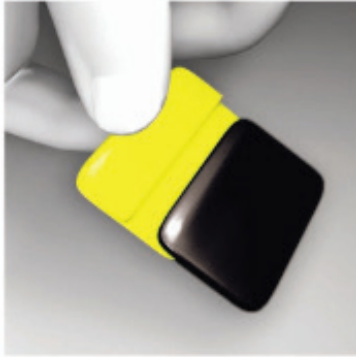

Step 1:

For use with Wireless Sensors or Phosphor Plates: Place wireless sensor or plate in barrier sleeve (not included). Insert sensor into Slip-Ease.



Step 2:

Remove "Peel and Stick" liner from patented safety strip. Press to stick for safety.



Step 3:

Take images and discard Slip-Ease after use.

Ref.#'s BWSEL, BWSES

Slip-Ease® is a registered trademark of Crosstex International, Inc.



Not made with natural rubber latex.



CROSSTEX
A CANTEL MEDICAL COMPANY

Manufactured for: Crosstex International, Inc.
10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | Crosstex@crosstex.com

Made in China

BWSE-IFU
Rev. 0 – 11/16