

INSTRUCTIONS FOR USE

Cleaning Solutions



SANI-Vac® Evacuation System Cleaner

Instructions for use:

1. Add 1 oz. (30mL) of SANI Vac to 100 oz. (3L) of warm water. Mix well.
2. Draw a minimum of 33 oz. (1L) of the prepared solution through the evacuation system tube of each line. The prepared solution services up to 3 lines.
3. Repeat procedure after each patient or at the end of each day to prevent a build-up of organic debris and odors and to ensure optimal performance of the high volume suction.



Storage and Disposal: Store in a cool, dry place away from direct sunlight. Dispose in accordance with local regulations.



Ref.#'s JVAC



WARNING

HAZARD STATEMENTS:

Causes eye irritation. May cause mild skin irritation.

Precautionary Statement(s): Wash hands thoroughly after handling. If skin irritation occurs, get medical attention. IF IN EYES: Rinse continuously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists, get medical attention. IF SWALLOWED: May cause nausea and vomiting. May be irritating to mouth and throat. IF INHALED: Can cause allergic type reaction in susceptible individuals upon repeated exposure.

Refer to Safety Data Sheet (SDS) for additional information. For 24 hour chemical emergency assistance call INFOTRAC: 1-800-535-5053 or 1-352-323-3500.

SANI-Vac® is a registered trademark of Crosstex International, Inc.



CROSSTEX®
A CANTEL MEDICAL COMPANY



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



Made in Canada

Manufactured for: Crosstex International, Inc.
10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | Crosstex@crosstex.com

JVAC-IFU
Rev. 0 – 3/18