

INSTRUCTIONS FOR USE

Preventives



Sparkle® Prophy Angle, EZ Contra Prophy Angle

Instructions for use:

1. Connect this polishing angle to a slow speed handpiece/ attachment. (Optimal speed is 3500 RPM at 30-35 psi. Do not run above 4000 RPM).
2. Seat angle and secure with tightening mechanism.
3. This polishing angle is a single use item. Discard after a single use on one patient.
4. Discard in accordance with local and national regulations.

Single use item, discard after a single use on one patient.
Contamination could occur if reused.



Ref.#'s TPASSC1, TPASFC1, TPASSC5,
TPASFC5, TPATBC1, TPASEZ, TPAFEZ

Sparkle® is a registered trademark of Crosstex International, Inc.



Do not reuse.



Not made with natural rubber latex.



CROSSTEX®
A CANTEL MEDICAL COMPANY

Manufactured for: Crosstex International, Inc.

10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | Crosstex@crosstex.com

Made in China

TPA-IFU
Rev. 0 – 3/17