ProGuard® Varnish

Carefully read instructions prior to use

Product Description:

ProGuard® Varnish is a fluoride containing dental varnish for surface application to enamel and dentin.

- ProGuard Varnish will adhere to wet surfaces and is tolerant to moisture and saliva.
- ProGuard Varnish is an ethanolic suspension of colophony with artificial flavors and sweetened with xylitol.
- 1 ml of ProGuard Varnish contains 50 mg of sodium fluoride equivalent to 22.6 mg of fluoride.

Indications:

ProGuard Varnish is intended for use:

- As a treatment for hypersensitive teeth.
- To seal the dentinal tubules for cavity preparations or on sensitive root surfaces.
- As a cavity liner under amalgam restorations.

Contraindications:

ProGuard Varnish is contraindicated in patients with ulcerative gingivitis or stomatitis or known allergies to colophony.

ProGuard Varnish is not intended for systemic treatment.

Methods of Use:

Recommended dosage:

- Treatment of patients with primary dentition:
 - o Use approx. 0.25 ml of ProGuard Varnish
- Treatment of patients with adolescent or permanent dentition:
 - o Use approx. 0.40 ml of ProGuard Varnish

ProGuard Varnish should be applied on a complete set of dentition in a very thin film. A thick film is more likely to peel off and does not provide enhanced therapeutic results. Unit dosage between the range of 0.25 to 0.40 ml.

Application:

1. Thoroughly clean calculus and biofilm from treatment areas.
2. For best results, eliminate any excessive moisture/saliva from area to be treated.
3. Use brush applicator or foam pellet to uniformly apply a thin film of ProGuard Varnish, covering the entire treatment surface area.
4. Allow area to become wet (either by gentle rinsing or natural salivary flow) to ensure setting of ProGuard Varnish.
5. Instruct patient to avoid hard food, alcohol, brushing or flossing for the next 4 hours after application.

Precautionary Measures:

Edematous swellings have been reported on rare occasions especially following application to extensive surfaces. Direct ingestion should be avoided. Dyspnea associated with asthmatic children, has been reported for similar preparations in extremely rare cases. Patients having sensitive digestive systems may experience nausea, usually after extensive treatment. In case of adverse reactions, remove varnish with a soft toothbrush and floss, and then rinse with warm water.

No treatment with other fluoride preparations on day of application. Suspend treatment with fluoride tablets for several days after treatment. Do not swallow.

According to its purpose, ProGuard Varnish has a sticky consistency.

Storage:

Store at temperatures between 39°F - 73°F (4°C - 23°C). Do not use after expiration date.

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. Crosstex recognizes its responsibility to replace products if proven to be defective. Crosstex does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

Warning: Avoid contact with eyes and skin. If contact occurs, flush immediately with plenty of water and consult a physician. Do not store together with food.

**Keep this material out of reach of children.
For dental use only.**

Manufactured For:

Crosstex
A Cintel Medical Company
10 Ranick Road
Hauppauge, NY 11788

Tel: 631 582 6777
Toll Free: 888 276 7783
Fax: 631 582 1726
Email: crosstex@crosstex.com

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