

September 15, 2020

Subject: Spray bottle refilling instructions for CaviCide™ and CaviCide1™ surface disinfectants

Dear Valued Customers,

This memo is to inform you of the proper process for refilling commercially available sprayer bottles with Metrex™ surface disinfectants and label them correctly. The following wet surface bottle material types are acceptable for refilling:

- polyethylene (HDPE, UHMWPE)
- Polypropylene
- Viton or equivalent (gaskets)
- Glass (ball valves and bottles)
- Styrene
- Silicone rubber (bottles and gaskets)

The container should be labelled only with the information provided on the attached quickcard. Metrex recommends the use of a coarse type sprayer, not to be used on fine-mist sprayer. Never use Metrex surface disinfectants on Foaming, Fogging, Gas, Electrostatic sprayers, or for Vapor applications. Refilled sprayer bottles are **for individual on-site use only** and are not to be used to for the repackaging and sale of Metrex products.

Regards,

Nam Nguyen
Product Manager