

SURFACE DISINFECTION – CAVIWIPES™

SURFACE DISINFECTION USING CAVIWIPES:

Using CaviWipes is an Easy 2 Step Process

STEP 1: CLEAN BEFORE DISINFECTING



A. Dispense CaviWipes towelette.



B. Clean surface with towelette to remove debris and bioburden.



C. Discard used towelette.

To prevent cross contamination, use CaviWipes to clean, disinfect and decontaminate exterior, hard, non-porous surfaces of:

- Infant incubators, cribs and warmers
- Anesthesia machines and respiratory therapy equipment
- Laboratory equipment
- Ambulance equipment
- Tables, chairs and workstations
- IV poles, bed railings and handrails
- Bathroom fixtures
- Telephones

Note: Follow standard precautions for personal protection as appropriate.

STEP 2: DISINFECT AFTER CLEANING



A. Dispense another CaviWipes towelette.



B. Disinfect precleaned surface with towelette. Follow label instructions for appropriate contact times.



C. Discard used towelette.

PART NO. US / Canada	PRODUCT DESCRIPTION	PACKAGING
13-1100 / 11-1100	CaviWipes – 160 wipes per canister	12 canisters/case
13-1150 / 11-1150	CaviWipesXL – 65 wipes per canister	12 canisters/case
13-1155 / 11-1155	CaviWipesXL – 50 wipes per box	6 boxes/case
13-1224 / 11-1224	CaviWipes in a Flat Pack – 45 wipes/pack	20 packs/case
13-1175	CaviWipes Canister Wall Bracket	12 brackets/case
13-1185	Flat Pack Wall Mount	20 packs/case

**CaviWipes
Family**
of surface
disinfectants



Metrex
Protecting People

For more information, visit metrex.com/education, or call 800.841.1428.

Lit #77-1006-6