

Adhesive



Indication

Cementation of veneers using RelyX[™] Veneer Cement or RelyX[™] Ultimate Cement



 Apply a commonly used phosphoric acid etching gel (about 35%), e.g., Scotchbond™ Universal Etchant to the enamel and dentin for 15 seconds.



• Rinse thoroughly with water for 10 seconds.



• Dry with cotton pellets.



Adhesive



 Apply Scotchbond™ Universal Adhesive immediately to the entire surface of the enamel and dentin and rub it in for 20 seconds.

1



• Gently air dry the adhesive for approximately 5 seconds to evaporate the solvent.

_



• Do not light cure!

6



Adhesive





 Apply Scotchbond[™] Universal Adhesive to the hydrofluoric acid etched veneer surface.



• Gently air dry the adhesive for approximately 5 seconds to evaporate the solvent.



• Do not light cure!



Adhesive



Cement the veneer with RelyX[™]
Veneer Cement or RelyX[™] Ultimate
Cement according to the pertinent
instructions for use.

10





- Seat the veneer in place using gentle pressure.
- Spot cure the veneer into place on the facial surface away from the margins using a small diameter light guide for 20 seconds to secure it in place.
- Remove the excess cement from the margins using a blunt instrument or dry brush.

11





 Light cure the gingival edges first for 30 seconds, followed by light curing the middle and incisal edges each for 30 seconds.

- 1



$\textbf{Scotchbond}^{\text{\tiny{TM}}} \ \textbf{Universal}$

Adhesive



Scotchbond™ Universal Adhesive

3M ESPE Customer Care/MSDS Information:

U.S.A. 1-800-634-2249 and Canada 1-888-363-3685

3M, ESPE, Scotchbond™ and RelyX™ are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2012. All rights reserved.

3M ESPE

Dental Products, 2510 Conway Avenue, St. Paul, MN 55144-1000 USA • www.3MESPE.com