

Container Use Directions

ProSpray™

Ready-to-Use Surface Disinfectant/Cleaner

Contents of Shipment: (1) 5 Gallon Drum of ProSpray Ready-to-Use Surface Disinfectant/Cleaner, (1) Spigot
Tools Needed for Installation: Hammer & Phillips Head Screwdriver (#1 or #2)

CAUTION!

Due to the rigidity of the container, the small vent may be difficult to puncture. DO NOT use curved sharp items such as lab knives to puncture.



1. Set container upright with the handle facing the ceiling.



2. Use screwdriver and hammer to puncture the indentation at the center of the large cap. (Do not attempt to remove cap.)



3. The puncture should be large enough to allow liquid to flow after the spigot is installed. If needed, puncture additional holes.



4. With the handle still facing the ceiling, screw the spigot tightly into the large cap.



5. Check to ensure spigot is in the "OFF" position.



6. Position in dispensing location with spigot pointing towards floor. Turn spigot to OPEN position to dispense product.



7. After liquid level in container falls below the small vent cap located above handle, turn spigot to OFF position. Reposition the container upright with handle facing the ceiling. Remove the cap from the vent and puncture a small hole to allow faster dispensing of liquid.

CAUTION! Do not use curved sharp items such as lab knives to puncture.

Reposition the container back in the dispensing location with spigot pointing towards floor.