

points

DISPOSABLE BRUSH APPLICATORS

EINWEG PINSELAPPLIKATOR

PINCEAUX-APPLICATEURS JETABLES

APPLICATORI MONOUSO

MICRO-PINCELES APLICADORES DESECHABLES

BRUSH-APLICADORES DESCARTÁVEIS

JEDNORAZOWE APLIKATORY

DISPOSABLE BORSTEL APPLICATOREN

ENGANGS MIKROBØRSTER

KERTAKÄYTTÖINEN ANNOSTELUTIKKU

APPLISERINGS BØRSTER

ΜΙΑΣ ΧΡΗΣΗΣ ΒΟΥΡΤΣΑΚΙΑ ΕΠΑΛΕΙΨΗΣ

JEDNORÁZOVÉ APLIKAČNÍ ŠTĚTEČKY

JEDNORÁZOVÉ APLIKAČNÉ KEFKY

ELDOBHATÓ APPLICATOR PAMACS

ŠČETKASTI APLIKATORJI ZA ENKRATNO UPORABO




VIENKARINIS ŠEPETĖLIS -APLIKATORIUS

VIENREIZ LIETOJAMIE APLIKATORI

ÜHEKORDSED APLIKATSIOONIHARJAKESED

ポインツ ディスポーザブル・ブラシアプリケーター

拋棄式塗刷

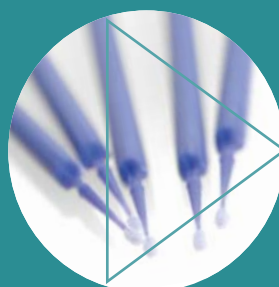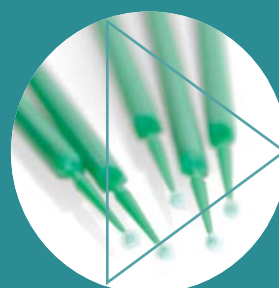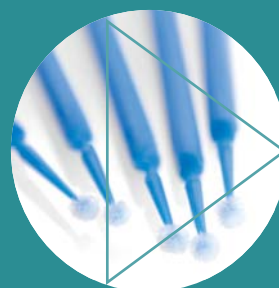

SDI

points



Points disposable brush applicators are used to quickly transfer small amounts of materials into difficult to reach areas. Points are resistant to common solvents in dental materials

Solvent resistant fibers

The fiber tips are resistant to harsh environments and solvents.

Non-absorbent, non-linting fibers

The fibers will not dislodge during the procedure. The solution is suspended in the fibers until applied.

Three sizes

The three sizes of Points, superfine, fine and medium, enable one to pick up different drop sizes.

Choice of colors

The various colors are ideal for color coding multistep procedures.

Soft flexible material


Points can be bent for maximum accessibility.

Indications

- Sealant application
- Placing bonding agents
- Applying etchants
- Placing cavity liners
- Styptic and disclosing solutions
- Mop up excess resin
- Condensing calcium hydroxide in root canals

instructions:

- 1 Points can be used either straight or bent

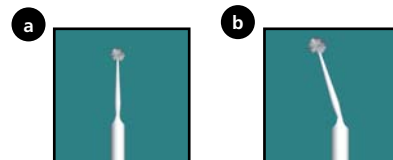

- 2 Dip the Point into the solution and place where required

- 3 Dispose of the Point after the application

Suggestion: When applying more than one solution, use a differently colored Point to avoid confusion



Made in Australia by SDI Limited
 Bayswater, Victoria 3153
 Australia 1 800 337 003
 Austria 00800 022 55734
 Brazil 0800 770 1735
 France 00800 022 55 734
 Germany 0800 100 5759
 Ireland 01 886 9577
 Italy 800 780625
 New Zealand 0800 734 034
 Spain 00800 022 55 734
 United Kingdom 00800 022 55 734
 USA & Canada 1 800 228 5166
www.sdi.com.au

Points purple - superfine
 1 x 400 Points purple - superfine
 Reorder 8100122

Points dark green - fine
 1 x 400 Points dark green - fine
 Reorder 8100116

Points dark blue - medium
 1 x 400 Points dark blue - medium
 Reorder 8100118

Points Dispenser
 Reorder 8100131

Points assortment - all sizes
 1 x 100 Points purple - superfine
 1 x 100 Points dark green - fine
 1 x 100 Points light green - fine
 1 x 100 Points dark blue - medium
 Reorder 8100123

Points light green - fine
 1 x 400 Points light green - fine
 Reorder 8100117

Points light blue - medium
 1 x 400 Points light blue - medium
 Reorder 8100119



M810301 D