SDI

INSTRUCTIONS FOR USE

Riva Star is a new generation desensitiser that immediately relieves tooth sensitivity.

INDICATIONS FOR USE:

Desensitising cervical tooth hypersensitivity.

CONTRA-INDICATIONS

- Do not desensitise teeth with carious lesions.
- Do not use on patients undergoing thyroid gland therapy, who have a history of chemical allergies such as silver, fluoride, potassium, iodine, ammonia and methacrylates.

Note: If in doubt, consult a doctor or an allergy specialist before use.

PROCEDURE:

- Using a non-fluoride prophylaxis paste in a rubber cup, clean the tooth/teeth to be treated and abutting teeth.
- Assess condition of teeth. Only teeth with sound dentine and enamel should be treated.
- Isolate area to be treated with cotton rolls.
- Dry teeth to be treated and adjoining gingival tissues.
- Apply a small amount of Gingival Barrier to the tissues, slightly overlapping the tooth (<1mm) and interproximal spaces.
- Light cure in a **fanning** motion. Note: Do not spot cure Gingival Barrier as this may cause burning of the soft tissues.

Step 1: Using the silver brush provided, pierce through the foil of the silver capsule and carefully apply solution to treatment site only.

Step 2: Immediately after, use green brush provided, pierce through the foil of the green capsule and apply a generous amount of the solution to treatment site. Apply same solution to any other tooth surfaces that may have come into contact with the solution from the silver capsule. Note: Treatment surface initially appears creamy white, keep applying solution from green capsule until it turns clear.

Blot dry, remove gingival barrier.

Discard used capsules and brushes in accordance with local regulations.

DOSAGE:

- Each Step 1 & Step 2 capsules can treat up to 5 teeth per patient.
- Limit of one (1) Step 1 & Step 2 capsule per visit. If needed a second application of Riva Star may be administered after 1 week.

PRECAUTIONS:

- For professional use only.
- Do not repeat treatment on pregnant or lactating women.
- Due to potential staining, it is recommended that anterior teeth not be treated unless diagnosing for caries.
- Patient must be informed of potential staining prior to treatment.
- When used as a desensitizer apply Gingival Barrier before applying Step 1 to avoid soft tissue burn.
- Avoid contact of solution from silver capsule (Step 1) with oral tissue, eyes and skin.
- Avoid contact of solution from green capsule (Step 2) with eyes.
- Do not desensitise teeth with carious lesions, unless a glass ionomer is used to cover over the treatment site.
- Keep out of reach of children.
- Do not ingest.
- Solution from silver capsule (Step 1) will stain (see "HANDLING STAINS").
- Wear gloves and eye protection when handling.
- Single use only.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- Solution from silver capsule (Step 1) contains 0.06g / mL fluoride ions.
- Do not be used with resin based composite restorative material.
- SDS is available at www.sdi.com.au or contact your SDI representative.

STORAGE AND HANDLING:

- Do not pierce foil of silver capsule or green capsule until ready to use.
- Store in a cool and dark place at temperatures between 2° and 8°C (35° - 45°F).
- Use at room temperature between 20° and 25°C (68° - 77°F).

FIRST AID:

- Eyes (contact):** Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre (Australia, 131 126) or a doctor, or for at least 15 minutes.
- Skin (contact):** Flush with running water (an eschar may form on soft tissue, this should disappear within 24-48 hours)
- Ingestion:** Not expected due to dose volume. If affected, do NOT induce vomiting. Give a glass of milk or water.
- Inhalation:** Not expected due to dose volume. If affected, remove from contaminated area.

HANDLING STAINS:

- Skin:** Apply solution from green capsule with gentle rubbing. Stain will slowly disappear. (If nothing is done stain will gradually disappear after a few days).
- Clothing:** Soak in Napisan* (or other fabric stain remover) overnight and wash per normal.
- Hard surfaces:** Immediately apply household bleach (sodium hypochlorite) then wipe off with water. For stubborn stains, saturate a paper napkin with household bleach and place over affected area. Leave overnight and mop up with water.

Napisan* is not the registered trademark of SDI Limited.

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Napisan* is geen geregistreerde handelsmerk van SDI Limited.